
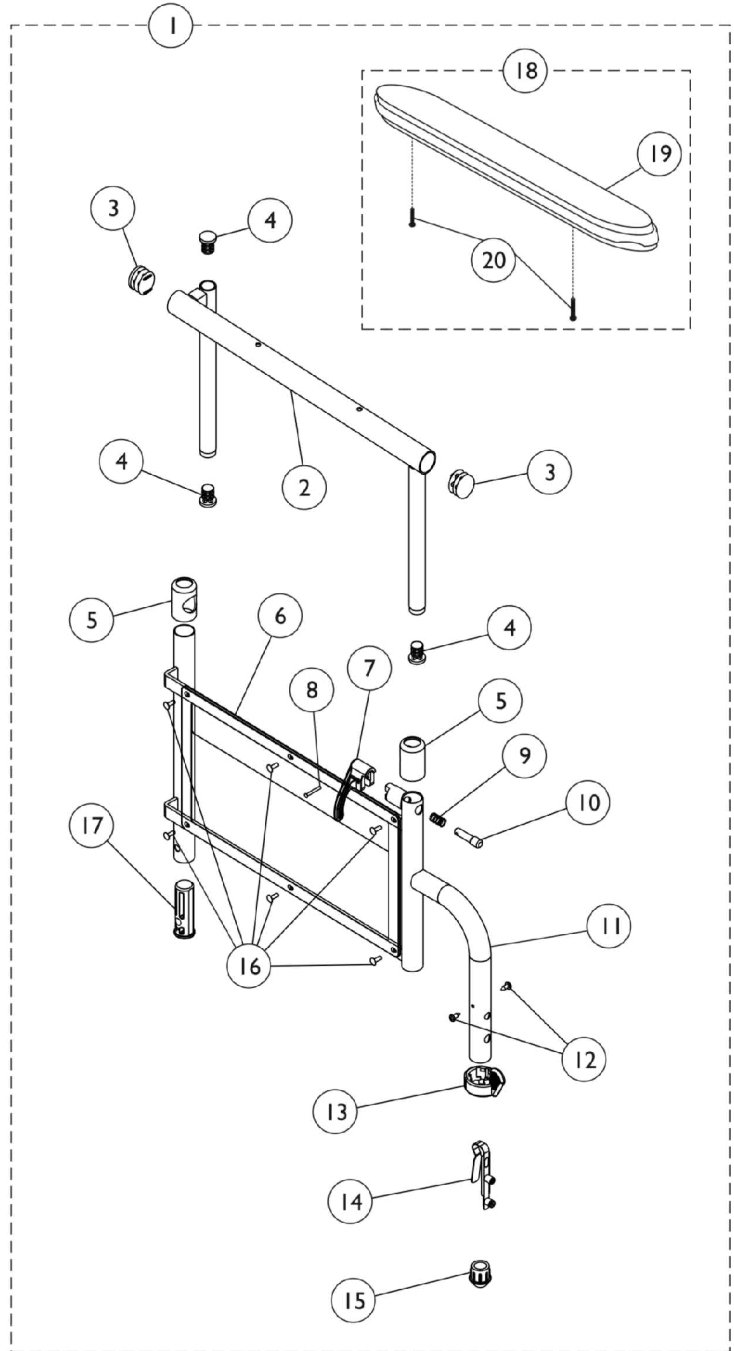
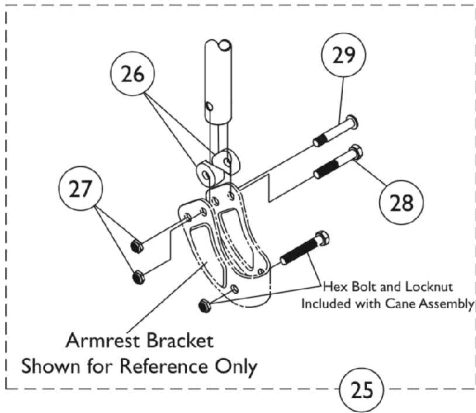


9000SL Adjustable Height Space Saver Arms

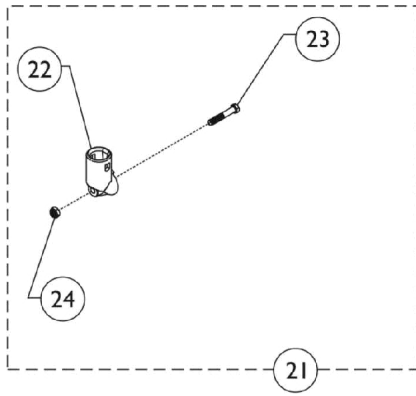

9000SL Adjustable Height Space Saver Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,F	8881130860	Kit, Armrest, Full Length, Black - Right (Style 79)	1	1
2			Tube, Arm, Black - Right	1	1
3			Plug Button, Black (7/8")	1	2
4			Plug Button, Black (5/8")	1	3
5			Anti-Rattle	1	2
6			Guard, Clothing	1	1
7			Handle, Lock, Black - Right	1	1
8			Rivet, Tubular	1	1
9			Spring, Arm Lock	1	1
10			Pin, Locking	1	1
11			Base, Arm, Black - Right	1	1
12			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
13			Handle, Quad Release	1	1
14			Spring, Torsion with Button Lock	1	1
15			Plug, Armrest, Black (3/4") (18GA)	1	1
16			Rivet	1	6
17			Insert, Arm, Black	1	1
19			NLA - Pad, Arm, Full Length w/ Law Label	1	1
20			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	1
6			A,F	1064129	Guard, Clothing
16	1028645	Rivet		1	6
1	8881130861	Kit, Armrest, Full Length, Black - Left (Style 79)		1	1
2		Tube, Arm, Black - Left		1	1
3		Plug Button, Black (7/8")		1	1
4		Plug Button, Black (5/8")		1	1
5		Anti-Rattle		1	2
6		Guard, Clothing		1	1
7		Handle, Lock, Black - Left		1	1
8		Rivet, Tubular		1	1
9		Spring, Arm Lock		1	1
10		Pin, Locking		1	1
11		Base, Arm, Black - Left		1	1
12		Screw, Phillips Pan Head Tap (#6-20 x 3/8")		1	2
13		Handle, Quad Release		1	1
14		Spring, Torsion with Button Lock		1	1
15		Plug, Armrest, Black (3/4") (18GA)		1	1
16		Rivet		1	6
17		Insert, Arm, Black		1	1
19		NLA - Pad, Arm, Full Length w/ Law Label		1	1
20		Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	1	
6		A,F	1064129	Guard, Clothing	1
16	1028645		Rivet	1	6
1	8881135546		Kit, Armrest, Desk Length, Black - Right (Style 89)	1	1
2			Tube, Armrest, Desk, Black - Right	1	1
3			Plug Button, Black (5/8")	1	3
4		Plug Button, Black (7/8")	1	2	
5	Anti-Rattle	1	2		

9000SL Adjustable Height Space Saver Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
6			Guard, Clothing	1	1
7			Handle, Lock, Black - Right	1	1
8			Rivet, Tubular	1	1
9			Spring, Arm Lock	1	1
10			Pin, Locking	1	1
11			Base, Arm, Black - Right	1	1
12			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
13			Handle, Quad Release	1	1
14			Spring, Torsion with Button Lock	1	1
15			Plug, Armrest, Black (3/4") (18GA)	1	1
16			Rivet	1	6
17			Insert, Arm, Black	1	1
19			Arm Pad, Desk Length w/ Law Label	1	1
20			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
6		1064129	Guard, Clothing	1	1
16		1028645	Rivet	1	6
1	A,F	8881135547	Kit, Armrest, Desk Length, Black - Left (Style 89)	1	1
2			Tube, Armrest, Desk, Black - Left	1	1
3			Plug Button, Black (5/8")	1	3
4			Plug Button, Black (7/8")	1	2
5			Anti-Rattle	1	2
6			Guard, Clothing	1	1
7			Handle, Lock, Black - Left	1	1
8			Rivet, Tubular	1	1
9			Spring, Arm Lock	1	1
10			Pin, Locking	1	1
11			Base, Arm, Black - Left	1	1
12			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
13			Handle, Quad Release	1	1
14			Spring, Torsion with Button Lock	1	1
15			Plug, Armrest, Black (3/4") (18GA)	1	1
16			Rivet	1	6
17			Insert, Arm, Black	1	1
19			Arm Pad, Desk Length w/ Law Label	1	1
20			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
6		1064129	Guard, Clothing	1	1
16		1028645	Rivet	1	6
18	B,F	8881127509	Kit, Arm Pad w/Screws - Full Length	1	1
19			NLA - Pad, Arm, Full Length w/ Law Label	1	1
20			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
18	B,F	8881127510	Kit, Arm Pad w/Screws - Desk Length	1	1
19			Arm Pad, Desk Length w/ Law Label	1	1
20			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
21	C	1058139	Kit, Armrest Hardware	1	1
22			Cup, Socket Pivot	1	1
23			NLA - Screw, Hex Head (1/4-20 x 1-3/8")	1	1
24			Locknut (1/4-20)	1	1

9000SL Adjustable Height Space Saver Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
25	D	1091072	Hardware, Armrest	1	1
26			Spacer, Coved (1/4 x 3/4 x 11/32")	1	4
27			Locknut (1/4-20)	1	6
28			Screw, Hex Head (1/4-20 x 1-1/2")	1	4
29			NLA - Screw, Socket Button Head (1/4-20 x 1-7/16")	1	2
	E	7100053	Arm Lock Assembly, Black - Right	1	1
			Handle, Lock, Black - Right	1	1
			Rivet, Tubular	1	2
			Pin, Locking	1	2
			Spring, Arm Lock	1	2
	E	7100054	Arm Lock Assembly, Black - Left	1	1
			Handle, Lock, Black - Left	1	1
			Rivet, Tubular	1	2
			Pin, Locking	1	2
			Spring, Arm Lock	1	2
NOTE: A - Includes items 2-20. B - Includes items 19 and 20. C - Includes items 22-24. D - Includes items 26-28. E - Not Shown. Includes items 7-10. F - Specify upholstery color when ordering. See page titled "Frame Finishes, Upholstery Colors, Decals and Packaging."					