


9000XT & 9000XDT Removable Space Saver Adjustable Height Arms

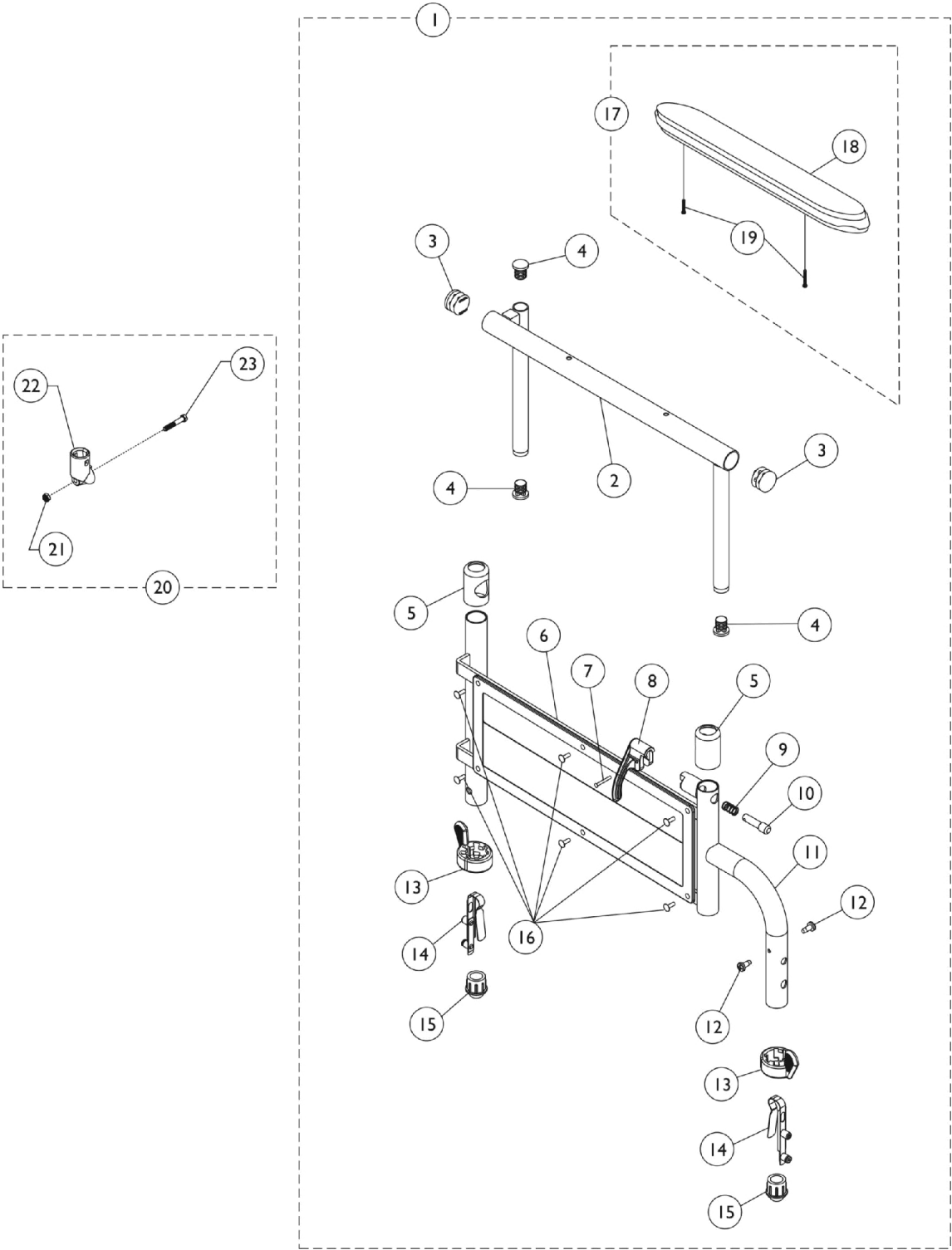

9000XT & 9000XDT Removable Space Saver Adjustable Height Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,D,G	8881130866	Kit, Armrest, Desk Length, Color - Right (Style 89)	1	1
2			Tube, Armrest, Desk, Black - Right	1	1
3			Plug Button, Black (7/8")	1	2
4			Plug Button, Black (5/8")	1	3
5			Anti-Rattle	1	2
6			Guard, Clothing	1	1
7			Rivet, Tubular	1	1
8			Pin, Locking	1	1
9			Spring, Arm Lock	1	1
10			Handle, Lock, Black - Right	1	1
11			Base, Arm, Color - Right	1	1
12			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
13			Handle, Quad Release	1	2
14			Spring, Torsion with Button Lock	1	2
15			Plug, Armrest, Black (3/4") (18GA)	1	1
15			Plug, Armrest, Black (3/4") (20GA)	1	1
16			Rivet	1	6
18			Arm Pad, Desk Length w/ Law Label	1	1
19			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	1
6	A,D,G	1064129	Guard, Clothing	1	1
16		1028645	Rivet	1	6
1		8881130867	Kit, Armrest, Desk Length, Color - Left (Style 89)	1	1
2			Tube, Armrest, Desk, Black - Left	1	1
3			Plug Button, Black (7/8")	1	1
4			Plug Button, Black (5/8")	1	1
5			Anti-Rattle	1	2
6			Guard, Clothing	1	1
7			Rivet, Tubular	1	1
8			Pin, Locking	1	1
9			Spring, Arm Lock	1	1
10			Handle, Lock, Black - Left	1	1
11			Base, Arm, Color - Left	1	1
12			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
13			Handle, Quad Release	1	2
14			Spring, Torsion with Button Lock	1	2
15			Plug, Armrest, Black (3/4") (18GA)	1	1
15			Plug, Armrest, Black (3/4") (20GA)	1	1
16			Rivet	1	6
18			Arm Pad, Desk Length w/ Law Label	1	1
19			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	1
6	A,D,G	1064129	Guard, Clothing	1	1
16		1028645	Rivet	1	6
1		8881135548	Kit, Armrest, Full Length, Color - Right (Style 79)	1	1
2			Tube, Arm, Black - Right	1	1
3			Plug Button, Black (7/8")	1	2
4			Plug Button, Black (5/8")	1	3

9000XT & 9000XDT Removable Space Saver Adjustable Height Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
5			Anti-Rattle	1	2
6			Guard, Clothing	1	1
7			Rivet	1	6
8			Handle, Lock, Black - Right	1	1
9			Spring, Arm Lock	1	1
10			Pin, Locking	1	1
11			Base, Arm, Color - Right	1	1
12			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
13			Handle, Quad Release	1	2
14			Spring, Torsion with Button Lock	1	2
15			Plug, Armrest, Black (3/4") (18GA)	1	1
15			Plug, Armrest, Black (3/4") (20GA)	1	1
16			Rivet, Tubular	1	1
18			NLA - Pad, Arm, Full Length w/ Law Label	1	1
19			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
6		1064129	Guard, Clothing	1	1
16		1028645	Rivet	1	6
1	A,D,G	8881135549	Kit, Armrest, Full Length, Color - Left (Style 79)	1	1
2			Tube, Arm, Black - Left	1	1
3			Plug Button, Black (7/8")	1	2
4			Plug Button, Black (5/8")	1	3
5			Anti-Rattle	1	2
6			Guard, Clothing	1	1
7			Rivet, Tubular	1	1
8			Handle, Lock, Black - Left	1	1
9			Spring, Arm Lock	1	1
10			Pin, Locking	1	1
11			Base, Arm, Color - Left	1	1
12			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
13			Handle, Quad Release	1	2
14			Spring, Torsion with Button Lock	1	2
15			Plug, Armrest, Black (3/4") (18GA)	1	1
15			Plug, Armrest, Black (3/4") (20GA)	1	1
16			Rivet	1	6
18			NLA - Pad, Arm, Full Length w/ Law Label	1	1
19			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
6		1064129	Guard, Clothing	1	1
16		1028645	Rivet	1	6
1	A,F,G	8881135550	Kit, Armrest, Desk Length, Black - Right (Style 89)	1	1
2			Tube, Armrest, Desk, Black - Right	1	1
3			Plug Button, Black (7/8")	1	2
4			Plug Button, Black (5/8")	1	3
5			Anti-Rattle	1	2
6			Guard, Clothing	1	1
7			Rivet, Tubular	1	1
8			Handle, Lock, Black - Right	1	1

9000XT & 9000XDT Removable Space Saver Adjustable Height Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
9			Spring, Arm Lock	1	1
10			Pin, Locking	1	1
11			Base, Arm, Chrome - Right	1	1
12			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
13			Handle, Quad Release	1	2
14			Spring, Torsion with Button Lock	1	2
15			Plug, Armrest, Black (3/4") (18GA)	1	1
16			Plug, Armrest, Black (3/4") (20GA)	1	1
17			Rivet	1	6
18			Arm Pad, Desk Length w/ Law Label	1	1
19			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
6		1064129	Guard, Clothing	1	1
16		1028645	Rivet	1	6
1	A,F,G	8881135551	Kit, Armrest, Desk Length, Black - Left (Style 89)	1	1
2			Tube, Armrest, Desk, Black - Left	1	1
3			Plug Button, Black (5/8")	1	3
4			Plug Button, Black (7/8")	1	2
5			Anti-Rattle	1	2
6			Guard, Clothing	1	1
7			Rivet, Tubular	1	1
8			Handle, Lock, Black - Left	1	1
9			Spring, Arm Lock	1	1
10			Pin, Locking	1	1
11			Base, Arm, Chrome - Left	1	1
12			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
13			Handle, Quad Release	1	2
14			Spring, Torsion with Button Lock	1	2
15			Plug, Armrest, Black (3/4") (18GA)	1	1
16			Plug, Armrest, Black (3/4") (20GA)	1	1
17			Rivet	1	6
18			Arm Pad, Desk Length w/ Law Label	1	1
19			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
6		1064129	Guard, Clothing	1	1
16		1028645	Rivet	1	6
1	A,F,G	8881130870	Kit, Armrest, Full Length, Chrome - Right (Style 79)	1	1
2			NLA - Tube, Arm, Full Length, Chrome - Right	1	1
3			Plug Button, Black (7/8")	1	2
4			Plug Button, Black (5/8")	1	3
5			Anti-Rattle	1	2
6			Guard, Clothing	1	1
7			Rivet, Tubular	1	1
8			Handle, Lock, Black - Right	1	1
9			Spring, Arm Lock	1	1
10			Pin, Locking	1	1
11			Base, Arm, Chrome - Right	1	1
12			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2

9000XT & 9000XDT Removable Space Saver Adjustable Height Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
13			Handle, Quad Release	1	2
14			Spring, Torsion with Button Lock	1	2
15			Plug, Armrest, Black (3/4") (18GA)	1	1
15			Plug, Armrest, Black (3/4") (20GA)	1	1
16			Rivet	1	6
18			NLA - Pad, Arm, Full Length w/ Law Label	1	1
19			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	1
6		1064129	Guard, Clothing	1	1
16		1028645	Rivet	1	6
1	A,F,G	8881130871	Kit, Armrest, Full Length, Chrome - Left (Style 79)	1	1
2			NLA - Tube, Arm, Full Length, Chrome - Left	1	1
3			Plug Button, Black (5/8")	1	3
4			Plug Button, Black (7/8")	1	2
5			Anti-Rattle	1	2
6			Guard, Clothing	1	1
7			Rivet, Tubular	1	1
8			Handle, Lock, Black - Left	1	1
9			Spring, Arm Lock	1	1
10			Pin, Locking	1	1
11			Base, Arm, Chrome - Left	1	1
12			Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
13			Handle, Quad Release	1	2
14			Spring, Torsion with Button Lock	1	2
15			Plug, Armrest, Black (3/4") (18GA)	1	1
15			Plug, Armrest, Black (3/4") (20GA)	1	1
16			Rivet	1	6
18			NLA - Pad, Arm, Full Length w/ Law Label	1	1
19			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	1
6		1064129	Guard, Clothing	1	1
16		1028645	Rivet	1	6
17	B,F	8881127509	Kit, Arm Pad w/Screws - Full Length	1	1
18			NLA - Pad, Arm, Full Length w/ Law Label	1	1
19			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
17	B,F	8881127510	Kit, Arm Pad w/Screws - Desk Length	1	1
18			Arm Pad, Desk Length w/ Law Label	1	1
19			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
20	C	1058139	Kit, Armrest Hardware	1	1
21			Locknut (1/4-20)	1	1
22			Cup, Socket Pivot	1	1
23			NLA - Screw, Hex Head (1/4-20 x 1-3/8")	1	1
	E	7100053	Arm Lock Assembly, Black - Right	1	1
			Handle, Lock, Black - Right	1	1
			Rivet, Tubular	1	2
			Pin, Locking	1	2
			Spring, Arm Lock	1	2
	E	7100054	Arm Lock Assembly, Black - Left	1	1

9000XT & 9000XDT Removable Space Saver Adjustable Height Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
			Handle, Lock, Black - Left	1	1
			Rivet, Tubular	1	2
			Pin, Locking	1	2
			Spring, Arm Lock	1	2
NOTE: A - Includes items 2-19. B - Includes items 18 and 19. C - Includes items 21-23. D - Specify frame finish and upholstery color when ordering. See page titled "Frame Finishes, Upholstery Colors, Decals and Packaging." E - Not shown. Includes items 7-10. F - Specify upholstery color when ordering. See page titled "Frame Finishes, Upholstery Colors, Decals and Packaging." G - Item #15 Plug Button for 18GA Tubing used in front only.					