


9000XT & 9000XDT Non-Removable Space Saver Adjustable Height Arms

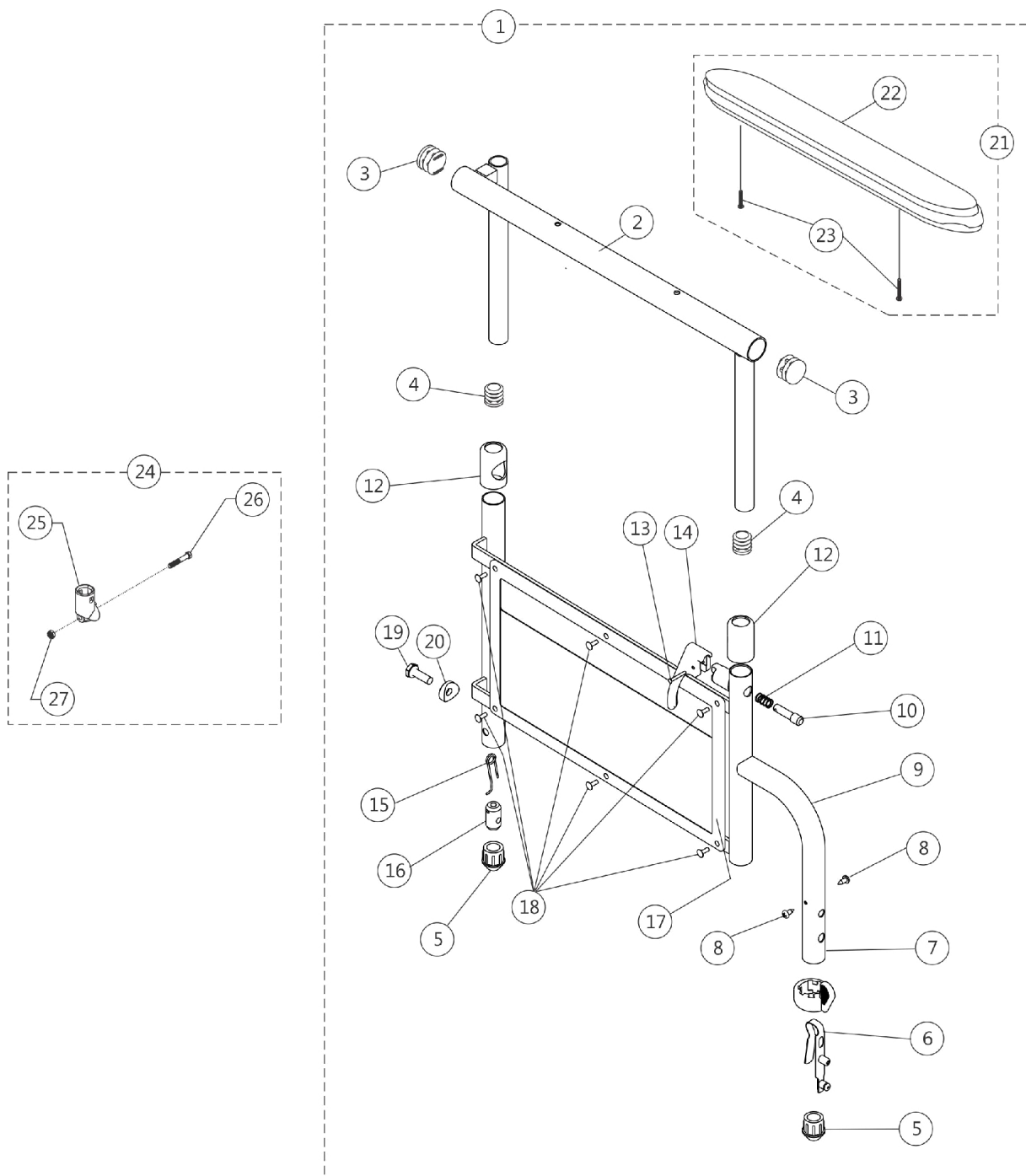

9000XT & 9000XDT Non-Removable Space Saver Adjustable Height Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,F,G	8881130856-NL A	NLA - Kit, Armrest, Desk Length, Chrome - Right (Style 89)	1	1
2			NLA - Tube, Arm, Desk Length, Chrome - Right	1	1
3			Plug Button, Black (7/8")	1	2
4			Plug Button, Black (5/8")	1	3
5			Plug, Armrest, Black (3/4") (18GA)	1	1
5			Plug, Armrest, Black (3/4") (20GA)	1	1
6			Spring, Torsion with Button Lock	1	1
7			Handle, Quad Release	1	1
8			Screw, Self Tapping (#6-20 x 1/2")	1	2
9			Base, Arm, Chrome - Right	1	1
10			Pin, Locking	1	1
11			Spring, Arm Lock	1	1
12			Anti-Rattle	1	2
13			Rivet, Tubular	1	1
14			Handle, Lock, Black - Right	1	2
15			Spring, Release	1	1
16			Insert, Threaded	1	1
17			Guard, Clothing	1	1
18			Rivet	1	6
19			NLA - Bolt, Hex Head Cap (1/4-20 x 3/4")	1	1
20			Spacer, Coved (1/4 x 11/16 x 1/2")	1	1
22			Arm Pad, Desk Length w/ Law Label	1	1
23			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	1
17	G	1064129	Guard, Clothing	1	1
18		1028645	Rivet	1	6
1		8881130857-NL A	NLA - Kit, Armrest, Desk Length, Chrome - Left (Style 89)	1	1
2			NLA - Tube, Arm, Desk Length, Chrome - Left	1	1
3			Plug Button, Black (7/8")	1	2
4			Plug Button, Black (5/8")	1	3
5			Plug, Armrest, Black (3/4") (18GA)	1	1
5			Plug, Armrest, Black (3/4") (20GA)	1	1
6			Spring, Torsion with Button Lock	1	1
7			Handle, Quad Release	1	1
8			Screw, Self Tapping (#6-20 x 1/2")	1	2
9			Base, Arm, Chrome - Left	1	1
10			Pin, Locking	1	1
11			Spring, Arm Lock	1	1
12			Anti-Rattle	1	2
13			Rivet, Tubular	1	1
14			Handle, Lock, Black - Left	1	1
15			Spring, Release	1	1
16			Insert, Threaded	1	1
17			Guard, Clothing	1	1
18			Rivet	1	6

9000XT & 9000XDT Non-Removable Space Saver Adjustable Height Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
19			NLA - Bolt, Hex Head Cap (1/4-20 x 3/4")	1	1
20			Spacer, Coved (1/4 x 11/16 x 1/2")	1	1
22			Arm Pad, Desk Length w/ Law Label	1	1
23			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	1
17		1064129	Guard, Clothing	1	1
18		1028645	Rivet	1	6
1	A,F,G	8881135538-NL A	NLA - Kit, Armrest, Full Length, Chrome - Right (Style 79)	1	1
2			NLA - Tube, Arm, Full Length, Chrome - Right	1	1
3			Plug Button, Black (5/8")	1	3
4			Plug Button, Black (7/8")	1	2
5			Plug, Armrest, Black (3/4") (18GA)	1	1
5			Plug, Armrest, Black (3/4") (20GA)	1	1
6			Spring, Torsion with Button Lock	1	1
7			Handle, Quad Release	1	1
8			Screw, Self Tapping (#6-20 x 1/2")	1	2
9			Base, Arm, Chrome - Right	1	1
10			Pin, Locking	1	1
11			Spring, Arm Lock	1	1
12			Anti-Rattle	1	2
13			Rivet, Tubular	1	1
14			Handle, Lock, Black - Right	1	1
15			Spring, Release	1	1
16			Insert, Threaded	1	1
17			Guard, Clothing	1	1
18			Rivet	1	6
19			NLA - Bolt, Hex Head Cap (1/4-20 x 3/4")	1	1
20			Spacer, Coved (1/4 x 11/16 x 1/2")	1	1
22			NLA - Pad, Arm, Full Length w/ Law Label	1	1
23			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
17		1064129	Guard, Clothing	1	1
18		1028645	Rivet	1	6
1	A,F,G	8881135539-NL A	NLA - Kit, Armrest, Full Length, Chrome - Left (Style 79)	1	1
2			NLA - Tube, Arm, Full Length, Chrome - Left	1	1
3			Plug Button, Black (5/8")	1	3
4			Plug Button, Black (7/8")	1	2
5			Plug, Armrest, Black (3/4") (18GA)	1	1
5			Plug, Armrest, Black (3/4") (20GA)	1	1
6			Spring, Torsion with Button Lock	1	1
7			Handle, Quad Release	1	1
8			Screw, Self Tapping (#6-20 x 1/2")	1	2
9			Base, Arm, Chrome - Left	1	1
10			Pin, Locking	1	1
11			Spring, Arm Lock	1	1
12			Anti-Rattle	1	2

9000XT & 9000XDT Non-Removable Space Saver Adjustable Height Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
13			Rivet, Tubular	1	1
14			Handle, Lock, Black - Left	1	1
15			Spring, Release	1	1
16			Insert, Threaded	1	1
17			Guard, Clothing	1	1
18			Rivet	1	6
19			NLA - Bolt, Hex Head Cap (1/4-20 x 3/4")	1	1
20			Spacer, Coved (1/4 x 11/16 x 1/2")	1	1
22			NLA - Pad, Arm, Full Length w/ Law Label	1	1
23			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
17		1064129	Guard, Clothing	1	1
18		1028645	Rivet	1	6
1	A,D,G	8881135536	Kit, Armrest, Desk Length, Color - Right (Style 89)	1	1
2			Tube, Armrest, Desk, Black - Right	1	1
3			Plug Button, Black (5/8")	1	3
4			Plug Button, Black (7/8")	1	2
5			Plug, Armrest, Black (3/4") (18GA)	1	1
5			Plug, Armrest, Black (3/4") (20GA)	1	1
6			Spring, Torsion with Button Lock	1	1
7			Handle, Quad Release	1	1
8			Screw, Self Tapping (#6-20 x 1/2")	1	2
9			Base, Arm, Color - Right	1	1
10			Pin, Locking	1	1
11			Spring, Arm Lock	1	1
12			Anti-Rattle	1	1
13			Rivet, Tubular	1	1
14			Handle, Lock, Black - Right	1	2
15			Spring, Release	1	1
16			Insert, Threaded	1	1
17			Guard, Clothing	1	1
18			Rivet	1	6
19			NLA - Bolt, Hex Head Cap (1/4-20 x 3/4")	1	1
20			Spacer, Coved (1/4 x 11/16 x 1/2")	1	1
22			Arm Pad, Desk Length w/ Law Label	1	1
23			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
17		1064129	Guard, Clothing	1	1
18		1028645	Rivet	1	6
1	A,D,G	8881135537	Kit, Armrest, Desk Length, Color - Left (Style 89)	1	1
2			Tube, Armrest, Desk, Black - Left	1	1
3			Plug Button, Black (5/8")	1	3
4			Plug Button, Black (7/8")	1	2
5			Plug, Armrest, Black (3/4") (18GA)	1	1
5			Plug, Armrest, Black (3/4") (20GA)	1	1
6			Spring, Torsion with Button Lock	1	1
7			Handle, Quad Release	1	1
8			Screw, Self Tapping (#6-20 x 1/2")	1	2

9000XT & 9000XDT Non-Removable Space Saver Adjustable Height Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
9			Base, Arm, Color - Left	1	1
10			Pin, Locking	1	1
11			Spring, Arm Lock	1	1
12			Anti-Rattle	1	2
13			Rivet, Tubular	1	1
14			Handle, Lock, Black - Left	1	1
15			Spring, Release	1	1
16			Insert, Threaded	1	1
17			Guard, Clothing	1	1
18			Rivet	1	6
19			NLA - Bolt, Hex Head Cap (1/4-20 x 3/4")	1	1
2			Spacer, Coved (1/4 x 11/16 x 1/2")	1	1
22			Arm Pad, Desk Length w/ Law Label	1	1
23			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
17		1064129	Guard, Clothing	1	1
18		1028645	Rivet	1	6
1	A,D,G	8881135552	Kit, Armrest, Full Length, Color - Right (Style 79)	1	1
2			Tube, Arm, Black - Right	1	1
3			Plug Button, Black (7/8")	1	2
4			Plug Button, Black (5/8")	1	3
5			Plug, Armrest, Black (3/4") (18GA)	1	1
5			Plug, Armrest, Black (3/4") (20GA)	1	1
6			Spring, Torsion with Button Lock	1	1
7			Handle, Quad Release	1	1
8			Screw, Self Tapping (#6-20 x 1/2")	1	2
9			Base, Arm, Color - Right	1	1
10			Pin, Locking	1	1
11			Spring, Arm Lock	1	1
12			Anti-Rattle	1	2
13			Rivet, Tubular	1	1
14			Handle, Lock, Black - Right	1	1
15			Spring, Release	1	1
16			Insert, Threaded	1	1
17			Guard, Clothing	1	1
18			Rivet	1	6
19			NLA - Bolt, Hex Head Cap (1/4-20 x 3/4")	1	1
20			Spacer, Coved (1/4 x 11/16 x 1/2")	1	1
22			NLA - Pad, Arm, Full Length w/ Law Label	1	1
23			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
17		1064129	Guard, Clothing	1	1
18		1028645	Rivet	1	6
1	A,D,G	8881135553	Kit, Armrest, Full Length, Color - Left (Style 79)	1	1
2			Tube, Arm, Black - Left	1	1
3			Plug Button, Black (7/8")	1	2
4			Plug Button, Black (5/8")	1	3
5			Plug, Armrest, Black (3/4") (18GA)	1	1

9000XT & 9000XDT Non-Removable Space Saver Adjustable Height Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
5			Plug, Armrest, Black (3/4") (20GA)	1	1
6			Spring, Torsion with Button Lock	1	1
7			Handle, Quad Release	1	1
8			Screw, Self Tapping (#6-20 x 1/2")	1	2
9			Base, Arm, Color - Left	1	1
10			Pin, Locking	1	1
11			Spring, Arm Lock	1	1
12			Anti-Rattle	1	2
13			Rivet, Tubular	1	1
14			Handle, Lock, Black - Left	1	1
15			Spring, Release	1	1
16			Insert, Threaded	1	1
17			Guard, Clothing	1	1
18			Rivet	1	6
19			NLA - Bolt, Hex Head Cap (1/4-20 x 3/4")	1	1
20			Spacer, Coved (1/4 x 11/16 x 1/2")	1	1
22			NLA - Pad, Arm, Full Length w/ Law Label	1	1
23			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
17		1064129	Guard, Clothing	1	1
18		1028645	Rivet	1	6
21	B,F	8881127509	Kit, Arm Pad w/Screws - Full Length	1	1
22			NLA - Pad, Arm, Full Length w/ Law Label	1	1
23			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
21	B,F	8881127510	Kit, Arm Pad w/Screws - Desk Length	1	1
22			Arm Pad, Desk Length w/ Law Label	1	1
23			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2
24	C	1058139	Kit, Armrest Hardware	1	1
25			Cup, Socket Pivot	1	1
26			NLA - Screw, Hex Head (1/4-20 x 1-3/8")	1	1
27			Locknut (1/4-20)	1	1
	E	7100053	Arm Lock Assembly, Black - Right	1	1
			Handle, Lock, Black - Right	1	1
			Rivet, Tubular	1	2
			Pin, Locking	1	2
			Spring, Arm Lock	1	2
	E	7100054	Arm Lock Assembly, Black - Left	1	1
			Handle, Lock, Black - Left	1	1
			Rivet, Tubular	1	2
			Pin, Locking	1	2
			Spring, Arm Lock	1	2

9000XT & 9000XDT Non-Removable Space Saver Adjustable Height Arms

NOTE: A - Includes items 2-23.
B - Includes items 22 and 23.
C - Includes items 25-27.
D - Specify frame finish and upholstery color when ordering. See category titled, "Frame Finishes, Upholstery Colors, Decals and Packaging."
E - Not shown. Includes items 10, 11, 13 and 14.
F - Specify upholstery color when ordering. See category titled, "Frame Finishes, Upholstery Colors, Decals and Packaging."
G - Item #5 Plug Button for 18GA Tubing used in front only.