



Space Saver Fixed Height Arms - Removable/Non-Removable

REMOVABLE

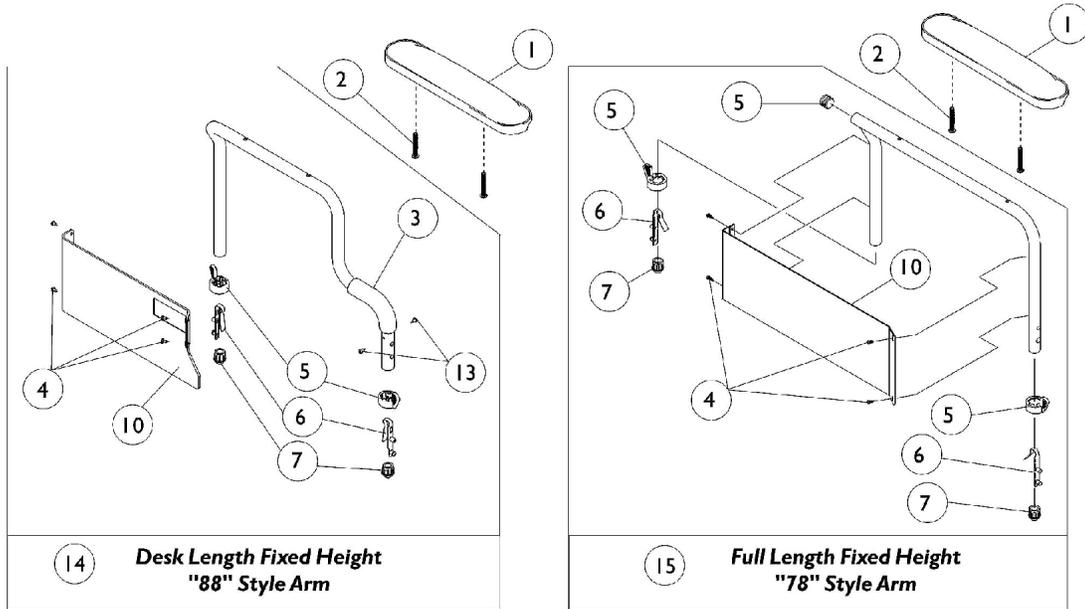

NON-REMOVABLE

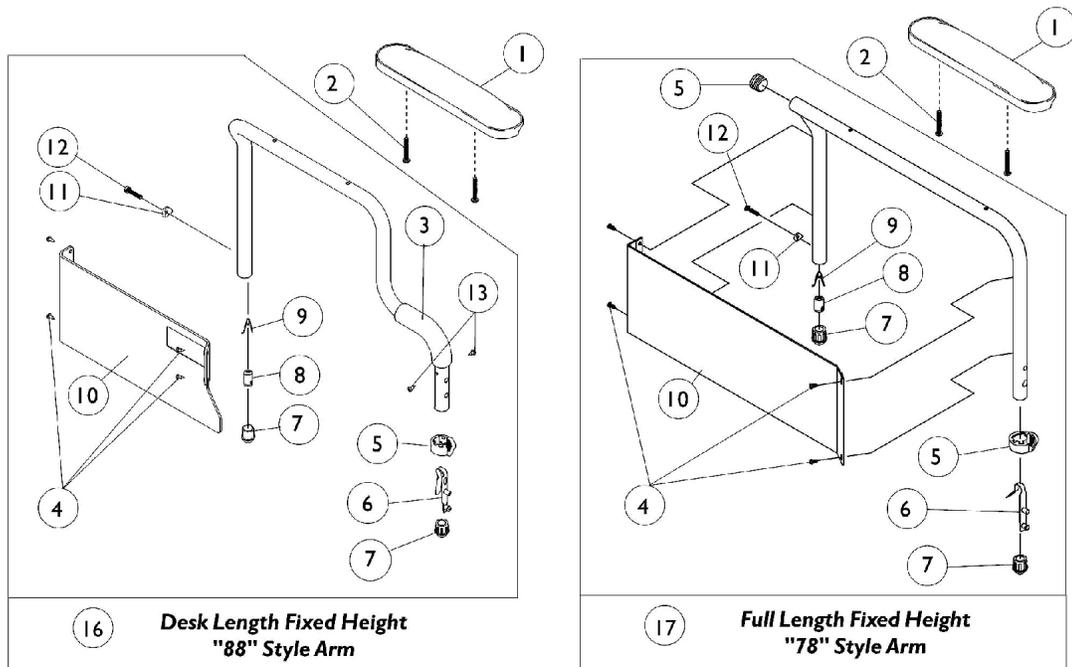

Space Saver Fixed Height Arms - Removable/Non-Removable

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881053570	Kit, Arm Pad w/ Screws, Desk Length (Pair)	2	1
1	A	8881053571	Kit, Arm Pad w/ Screws, Full Length (Pair)	2	1
1	G	1117327	Kit, Arm Pad w/ Screws, Desk Length, Urethane Waterfall, Black (9" L x 2-5/8" W)	1	1
1	G	1117328	Kit, Arm Pad w/ Screws, Full Length, Urethane Waterfall, Black (13" L x 2-5/8" W)	1	1
1	G	1110817	Kit, Arm Pad w/ Screws, Desk Length, Urethane Waterfall, Black (9" L x 2-5/8" W) (Pair)	2	1
1	G	1110818	Kit, Arm Pad w/ Screws, Full Length, Urethane Waterfall, Black (13" L x 2-5/8" W) (Pair)	2	1
2		1034083-NLA	NLA - Package, Screw Phillips Round Head (#10-32 x 1-1/8")	2	2
3		1037719	Grip, Hand, Foam (6 1/2")	1	1
4		1116900	Screw, Self Tapping (#6-20 x 1/2")	1	4
5		1015450	Handle, Quad Release	1	2
6		1018225	Spring, Torsion with Button Lock	1	2
7		1036484	Plug, Armrest, Black (3/4") (18GA)	1	2
8		1005834	Insert, Threaded	1	1
9		40188X002	Spring, Release	1	1
10	B	1123307	Kit, Clothing Guard, Space Saver, Desk Length w/ Screws and Decals (Pair)	2	1
10	B	1123308	Kit, Clothing Guard, Space Saver, Full Length w/ Screws & Decals (Pair)	2	1
11		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	1
12		1025708-NLA	NLA - Bolt, Hex Head Cap (1/4-20 x 3/4")	1	1
13		1020022	Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	2
14	C,F,H	8881056986	Arm Assembly, Space-Saver Desk Length w/o Arm Pad, Color - Right	1	1
14	C,F,H	8881056987	Arm Assembly, Space-Saver Desk Length w/o Arm Pad, Color - Left	1	1
15	F	8881057980	Arm Assembly, Space-Saver, Full Length w/o Arm Pad, Color - Right	1	1
15	F	8881057981	Arm Assembly, Space-Saver, Full Length w/o Arm Pad, Color - Left	1	1
16	F	8881058076	Arm Assembly, Non-Rem. Space-Saver Desk Length w/o Arm Pad, Color - Right	1	1
16	F	8881058077	Arm Assembly, Non-Rem. Space-Saver Desk Length w/o Arm Pad, Color - Left	1	1
17	F	8881058084	Arm Assembly, Non-Rem. Space-Saver Full Length w/o Arm Pad, Color - Right	1	1
17	F	8881058085	Arm Assembly, Non-Rem. Space-Saver Full Length w/o Arm Pad, Color - Left	1	1
	D,E	1040328-NLA	NLA - Hardware, Non-Removable, Space Saver Arm	2	1

Space Saver Fixed Height Arms - Removable/Non-Removable

NOTE: A - Specify upholstery color, see page titled "Upholstery Color Options" when ordering.
B - Clothing Guards not sold individually. Kit includes clothing guards, screws, and decals.
C - Due to One Piece Fabrication Process, order Right and Left Arm Assemblies in pairs, for chairs manufactured before 8-16-01.
D - Includes items 3-13
E - Not Shown
F - Due to Clothing Guard change, it is recommend (for chairs built Before 11/25/03) when only one Arm is requested the appropriate Clothing Guard Kit (item 10) be ordered also so that the arms will match.
G - Urethane Waterfall Armpads were changed from Upholstered Waterfall Armpads on or around 01/03/2002.
H - Must order the Hand Grip separately.

78 & 88 Style Arms have a height range of 10".