


Ventilator Tray and Battery Box

Ventilator Tray accommodates 16" and wider chairs with a seat to floor height of 18" or higher. Ventilator Tray is not offered on chairs with a HDPG Heavy Duty Package.

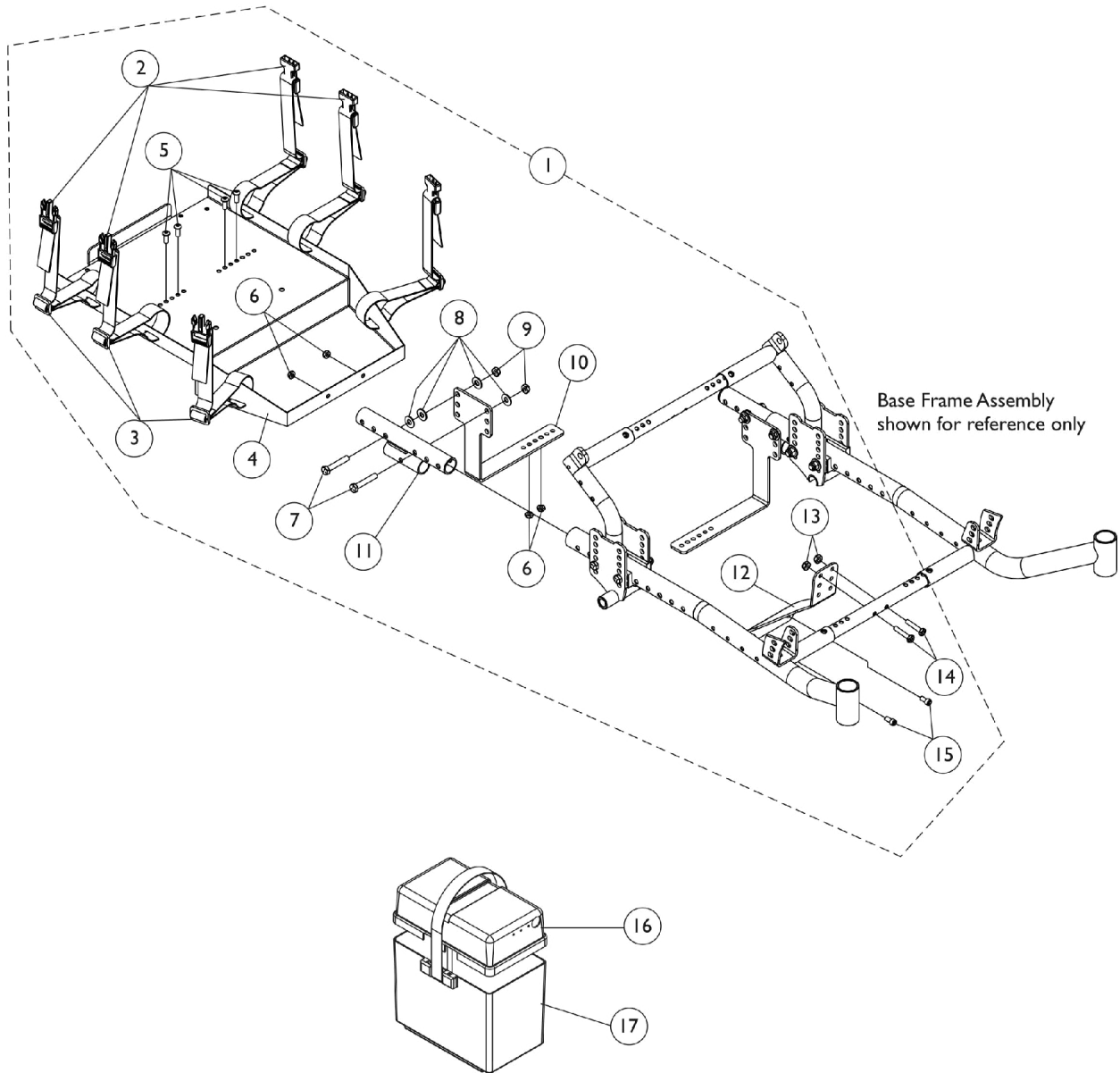

Ventilator Tray and Battery Box

Ventilator Tray accommodates 16" and wider chairs with a seat to floor height of 18" or higher. Ventilator Tray is not offered on chairs with a HDPG Heavy Duty Package.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1164327	Kit, Ventilator Tray, Solara 3G	1	1
2			NLA - Buckle, Slide Release	1	6
3			NLA - Strap with Slide Release Buckle (22" Long)	1	6
4			NLA - Tray, Ventilator	1	1
5			Screw, Socket Button Head (1/4-28 x 5/8")	1	4
6			Locknut (1/4-28)	1	6
7			NLA - Screw, Hex Head (5/16-18 x 1-3/4")	1	4
8			Washer (5/16 x 3/4 x 1/16")	1	8
9			Locknut (5/16-18)	1	4
10			Bracket, Vent Tray, Black	1	2
11			Tube, Vent Tray Support, Black	1	2
12			Bracket, Vent Tray, Front, Black	1	1
13			Locknut, Thin (1/4-28)	1	2
14			Screw, Socket Head (1/4-28 x 1-1/4")	1	2
15			Screw, Socket Head (1/4-28 x 1/2")	1	2
			Instruction Sheet, Solara 3G Vent Tray	1	1
			Instruction Sheet, FC, Solara 3G Vent Tray	1	1
16		1095747	Top, Battery Box	1	1
17		1057944	Battery Box Bottom with Strap, 22NF	1	1

NOTE: A - Includes items 2-15 and Instruction Sheets for U.S. and French Canadian