



Power Unit

Turn-Q Plus

Right End Panel

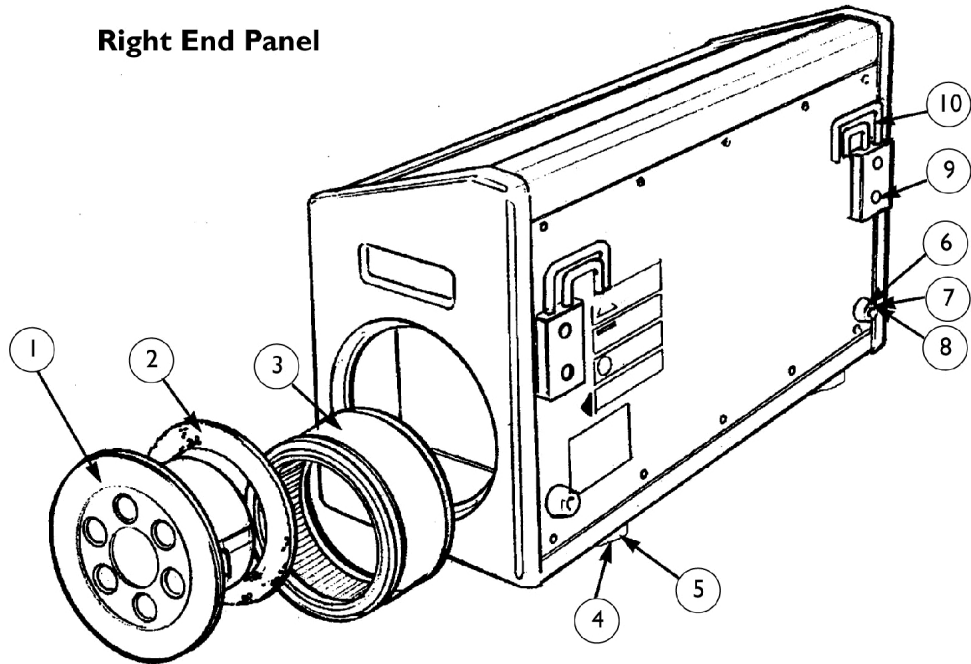

Left End Panel

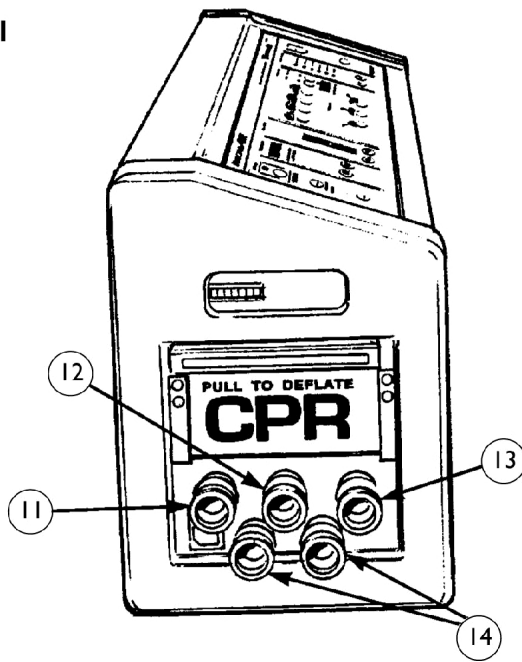

Power Unit Turn-Q Plus

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		9610045-NLA	NLA - Filter Door Assembly (Includes: Door, Spring, Muffler, Gasket and Screws)	1	1
2		9610035-NLA	NLA - Gasket, Filter	1	1
3		9172069-NLA	NLA - Filter, Air	10	1
4		1101057-NLA	NLA - Bumper, Rubber, Grey	1	4
5		1101058-NLA	NLA - Screw (1/4-20 x 5/8")	1	4
6		686-NLA	NLA - Bumper, Composite, Black	1	2
7		393-NLA	NLA - Screw (#8-32 x 1/2")	1	2
8		654-NLA	NLA - Washer, Lock (#10)	1	2
9		805-NLA	NLA - Screw (1/4-20 x 3/4")	1	4
10	A	9382029-NLA	NLA - Hook, Interchangeable	1	2
11		9510100-NLA	NLA - Coupler, Quick Release, Green	1	1
12		9510099-NLA	NLA - Coupler, Quick Release, Blue	1	1
13		9510098-NLA	NLA - Coupler, Quick Release, Red	1	1
14		9510097-NLA	NLA - Coupler, Quick Release, Yellow	1	2
	B	BB9612014	NLA - Unit, Power, Turn-Q Plus	1	1

NOTE: A - See page titled "Miscellaneous Accessories" page for Hook Replacement Kit.
B - Not Shown.