




# Axles - Axle Mounting Brackets and Hardware

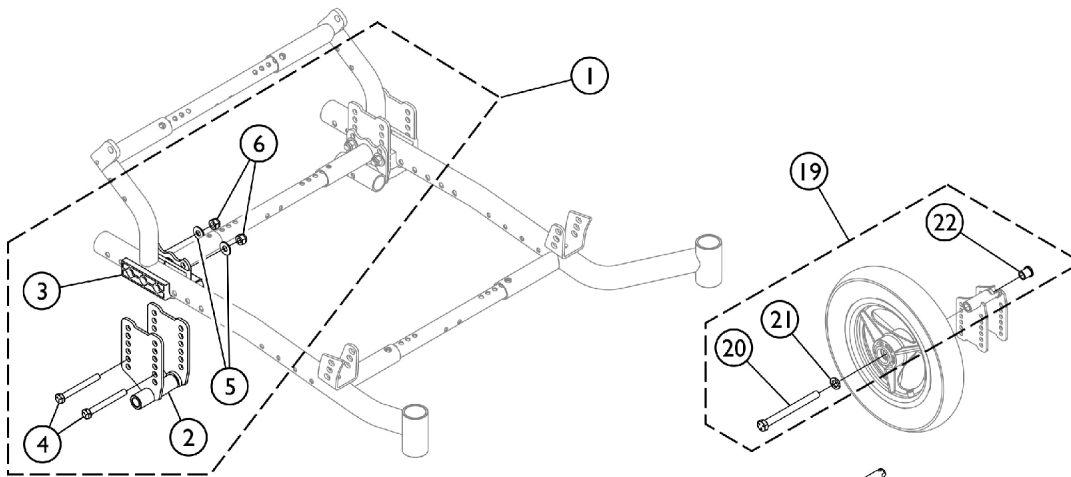

**Axle Brackets And Hardware  
For Chairs Without Hublocks  
And Chairs Without  
Heavy Duty Package**

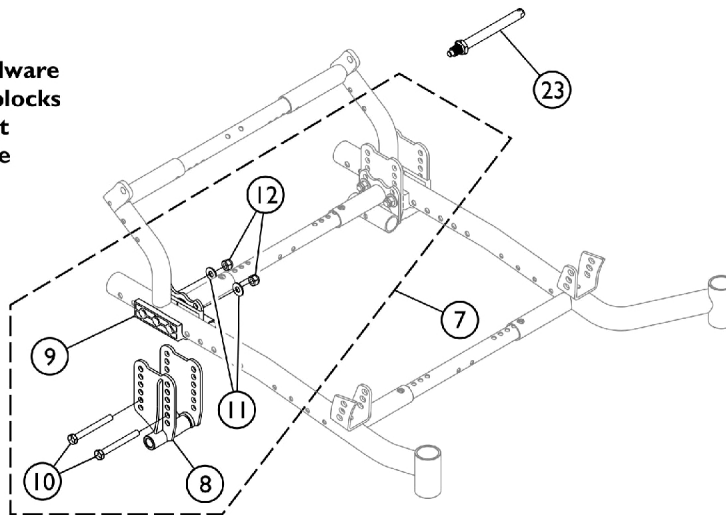

**Axle Brackets And Hardware  
For Chairs With Heavy Duty Package**

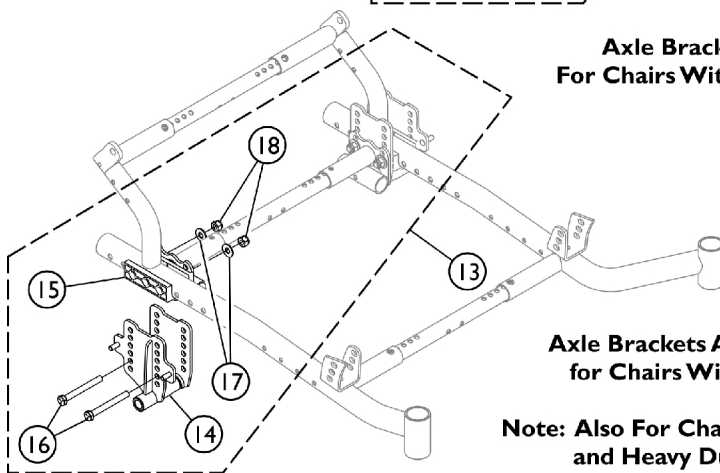

**Axle Brackets And Hardware  
for Chairs With Hublocks**

**Note: Also For Chairs With Hublocks  
and Heavy Duty Package**

## Axles - Axle Mounting Brackets and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1154002	Kit, Axle Brackets w/ Hardware, Non-Heavy Duty (Pair)	1	1
2			Bracket, Axle, Black - Left	1	1
2			Bracket, Axle, Black - Right	1	1
3			Bracket, Axle Spacer, Black	1	2
4			Screw, Hex Head (5/16-24 x 2-3/4")	1	4
5			Washer (5/16 x 3/4 x 1/16")	1	4
6			Locknut (5/16-24)	1	4
7	B	1159505	Kit, Axle Brackets w/ Hardware, Heavy Duty (Pair)	1	1
8			Bracket, Axle, Black - Left	1	1
8			Bracket, Axle, Black - Right	1	1
9			Bracket, Axle Spacer, Black	1	2
10			Screw, Hex Head (5/16-24 x 2-3/4")	1	4
11			Washer (5/16 x 3/4 x 1/16")	1	4
12			Locknut (5/16-24)	1	4
13	C	1162955	Kit, Axle Brackets w/ Hardware, Hublock (Pair)	1	1
14			Bracket, Axle, Black - Left	1	1
14			Bracket, Axle, Black - Right	1	1
15			Bracket, Axle Spacer, Black	1	2
16			Screw, Hex Head (5/16-24 x 2-3/4")	1	4
17			Washer (5/16 x 3/4 x 1/16")	1	4
18			Locknut (5/16-24)	1	4
19	D,E	1162958	Kit, Axle, Permanent w/ Hardware (Pair)	1	1
20			Screw, Hex Head (1/2-20 x 5-1/4")	1	2
21			Washer, Lock (1/2")	1	2
22			Nut, Barrel (1/2-20)	1	2
23		1040361	Axle, Quick Release w/ Locknut (1/2" x 5")	1	2

NOTE: A - Includes items 2-6  
 B - Includes items 8-12  
 C - Includes items 14-18. Hardware also used for chairs that have Hublocks AND Heavy Duty Package  
 D - Includes items 20-22  
 E - Add Loctite 242 to Item 22 at assembly.