


Seat Pan and Attaching Hardware (After 4/14/97)

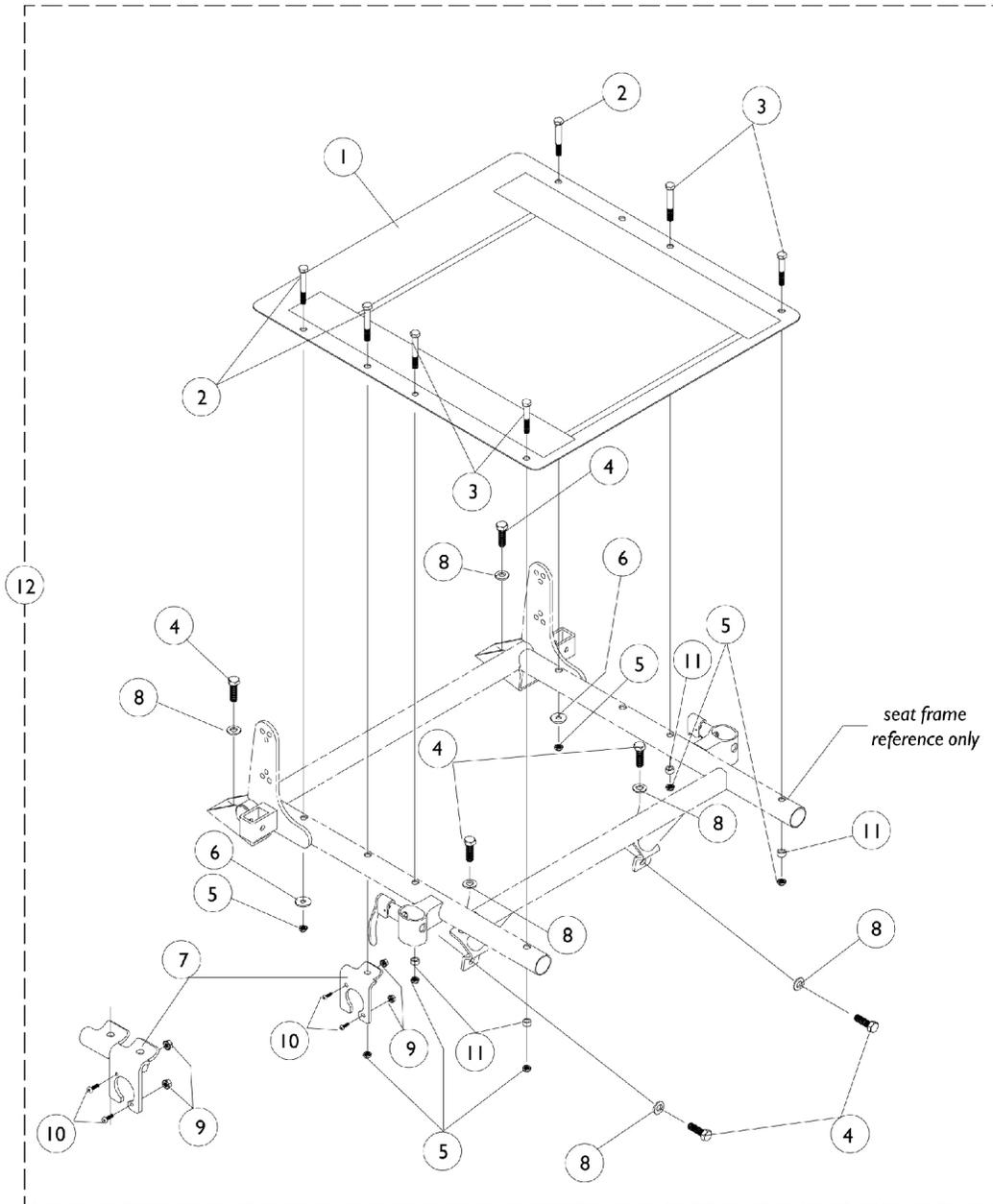

Seat Pan and Attaching Hardware (After 4/14/97)

Item # 1 - Seat Pan

Width	16" Deep	17" Deep	18" Deep	19" Deep	20" Deep	21" Deep	22" Deep
12" W	1066009-NLA	N/A	N/A	N/A	N/A	N/A	N/A
13" W	1066010-NLA	N/A	N/A	N/A	N/A	N/A	N/A
14" W	1066011-NLA	1066022-NLA	1066033-NLA	1066044-NLA	1066055-NLA	1066066-NLA	1066077-NLA
15" W	1066012-NLA	1066023-NLA	1066034-NLA	1066045-NLA	1066056-NLA	1066067-NLA	1066078-NLA
16" W	1066013-NLA	1066024	1066035-NLA	1066046-NLA	1066057-NLA	1066068-NLA	1066079-NLA
17" W	1066014-NLA	1066025-NLA	1066036-NLA	1066047-NLA	1066058-NLA	1066069-NLA	1066080-NLA
18" W	1066015-NLA	1066026	1066037-NLA	1066048-NLA	1066059-NLA	1066070-NLA	1066081-NLA
19" W	1066016-NLA	1066027-NLA	1066038-NLA	1066049-NLA	1066060	1066071-NLA	1066082-NLA
20" W	1066017-NLA	1066028-NLA	1066039-NLA	1066050-NLA	1066061-NLA	1066072-NLA	1066083-NLA
21" W	1066018-NLA	1066029-NLA	1066040-NLA	1066051-NLA	1066062-NLA	1066073-NLA	1066084-NLA
22" W	1066019-NLA	1066030-NLA	1066041-NLA	1066052-NLA	1066063-NLA	1066074-NLA	1066085-NLA
23" W	1066020-NLA	1066031-NLA	1066042-NLA	1066053-NLA	1066064-NLA	1066075-NLA	1066086-NLA
24" W	1066021-NLA	1066032-NLA	1066043-NLA	1066054-NLA	1066065-NLA	1066076-NLA	1066087-NLA

NOTE: For chair depths of 12, 13, 14, or 15" the 16" deep seat pan is used.

Seat Pan and Attaching Hardware (After 4/14/97)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		See Chart	# 1 Seat Pan (After 4/14/97)	1	1
2		1024793-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	3
3		1030949	Screw, Hex Head (1/4-20 x 1-1/2")	1	4
4		1039860	Screw, Hex Head w/ Patch (5/16-18 x 3/4")	1	6
5		7930333	Locknut (1/4-20)	25	6
6		1026900	Washer (1/4 x 23/32 x 1/16")	1	2
7		1053837-NLA	NLA - Bracket, Charger Connector, Black	1	1
7	A,A1	1073872-NLA	NLA - Bracket, Charger Connector, Black	1	1
8		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	6
9		P0870X000	Nut, Hex Keps (#4-40)	1	2
10		7930318-NLA	NLA - Package, Screw, Phillips Pan Head (#4-40 x 1/2")	25	2
11		1047449-NLA	NLA - Spacer (1/4 x 3/8 x 3/16")	1	4
12	B	1068439-NLA	NLA - Hardware, Seat Mounting	1	1
12	A,A1,B	1070567-NLA	NLA - Hardware, Seat Mounting	1	1

NOTE: A - When all four of the following parameters (14" - 15" wide x 18" deep, low seat-to-floor and group 24 batteries) are configured together on a chair, then part numbers noted with an (A) will be used.
A1 - All other configurations will use #1053837 or #1068439.
B - Includes items 2 thru 11.