
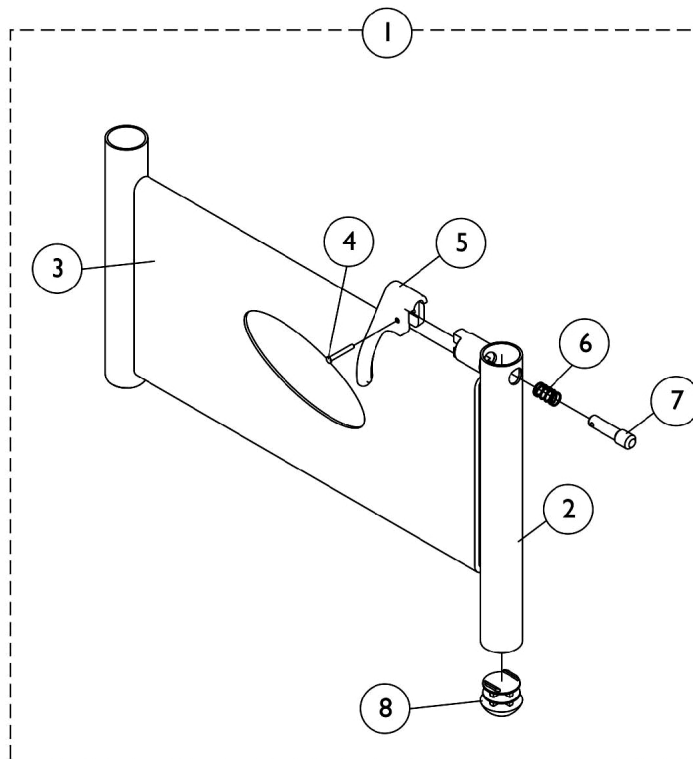
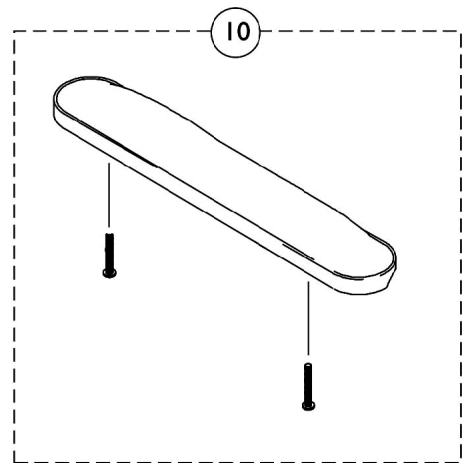


Adjustable Height Arms

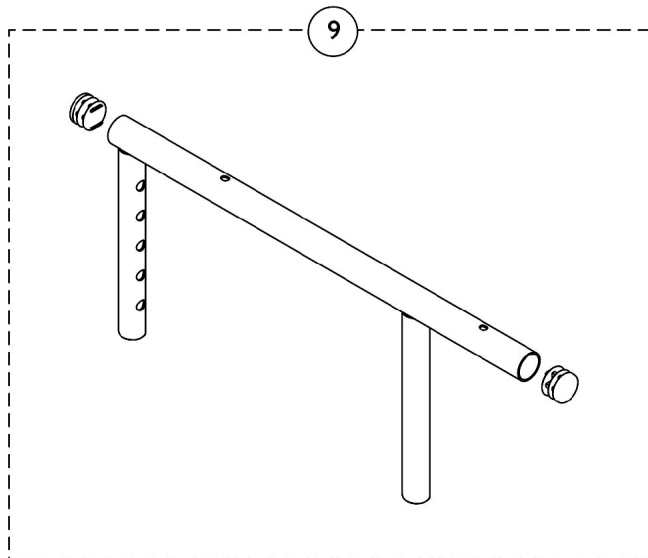

Adjustable Height Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1136591	Arm Base Assembly, Black, FADJDESK, Full - Right (Standard)	1	1
2			Base, Arm, Black - Right/Left	1	1
3			Cover, Clothing Guard w/ Pocket, FXDESK & ADJFULL/ADJDESK	1	1
4			Pin, Arm Lock, Black	1	1
5			Handle, Lock, Black - Right	1	1
6			Spring, Arm Lock	1	1
7			Rivet, Tubular	1	1
8			Plug Button, Black (5/8")	1	1
1	A	1136592	Arm Base Assembly, Black, FADJDESK, Full - Left (Standard)	1	1
2			Base, Arm, Black - Right/Left	1	1
3			Cover, Clothing Guard w/ Pocket, FXDESK & ADJFULL/ADJDESK	1	1
4			Pin, Arm Lock, Black	1	1
5			Handle, Lock, Black - Left	1	1
6			Spring, Arm Lock	1	1
7			Rivet, Tubular	1	1
8			Plug Button, Black (5/8")	1	1
9		1165476	Arm Tube Assembly w/ Plug Buttons, Desk Length, Black - Right/Left	1	1
9		1165478	Arm Tube Assembly w/ Plug Buttons, Full Length, Black - Right/Left	1	1
10		8881053577	Kit, Arm Pad w/ Screws, Full Length (14" L x 2" W) (Pair)	2	1
10		8881053578	Kit, Arm Pad w/ Screws, Desk Length (9" L x 2" W) (Pair)	2	1

NOTE: A - Includes items 2-8.