



Tilt Hardware - Foot Pedal Tilt Release Lockout Assembly


If a chair is ordered as a Foot Release or a Hand Release, it CANNOT be retrofitted in the field to accept the other Tilting function. A new wheelchair must be ordered.

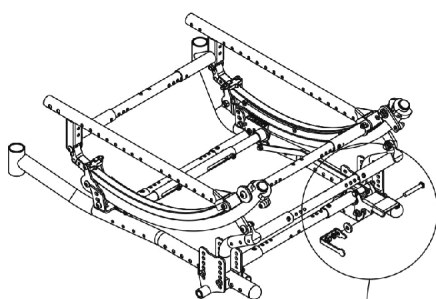


**Base And Seat Frame Assembly
Shown For Reference Only**

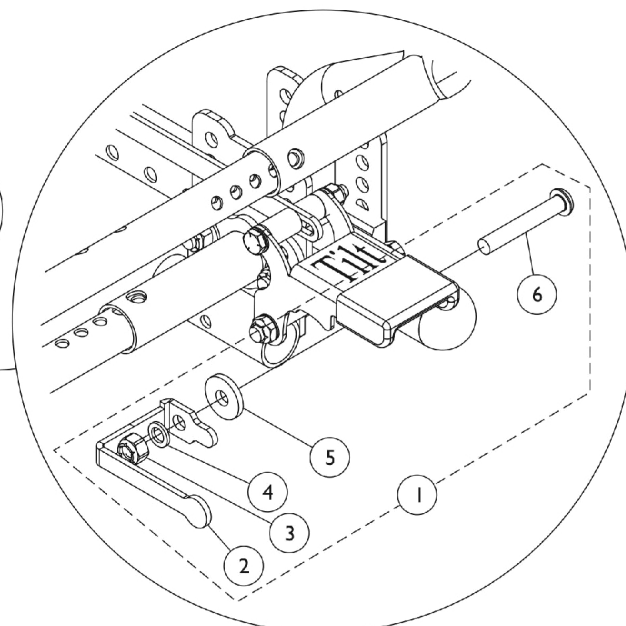

Foot Pedal Tilt Release Lockout Assembly w/o Vent Tray Option

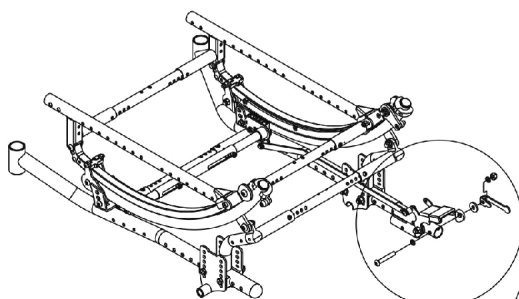

Right Side Shown

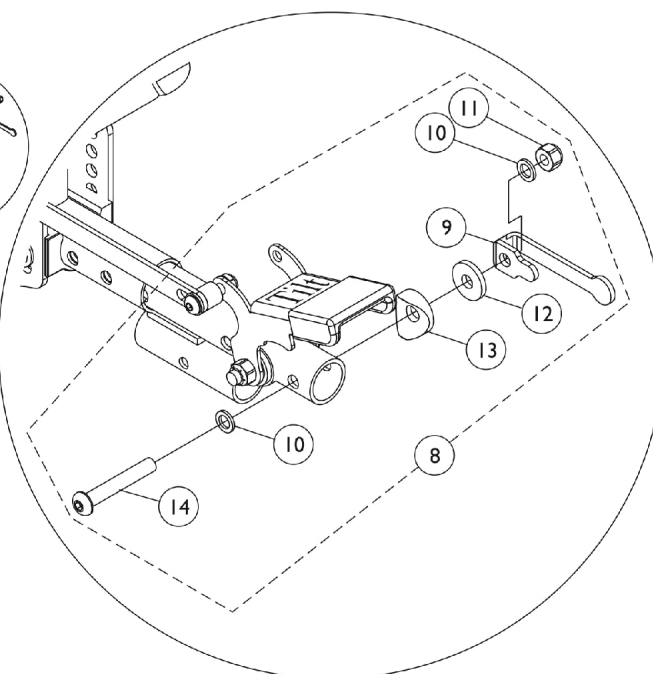

Foot Pedal Tilt Release Lockout Assembly w/ Vent Tray Option

Tilt Hardware - Foot Pedal Tilt Release Lockout Assembly

If a chair is ordered as a Foot Release or a Hand Release, it CANNOT be retrofitted in the field to accept the other Tilting function. A new wheelchair must be ordered.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1163916	Foot Pedal Tilt Lockout Assembly w/o Vent	1	1
2			Bracket, Foot Pedal Tilt Release Lockout, Black	1	1
3			Locknut (5/16-18)	1	1
4			NLA - Washer, Nylon (5/16 x 1/2 x 1/16")	1	1
5			Washer, Nylon (5/16 x 7/8 x 1/8")	1	1
6			Screw, Socket Button Head (5/16-18 x 2")	1	1
7	B	1163917	Foot Pedal Tilt Lockout Assembly w/ Vent	1	1
8			Bracket, Foot Pedal Tilt Release Lockout w/ Vent, Black	1	1
9			NLA - Washer, Nylon (5/16 x 1/2 x 1/16")	1	2
10			Locknut (5/16-18)	1	1
11			Washer, Nylon (5/16 x 7/8 x 1/8")	1	1
12			NLA - Spacer, Coved (3/8 x 1 x 9/16")	1	1
13			Screw, Socket Button Head (5/16-18 x 2")	1	1
NOTE: A - Includes items 2 thru 6. B - Includes items 8 thru 13.					