


# Walker (Single Release)

With 3" Fixed Wheel and Glide Tips  
Models 6281-3F (Discontinued 6/07)

Model 6281-3F (4 Pack)

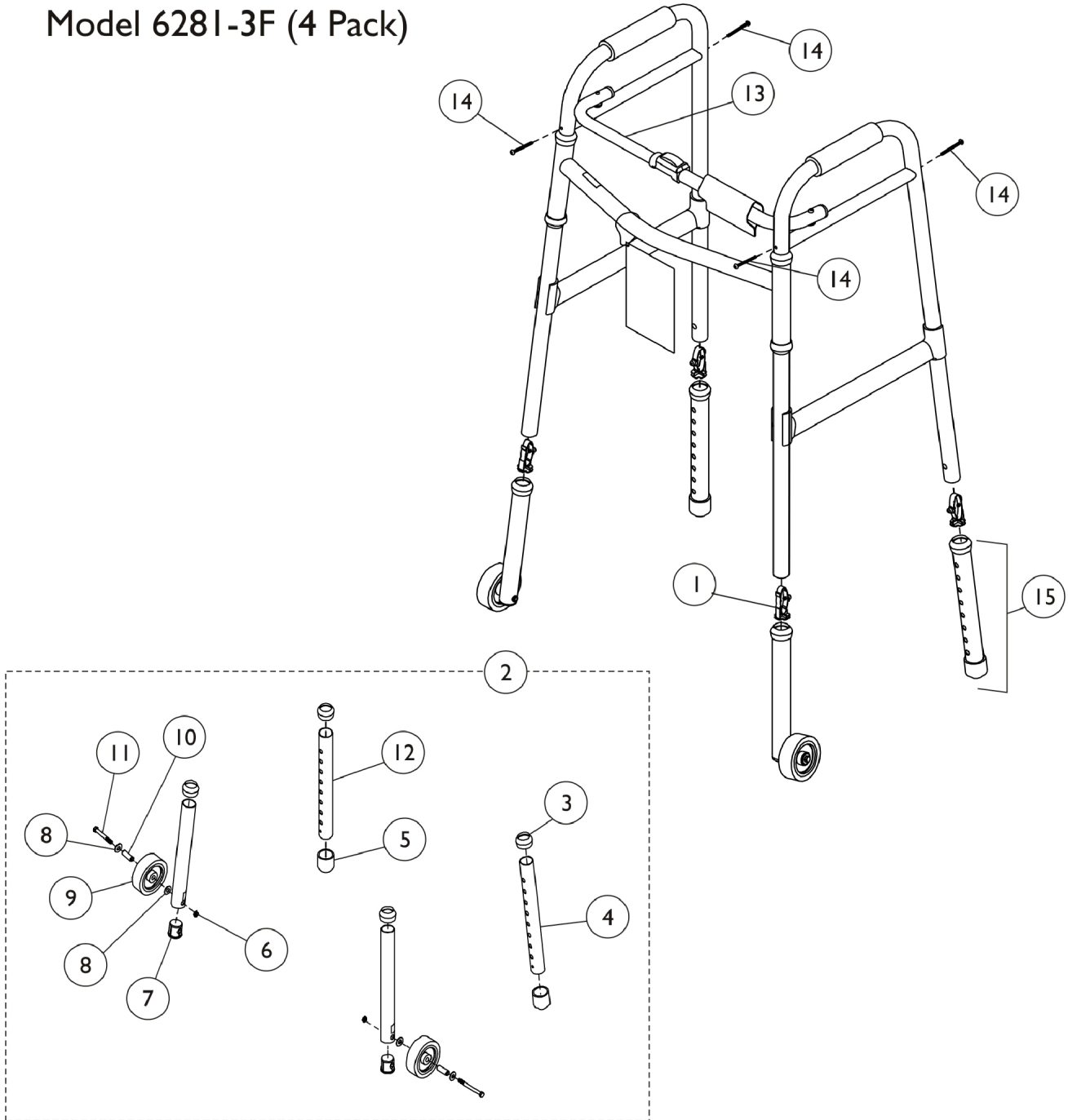

## Walker (Single Release)

### With 3" Fixed Wheel and Glide Tips

### Models 6281-3F (Discontinued 6/07)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	E	1074462-NLA	NLA - Button, Snap	1	4
1	F	1130780-NLA	NLA - Kit, Snap Buttons	10	1
1		1094945	Button, Snap	1	4
2	A,D	1060590	Kit, Leg Extension (3"), Fixed	1	1
3			Anti-Rattle, Grey (1-1/8")	1	4
4			Leg Extension Tube with Axle Hole	1	2
5			NLA - Tip, Glide, Grey (1-1/8")	1	2
6			Locknut (1/4-20)	1	2
7			Bushing (1")	1	2
8			NLA - Washer, Steel Flat (1/4 x 5/8 x 1/16")	1	4
9			NLA - Wheel (3" x 1")	1	2
10			Bushing (1/4 x 3/8 x 1/13")	1	2
11			Screw, Hex Head (1/4-20 x 2-1/2")	1	2
12			NLA - Leg, Adjustable	1	2
13		1060558-NLA	NLA - Folding Mechanism - Adult	1	1
13		1060559-NLA	NLA - Folding Mechanism - Junior	1	1
14		17564X000-NLA	NLA - Screw, Phillips Pan Head (#10-24 x 1-3/4")	1	4
15	B,C	1058000	Leg Extension Assembly (10")	1	2

NOTE: A - Includes (2) of each style.  
 B - Due to style changes, when replacing Leg Extensions before 6/25/98, order (4) each. Models built after 6/24/98 can order Single Assemblies.  
 C - Includes Leg, Rubber Tip and Anti-Rattle.  
 D - Kit Includes Items 3 - 12.  
 E - Individual item no longer available, use 1130780 as its replacement.  
 F - Kit is no longer available, use 1094945 as its replacement.