


# Walking Beam, Lower Pivot Link and Mounting Hardware

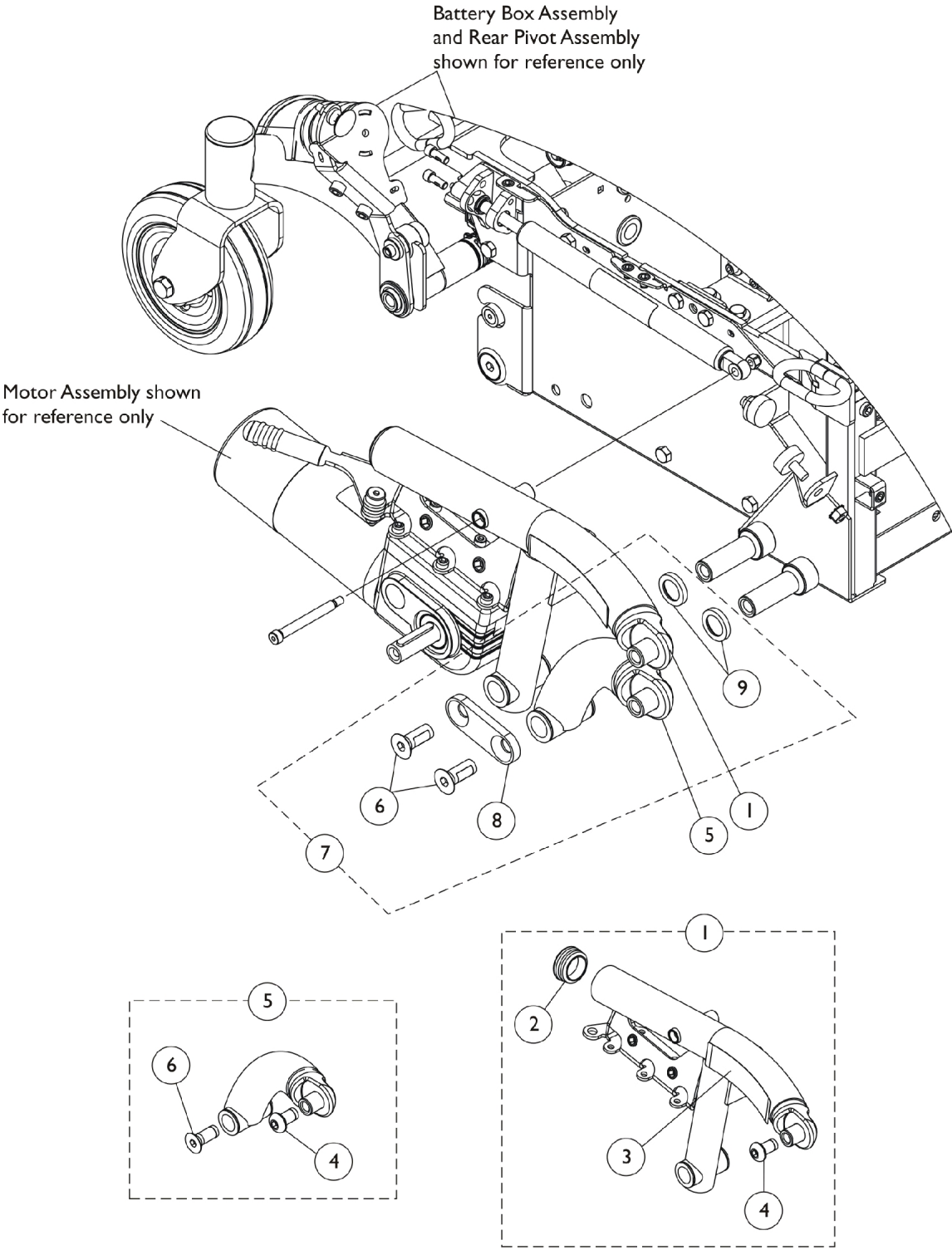

# Walking Beam, Lower Pivot Link and Mounting Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	H,A,C	8881151598	Kit, Walking Beam Assembly w/ Yellow Decal, Color - Right	1	1
2			NLA - Plug Button, Round, Black (1-1/2")	1	1
3			Decal, TDX SC, Yellow	1	1
4			Screw, Socket Button Head Cap w/ Patch (1/2-13 x 7/8")	1	1
1	H,A,C	8881151599	Kit, Walking Beam Assembly w/ Yellow Decal, Color - Left	1	1
2			NLA - Plug Button, Round, Black (1-1/2")	1	1
3			Decal, TDX SC, Yellow	1	1
4			Screw, Socket Button Head Cap w/ Patch (1/2-13 x 7/8")	1	1
1	H,B,C	8881151600	Kit, Walking Beam Assembly w/ Black Decal, Color - Right	1	1
2			NLA - Plug Button, Round, Black (1-1/2")	1	1
3			Decal, TDX SC, Black	1	1
4			Screw, Socket Button Head Cap w/ Patch (1/2-13 x 7/8")	1	1
1	H,B,C	8881151601	Kit, Walking Beam Assembly w/ Black Decal, Color - Left	1	1
2			NLA - Plug Button, Round, Black (1-1/2")	1	1
3			Decal, TDX SC, Black	1	1
4			Screw, Socket Button Head Cap w/ Patch (1/2-13 x 7/8")	1	1
1	E,H,C	8881151602	Kit, Walking Beam Assembly w/ Yellow Decal, Color - Right	1	1
2			NLA - Plug Button, Round, Black (1-1/2")	1	1
3			Decal, TDX SPREE w/ TDX Yellow Letters	1	1
4			Screw, Socket Button Head Cap w/ Patch (1/2-13 x 7/8")	1	1
1	E,H,C	8881151603	Kit, Walking Beam Assembly w/ Yellow Decal, Color - Left	1	1
2			NLA - Plug Button, Round, Black (1-1/2")	1	1
3			Decal, TDX SPREE w/ TDX Yellow Letters	1	1
4			Screw, Socket Button Head Cap w/ Patch (1/2-13 x 7/8")	1	1
1	F,H,C	8881154123	Kit, Walking Beam Assembly w/ Black Decal, Color - Right	1	1
2			NLA - Plug Button, Round, Black (1-1/2")	1	1
3			NLA - Decal, TDX SPREE w/ TDX Black Letters	1	1
4			Screw, Socket Button Head Cap w/ Patch (1/2-13 x 7/8")	1	1
1	F,H,C	8881154124	Kit, Walking Beam Assembly w/ Black Decal, Color - Left	1	1
2			NLA - Plug Button, Round, Black (1-1/2")	1	1
3			NLA - Decal, TDX SPREE w/ TDX Black Letters	1	1
4			Screw, Socket Button Head Cap w/ Patch (1/2-13 x 7/8")	1	1
3	A	1149308	Decal, TDX SC, Yellow	1	1
3	B	1149297	Decal, TDX SC, Black	1	1

# Walking Beam, Lower Pivot Link and Mounting Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
3	E	1151353	Decal, TDX SPREE w/ TDX Yellow Letters	1	1
3	F	1153865-NLA	NLA - Decal, TDX SPREE w/ TDX Black Letters	1	1
5	G,C	8881142330	Kit, Link Assembly, Lower Pivot, Color - Right	1	1
4			Screw, Socket Button Head Cap w/ Patch (1/2-13 x 7/8")	1	1
6			Screw, Socket Flat Head w/ Patch (1/2-13 x 1-1/4")	1	1
5	G,C	8881142331	Kit, Link Assembly, Lower Pivot, Color - Left	1	1
4			Screw, Socket Button Head Cap w/ Patch (1/2-13 x 7/8")	1	1
6			Screw, Socket Flat Head w/ Patch (1/2-13 x 1-1/4")	1	1
7	D	1151610	Kit, Walking Beam & Lower Pivot Link Mounting Hardware	1	1
6			Screw, Socket Flat Head w/ Patch (1/2-13 x 1-1/4")	1	2
8			Plate, Washer	1	1
9			Spacer, Coved (3/4 x 1-1/8 x 1-1/32")	1	2
<b>NOTE:</b> A - The TDX SC yellow decal is used for all frame colors except 60P Silver Metallic B - The TDX SC black decal is used for 60P Silver Metallic frame color only C - Specify frame color when ordering. See page titled "Frame Finishes, Touch-Up Paints and Upholstery Options". D - Includes items 6, 8 and 9 E - The TDX SPREE decal w/ yellow TDX Letters is used for all frame colors except 30P Sunny Yellow F - The TDX SPREE decal w/ black TDX Letters is used for 30P Sunny Yellow frame color only G - Includes the Lower Pivot Link Assembly, items 4 and 6 H - Includes the Walking Beam Assembly and items 2-4. When a new Walking Beam is ordered, new mounting hardware item #7 must be ordered also.					