


MK5 TAC/TRCM and Mounting Hardware For Round Back Canes

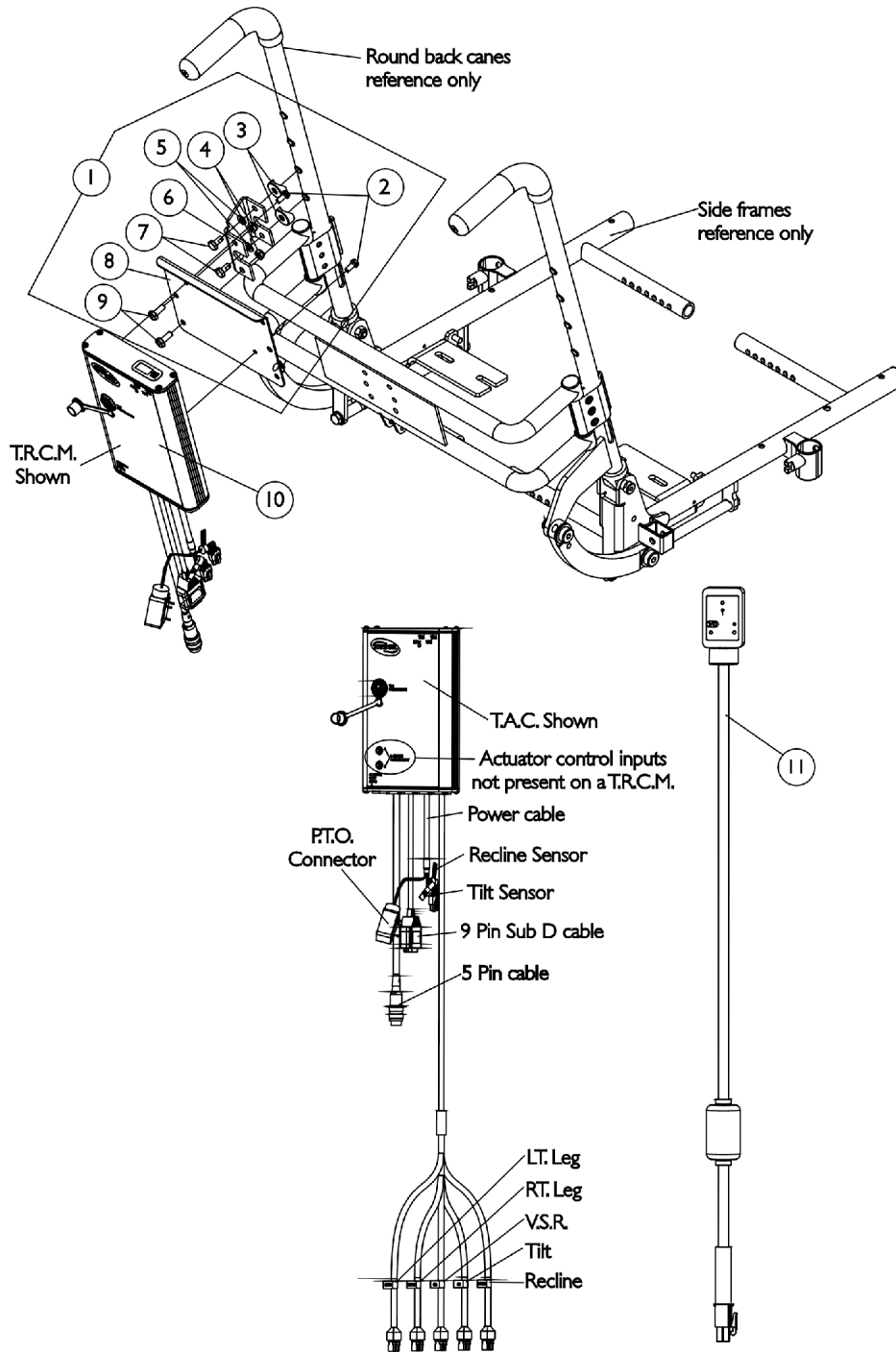

MK5 TAC/TRCM and Mounting Hardware For Round Back Canes

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1121181-NLA	NLA - Hardware, T.A.C./T.R.C.M. Mounting, Round Back Canes	1	1
2		1079316-NLA	NLA - Screw, Phillips Pan Head w/ Patch (#10-32 x 3/8")	1	2
3		1082908-NLA	NLA - Spacer, Coved (5/16 x 29/32 x 7/16")	1	2
4		60275X000	Locknut (1/4-20)	1	2
5		1026907	Washer (1/4 x 1/2 x 1/16")	1	2
6	B	1113638-NLA	NLA - Bracket, Vent-Connect Upper, Black - Right	1	1
7		1078214	Screw, Hex Head w/ Patch (1/4-20 x 1/2")	1	2
8		1113601-NLA	NLA - Bracket, Controller Mounting, Black	1	1
9		1121024	Screw, Hex Head w/ Patch (1/4-20 x 5/8")	1	2
10	C	1118728-NLA	NLA - Controller, MK5 TAC	1	1
10	F	1118729-NLA	NLA - Controller, MK5 TRCM	1	1
10		1118729E-NLA	NLA - Reconditioned - Controller, MK5 TRCM	1	1
11		1119418	Cable, Auxiliary Power	1	1
	D	1142593-NLA	Kit, MK5 SAC To MK5 TAC Conversion For Tarsys 2GT w/ Round Back Canes on TDX	1	1
2			NLA - Screw, Phillips Pan Head w/ Patch (#10-32 x 3/8")	1	2
3			NLA - Spacer, Coved (5/16 x 29/32 x 7/16")	1	2
4			Locknut (1/4-20)	1	2
5			Washer (1/4 x 1/2 x 1/16")	1	2
6			NLA - Bracket, Vent-Connect Upper, Black - Right	1	1
7			Screw, Hex Head w/ Patch (1/4-20 x 1/2")	1	2
8			NLA - Bracket, Controller Mounting, Black	1	1
9			Screw, Hex Head w/ Patch (1/4-20 x 5/8")	1	2
10			NLA - Controller, MK5 TAC	1	1
			Tie Wrap (11-1/2" L)	1	10
			Tie Wrap (5-5/8" L)	1	10
			NLA - Instruction Sheet, MK5 SAC To MK5 TAC Conversion For Tarsys 2GT w/ Round & Extruded Back Canes on TDX	1	1
	E	1142593E-NLA	NLA - WARR RPLMT - Kit, MK5 SAC to MK5 TAC Conversion Kit for Tarsys 2GT w/ Round Back Canes on TDX	1	1
1			NLA - Hardware, T.A.C./T.R.C.M. Mounting, Round Back Canes	1	1
10			NLA - WARR RPLMT - Controller, MK5 TAC	1	1
			Tie Wrap (11-1/2" L)	1	10
			Tie Wrap (5-5/8" L)	1	10
			NLA - Instruction Sheet, MK5 SAC To MK5 TAC Conversion For Tarsys 2GT w/ Round & Extruded Back Canes on TDX	1	1
	E	1095503-NLA	NLA - Package, Tie Wrap (5-5/8" L)	10	1
	E	1095502-NLA	NLA - Package, Tie Wrap (11-1/2" L)	10	1
	E	1141485-NLA	NLA - Instruction Sheet, MK5 SAC To MK5 TAC Conversion For Tarsys 2GT w/ Round & Extruded Back Canes on TDX	1	1

MK5 TAC/TRCM and Mounting Hardware For Round Back Canes

NOTE: A - Includes items 2-9
B - Vent-connect upper right bracket used on the left side when mounting a MK5 TAC or TRCM
C - When converting a MK5 SAC to a MK5 TAC, refer to the Conversion Kit part # 1142593. Conversion Kit includes all parts needed to convert a (Tarsys 2GT w/ Round Canes on TDX)
D - Not Shown. Kit components for a Tarsys 2GT w/ Round Canes on TDX uses items 2-10, Cable Ties and Instruction Sheet
E - Not Shown
F - 1118729 is no longer available. Order part number 1118729E as a replacement