


Rollator (Three Wheeled) Model 65510 & P429/2 (Current) Model 65410 (Before 9/17/07)

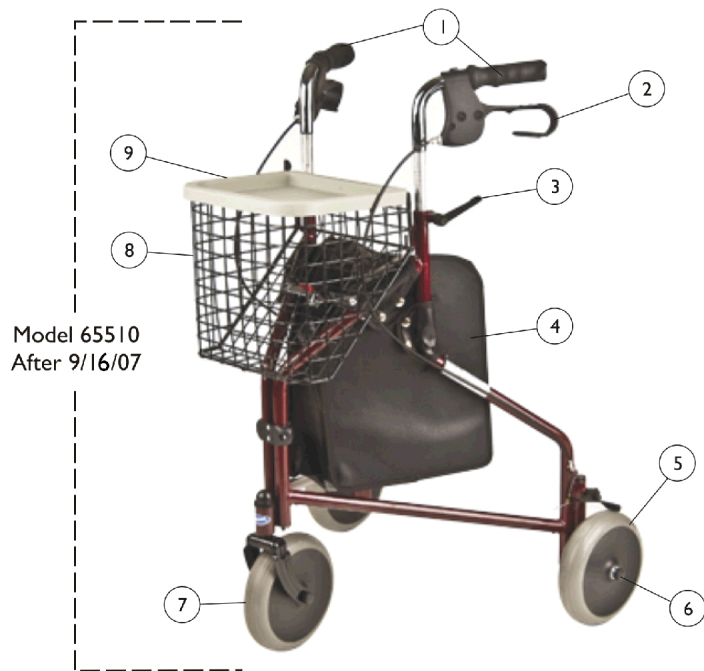

Rollator (Three Wheeled) Model 65510 & P429/2 (Current) Model 65410 (Before 9/17/07)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1142628	Grip, Hand (Pair)	2	1
2	G	1165507	Brake Assembly w/ Handle and Grip - Right	1	1
2	G	1165508	Brake Assembly w/ Handle and Grip - Left	1	1
3	A,E	1144499-NLA	NLA - Knob, Adjustable (Pair)	2	1
3	D,F	1142618	Lever, Hand (Pair)	2	1
4		1144504	Bag, Tote	1	1
5	B,E	1144500	Wheel, Rear	1	1
5	B,F	1452451	Kit, Rear Wheel w/ Hardware	1	1
6		1144505-NLA	NLA - Package, Cap, Plastic, Black	10	1
7	C,E	1144501-NLA	NLA - Wheel, Front w/ Hardware	1	1
7	C,F	1515541	Wheel, Front w/ Hardware	1	1
8		1144503	Basket	1	1
9		1144502	Tray	1	1

NOTE: A - Kit includes (2) Adjustable Knobs and Hardware.
B - Kit includes (1) Caster, Attachment Hardware and Cap for Bolt.
C - Kit includes (1) Wheel with Fork, Attachment Hardware and Cap for Bolt.
D - For models built after 9/16/2007. Kit includes (2) Levers and (2) of all Hardware.
E - For models built before 9/17/2007.
F - For models built after 9/16/2007.
G - Item #2 part numbers 1165507 and 1165508 are not interchangeable with older models manufactured before 9/17/07. Older models had holes to receive adjustable star knobs and model 65510 does not have holes, but engages with a lever style adjustment.

Note: As of 12/09/09 components of 65510 and P429/2 are interchangeable.