


Seat Frame Mounting Hardware

CGTE/CGTRE

Non-serviceable parts in the field require the wheelchair to be returned to Invacare for repair. Call Technical Services for assistance (1-800-832-4707)

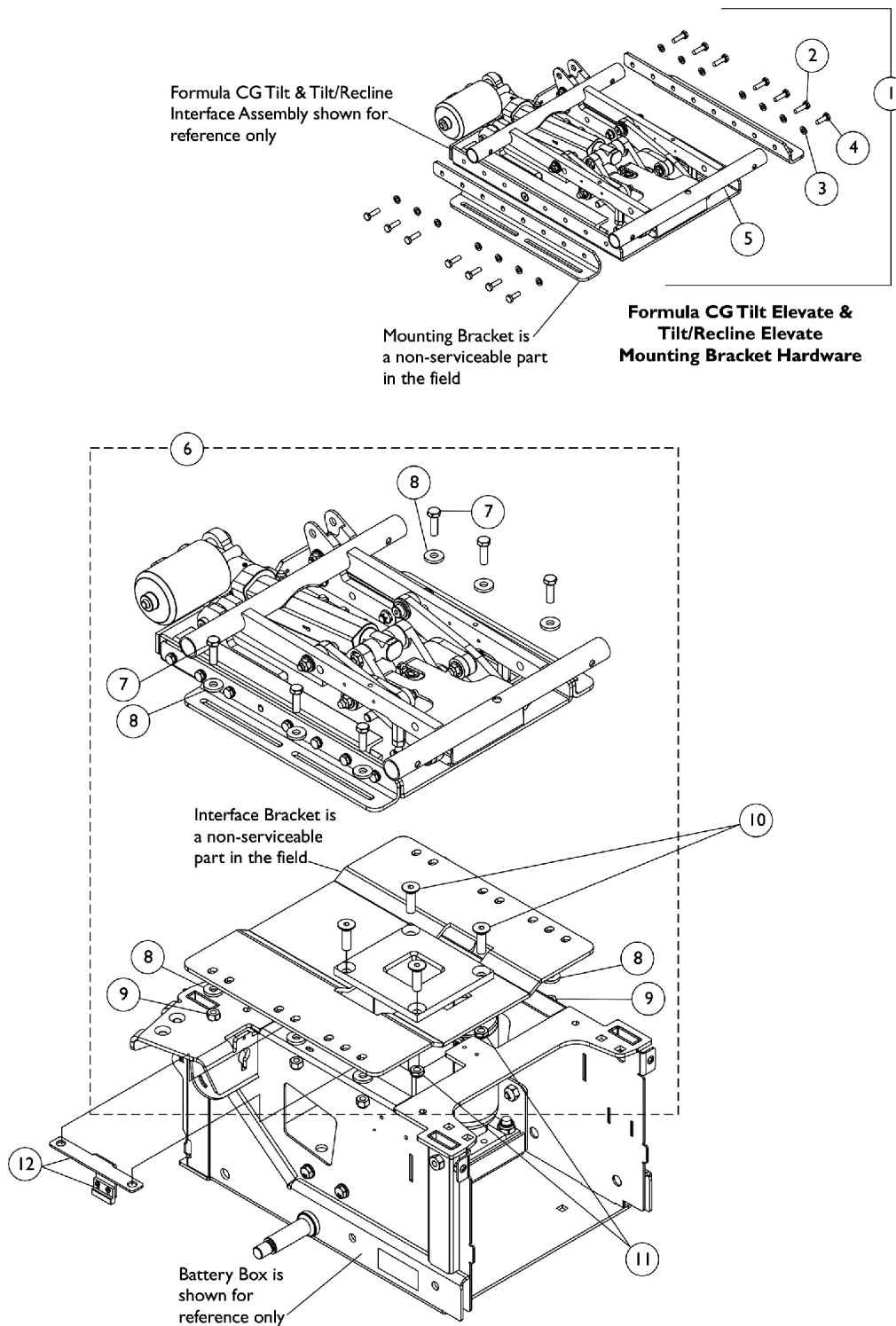

Seat Frame Mounting Hardware

CGTE/CGTRE

Non-serviceable parts in the field require the wheelchair to be returned to Invacare for repair. Call Technical Services for assistance (1-800-832-4707)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1121713-NLA	NLA - Kit, Formula CG Tilt & Tilt/Recline Mounting Bracket Hardware	1	1
2			Screw, Hex Head (1/4-20 x 7/8")	1	12
3			Lockwasher, Split (1/4")	1	12
4			NLA - Bolt, Hex Head Cap (1/4-20 x 3/4")	1	2
5			Locknut (1/4-20)	1	2
3		1007453	Lockwasher, Split (1/4")	1	2
6	B	1168996-NLA	NLA - Kit, Seat Frame Mounting Hardware, Elevate, Formula CG	1	1
7			Screw, Hex Head (5/16-18 x 1")	1	6
8			Washer, Flat (21/64 x 53/64 x 7/64")	1	12
9			Locknut (5/16-18)	1	6
10		1149787-NLA	NLA - Screw, Socket Flat Head (3/8-16 x 1-1/4")	1	4
11		1028486	Locknut, Low Profile (3/8-16)	1	4
12		1166200-NLA	NLA - Elevate Actuator Switch and Mounting Bracket Assembly	1	1
NOTE: A - Includes items 2-5. When item #1 1121713 is ordered, (2) ea. additional Split Lockwashers are required part #1007453. B - Includes items 7-9					