
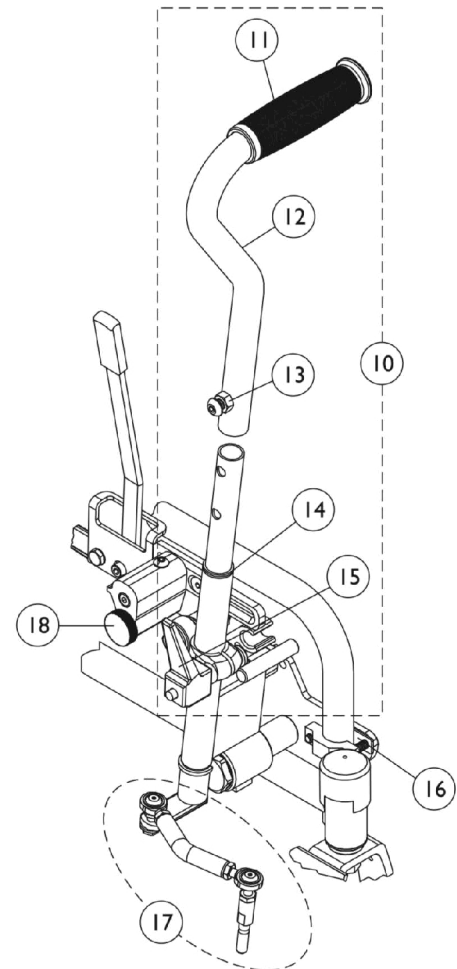
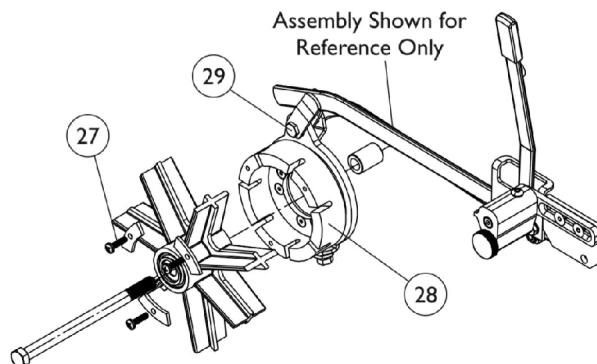
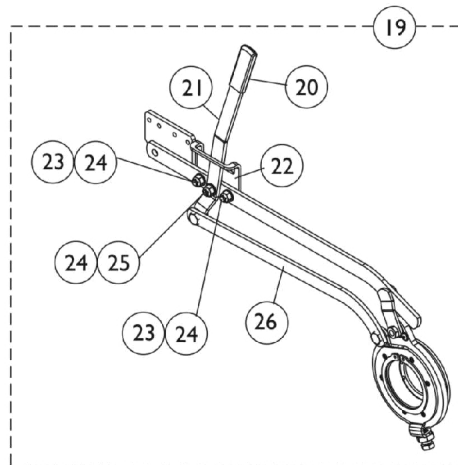


Cyclical Lever Drive -7 Spoke - Before 10/01

NOTE: Installation of the CLD **MUST** be performed by a qualified dealer. If a non-certified individual performs any work on these units, the Manufacturer's Warranty is Void.

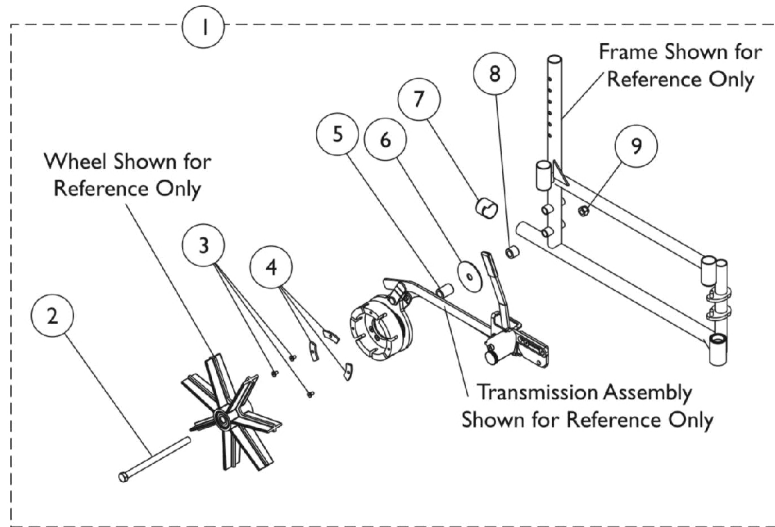

Cyclical Lever Drive -7 Spoke - Before 10/01

NOTE: Installation of the CLD MUST be performed by a qualified dealer. If a non-certified individual performs any work on these units, the Manufacturer's Warranty is Void.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1087540-NLA	NLA - Kit, Hub Attachment Hardware - Rear - Right/Left	1	1
2		1087407-NLA	NLA - Screw, Hex Head (7/16-20 x 6-1/2")	1	1
3		1025716	Screw, Phillips Pan Head (#10-32 x 11/16")	1	3
4		1087504-NLA	NLA - Plate, Support Rear Outer Hub	1	3
5		1087354-NLA	NLA - Spacer, Hub Inner (7/16 x 3/4 x 1-1/16")	1	1
6		1084822-NLA	NLA - Washer, Axle (7/16 x 2-3/8" x 1/16")	1	1
7		361441	Cap, End, Notched, Black	1	1
8		1087908-NLA	NLA - Spacer, Hub Outer Rear (7/16 x 3/4 x 19/32")	1	1
9		1045608	Locknut (7/16-20)	1	1
10	B	1087760-NLA	NLA - Drive Arm Assembly - Right	1	1
10	B	1087759-NLA	NLA - Drive Arm Assembly - Left	1	1
11		1087905-NLA	NLA - Grip, Handle (7/8")	1	1
12		1088108	Handle, Black	1	1
13		1088117	Screw, Socket Button Head (5/16-18 x 1-1/4")	1	1
14		1088110	Plunger	1	1
15		1087420-NLA	NLA - Handle, Adjustment, Tapped Type, Black	1	1
16		1088097	Support Bracket Assembly - Right	1	1
16		1088098	Support Bracket Assembly - Left	1	1
17		1087477	Steering Link Assembly - Right	1	1
17		1087478	Steering Link Assembly - Left	1	1
18		1088010-NLA	NLA - Wheel Lock Assembly	1	1
19	C	1087733-NLA	NLA - Transmission Assembly - Right	1	1
19	C	1087734	Transmission Assembly - Left	1	1
20		1087761-NLA	NLA - Knob, Shift Lever, Black	1	1
21		1087739-NLA	NLA - Lever, Shift	1	1
22		1087738-NLA	NLA - Plate, Shift	1	1
23		7930326-NLA	NLA - Package, Screw, Hex Head (1/4-20 x 3/4")	25	2
24		1087204-NLA	NLA - Washer, Flat (1/4 x 9/16 x 1/32")	1	6
25		1087752-NLA	NLA - Screw, Socket Head (1/4-20 x 3/4")	1	1
26		1088201-NLA	NLA - Transmission Lever	1	1
27		1025716	Screw, Phillips Pan Head (#10-32 x 11/16")	1	3
28		1087278-NLA	NLA - Hub, Outer - Rear (7 Spoke)	1	1
29		1087744-NLA	NLA - Bolt, Drive Lever	1	1

NOTE: A - Includes Items 2-9.
 B - Includes Items 11-15.
 C - Includes all of the components shown. Items 20-26 are sold separately.

NOTE: For CLD release handle see page titled, "Fork and Release Handle for Cyclical Lever Drive."