


Cyclical Lever Drive - 9 Spoke - Canadian Chairs Only

NOTE: Installation of the CLD **MUST** be performed by a qualified dealer. If a non-certified individual performs any work on these units, the Manufacturer's Warranty is Void.

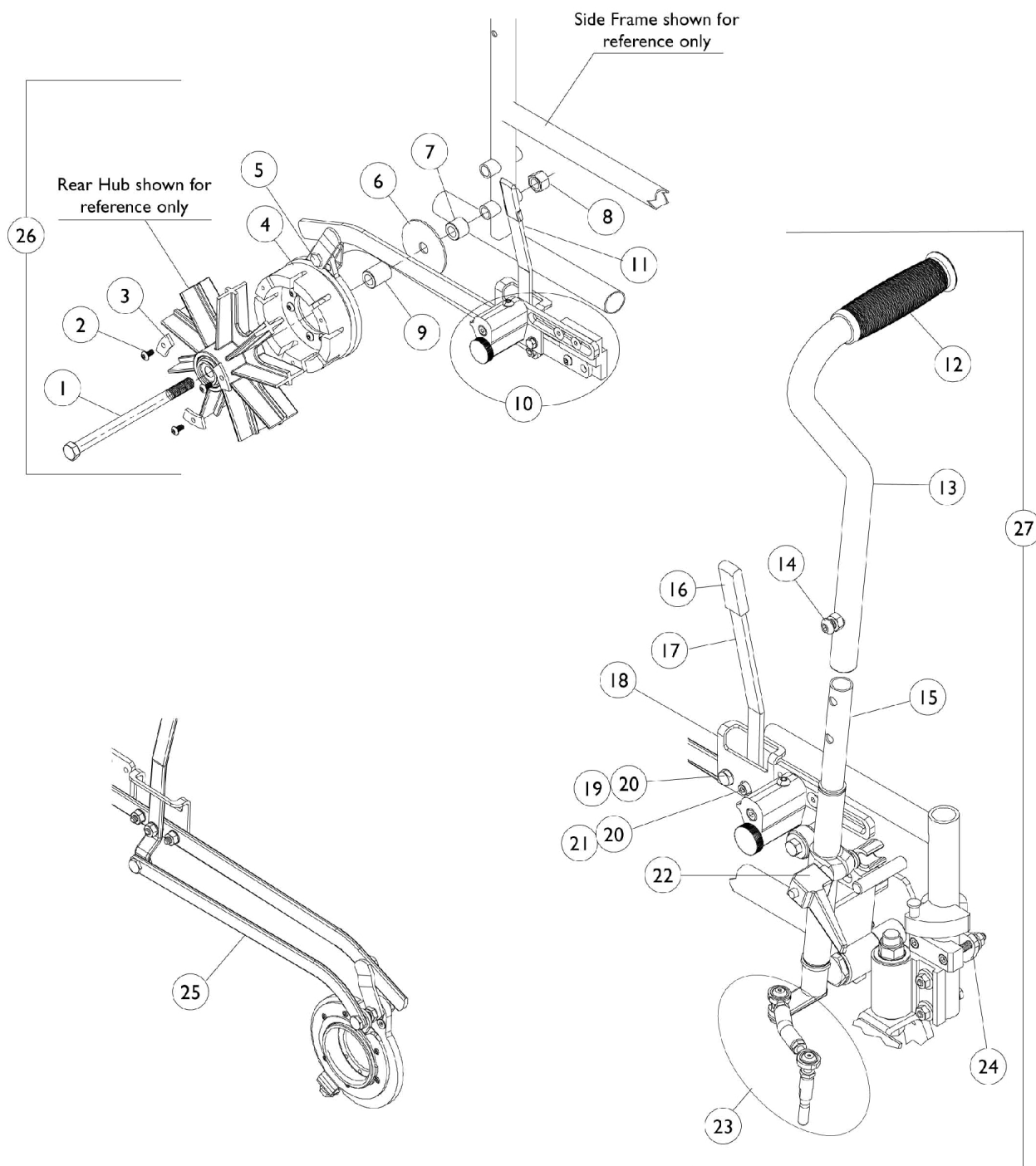

Cyclical Lever Drive - 9 Spoke - Canadian Chairs Only

NOTE: Installation of the CLD MUST be performed by a qualified dealer. If a non-certified individual performs any work on these units, the Manufacturer's Warranty is Void.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1087407-NLA	NLA - Screw, Hex Head (7/16-20 x 6-1/2")	1	1
2		1088180-NLA	NLA - Screw, Pan Head, Phillips (#10-32 x 3/4")	1	6
3		1087909-NLA	NLA - Plate, Hub Support	1	6
4		1087907-NLA	NLA - Hub, Outer - Rear (9 Spoke)	1	1
5		1087744-NLA	NLA - Bolt, Drive Lever	1	1
6		1084822-NLA	NLA - Washer, Axle (7/16 x 2-3/8" x 1/16")	1	1
7		1087908-NLA	NLA - Spacer, Hub Outer Rear (7/16 x 3/4 x 19/32")	1	1
8		1045608	Locknut (7/16-20)	1	1
9		1087912-NLA	NLA - Spacer, Hub Inner (7/16 x 3/4 x 15/16")	1	1
10		1087592	Wheel Lock Assembly - Right	1	1
10		1087463	Wheel Lock Assembly - Left	1	1
11	A	1087733-NLA	NLA - Transmission Assembly - Right	1	1
11	A	1087734	Transmission Assembly - Left	1	1
12		1087905-NLA	NLA - Grip, Handle (7/8")	1	1
13		1088108	Handle, Black	1	1
14		1088117	Screw, Socket Button Head (5/16-18 x 1-1/4")	1	1
15		1088110	Plunger	1	1
16		1087761-NLA	NLA - Knob, Shift Lever, Black	1	1
17		1087739-NLA	NLA - Lever, Shift	1	1
18		1087738-NLA	NLA - Plate, Shift	1	1
19		7930326-NLA	NLA - Package, Screw, Hex Head (1/4-20 x 3/4")	25	2
20		1087204-NLA	NLA - Washer, Flat (1/4 x 9/16 x 1/32")	1	6
21		1087752-NLA	NLA - Screw, Socket Head (1/4-20 x 3/4")	1	1
22		1087420-NLA	NLA - Handle, Adjustment, Tapped Type, Black	1	1
23		1087477	Steering Link Assembly - Right	1	1
23		1087478	Steering Link Assembly - Left	1	1
24		1087981-NLA	NLA - Support Bracket Assembly - Right	1	1
24		1087982-NLA	NLA - Support Bracket Assembly - Left	1	1
25		1088201-NLA	NLA - Transmission Lever	1	1
26	B	1087979-NLA	NLA - Kit, Hub Attachment - Rear	1	1
27	C	1087760-NLA	NLA - Drive Arm Assembly - Right	1	1
27	C	1087759-NLA	NLA - Drive Arm Assembly - Left	1	1

NOTE: A - Includes Items 1-3, 6-9
 B - Includes Items 1-3 and 6-9
 C - Includes Items 12-15 and 22-24

NOTE: For CLD release handle see page titled, "Fork and Release Handle for Cyclical Lever Drive."