
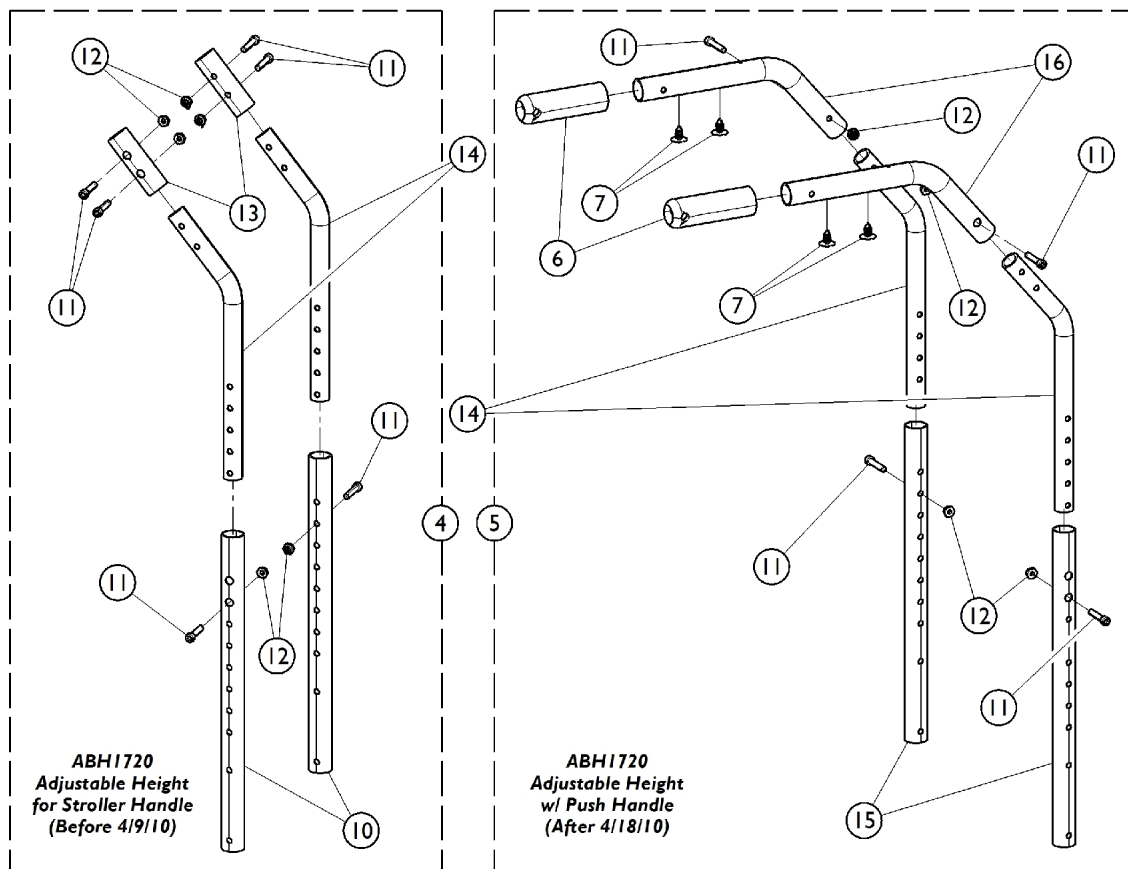


Back Cane Assembly - Chairs w/o Rocker Back (DROCK) Option

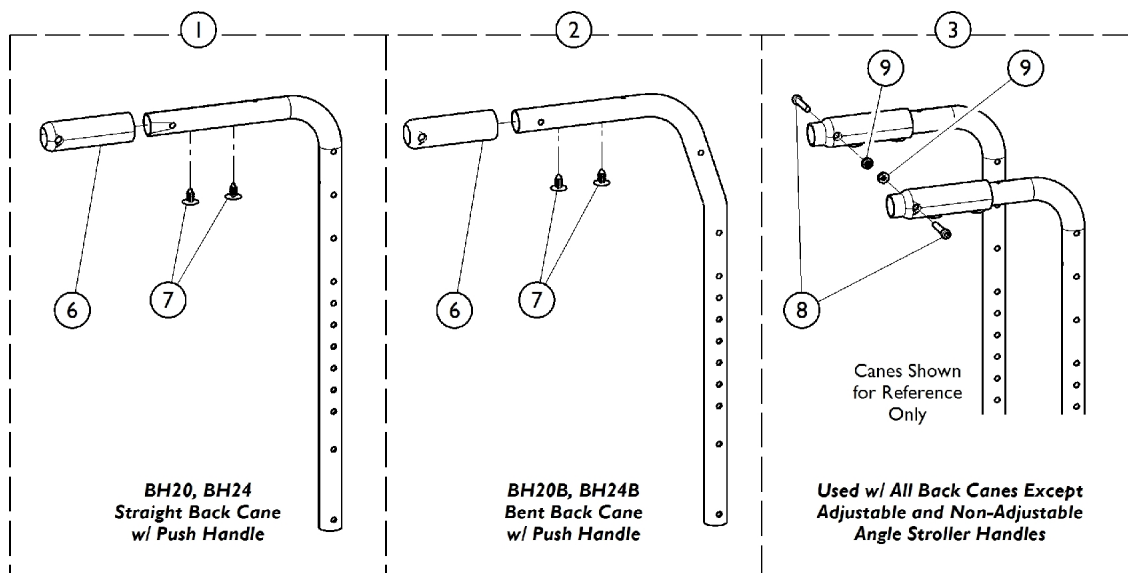

Back Cane Assembly - Chairs w/o Rocker Back (DROCK) Option

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	See Chart	#1 Straight Back Cane Assembly w/ Push Handle (BH20, BH24)	1	2
2	A	See Chart	#2 Bent Back Cane Assembly w/ Push Handle (BH20B, BH24B)	1	2
3	B	1166103	Kit, Grip Retaining Hardware	1	1
8			Screw, Socket Head (1/4-28 x 1-1/4")	1	2
9			Locknut (1/4-28)	1	2
4	C	1134189	Adjustable Height Cane Assembly (Pair)	1	1
10			Tube, Receiver Cane, Adjustable Height, Black	1	2
11			Screw, Allen Head (1/4-28 x 1-1/4")	1	6
12			Locknut (1/4-28)	1	6
13			Tube, Adapter, Stroller Handle, Black	1	2
14		1134187	Tube, Cane, Adjustable Height, Black (16 - 20")	1	2
5	D	1166079	Adjustable Height Cane Assembly (Pair)	1	1
6			Grip, Hand, Foam (4")	1	2
7			Panel Fastener, Snap In	1	4
11			Screw, Socket Head (1/4-28 x 1-1/4")	1	4
12			Locknut, Thin (1/4-28)	1	4
15			Tube, Receiver Cane, Adjustable Height, Black	1	2
16			Handle, Push - Right	1	1
16			Handle, Push - Left	1	1
NOTE: A - Includes the Back Cane and items 6 (Grip) and 7 (Snap In Fastener). B - Includes items 8 and 9. Used with all cane assemblies except for Stroller Handles. C - Includes items 10 - 13. Used with Standard Aluminum (SCANE) and Heavy Duty Package (HDPG) chairs. D - Includes items 6, 7, 11, 12, 15, and 16. Used with Std. Alum. (SCANE), Heavy Duty Package (HDPG).					

#1 Straight Back Cane Assembly w/ Push Handle (BH20, BH24)

Back Cane Type	BH20 Straight Cane	BH24 Straight Cane
Standard Aluminum (SCANE)	1127685	1127686
Steel Reinforced (RCANE)	1121600	1121601
Heavy Duty Package (HDPG)	1121600	1121601

#2 Bent Back Cane Assembly w/ Push Handle (BH20B, BH24B)

Back Cane Type	BH20B Cane	BH24B Cane
Standard Aluminum (SCANE)	1167661	1166076 See Note(s) A
Steel Reinforced (RCANE)	1167662	1166077 See Note(s) A

Back Cane Assembly - Chairs w/o Rocker Back (DROCK) Option

#2 Bent Back Cane Assembly w/ Push Handle (BH20B, BH24B)

Back Cane Type	BH20B Cane	BH24B Cane
Heavy Duty Package (HDPG)	1167662	1166077 See Note(s) A
NOTE: A - Back Cane handle shortened by 1/2 inch on 1/25/10. It is recommended for chairs built before 1/25/10 that the canes be ordered in pairs.		