


Rocker Back and Mounting Hardware (DROCK) Option

When ordering this kit, you must order new Back Canes also.
(See Category "Canes - Back - Chairs built w/ DROCK Option")

Not available with HDPG, TRRO, LOSHEAR, LOWSHEARWO, RCANE, or FLDBACK.

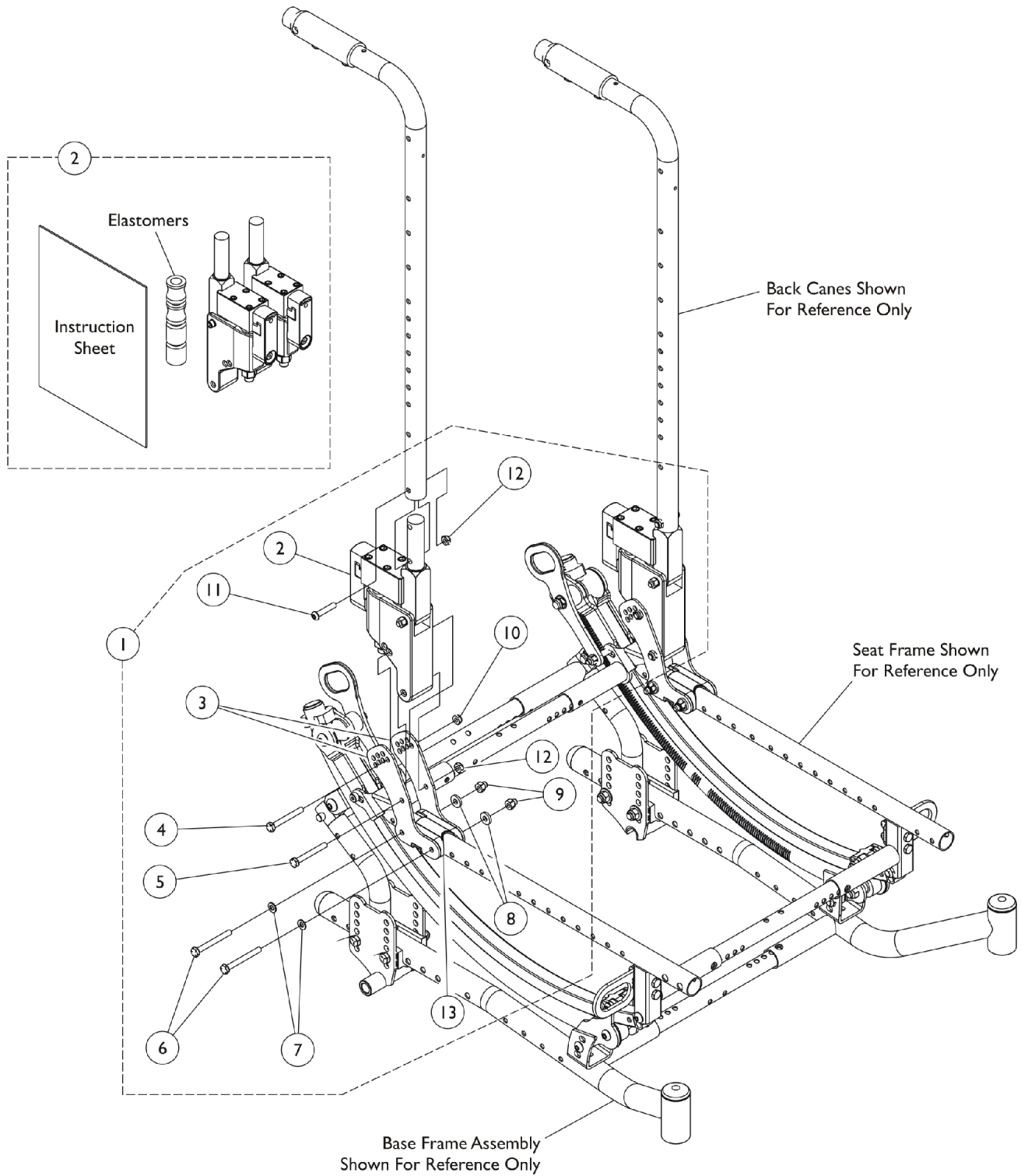

Rocker Back and Mounting Hardware (DROCK) Option

When ordering this kit, you must order new Back Canes also.
 (See Category "Canes - Back - Chairs built w/ DROCK Option")

Not available with HDPG, TRRO, LOSHEAR, LOWSHEARWO, RCANE, or FLDBACK.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1167665	Kit, Rocker Back Assembly and Mtg. Hardware (Pair)	1	1
2			Rocker Back Assembly (Pair)	2	1
3			Bracket, Backrest, Black	1	4
4			Screw, Hex Head (1/4-28 x 2")	1	2
5			Screw, Hex Head (1/4-28 x 2")	1	2
6			Screw, Hex Head (1/4-28 x 2-1/4")	1	4
7			NLA - Washer, Steel (1/4 x 1/2 x 3/64")	1	4
8			Washer, Steel (1/4 x 5/8 x 3/32")	1	4
9			Nut, Hex Nylon (1/4-28)	1	4
10			Locknut, Thin (1/4-28)	1	4
11			Screw, Hex Head (1/4-28 x 2")	1	2
12			Locknut (1/4-28)	1	4
13			Uni-Washer, Dual	1	2
2	B	1166090	Rocker Back Assembly (Pair)	1	1

NOTE: A - Includes items 2 thru 13.
 B - Includes Rocker Backs, Instruction Sheet and Elastomers. All components are sold together and are non-saleable as individual items.