


9" Caster

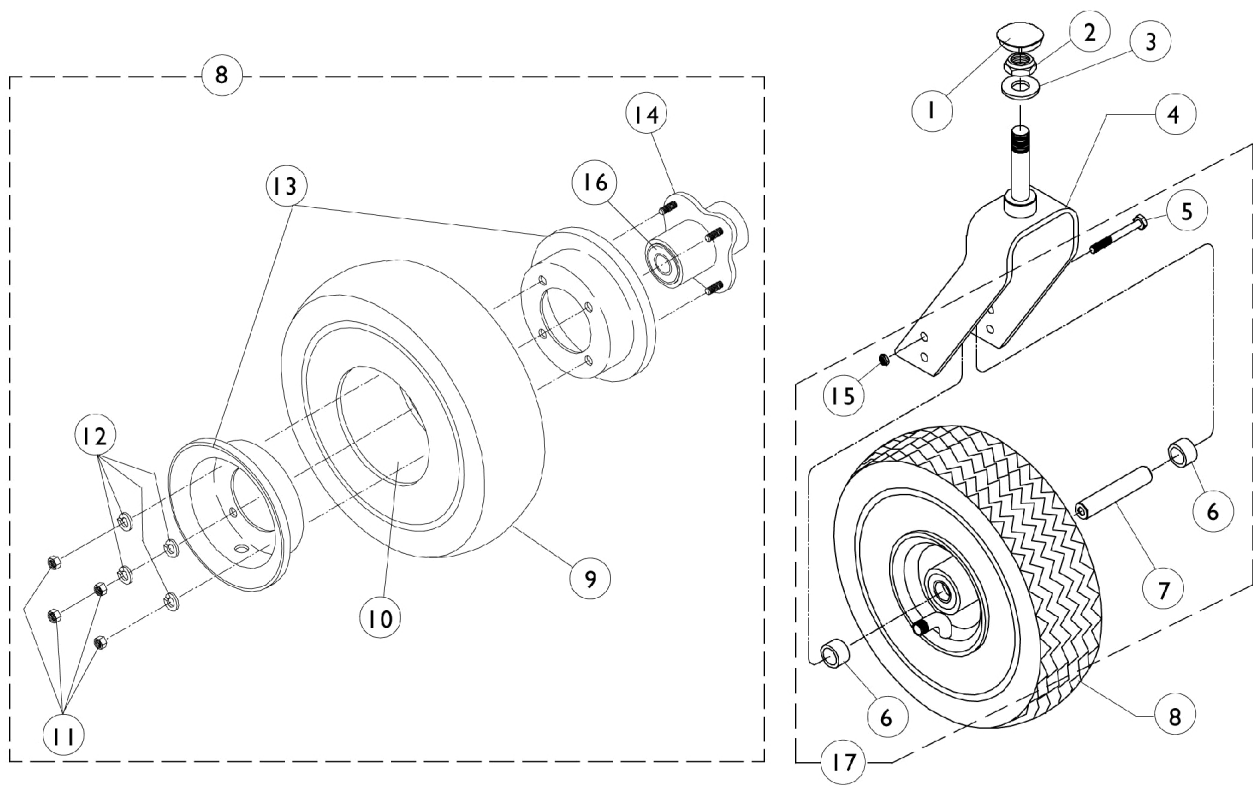

9" Caster

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1027600-NLA	NLA - Cap, Black	1	1
2		1028550	Locknut (5/8-18)	1	1
3		1041444	Washer, Nylon (5/8 x 1-5/16 x 1/8")	1	1
4	A,D	8881039754-NLA	NLA - Fork and Stem Assembly (9" and 10" Caster)	1	1
4	A,E	8881041477-NLA	NLA - Fork and Stem Assembly, Heavy Duty, Color (9" Caster)	1	1
8		1041498-NLA	NLA - Wheel Assembly, Pneumatic (9" x 2-3/4")	1	1
8	B	1029871-NLA	NLA - Wheel Assembly, Foam Filled (9 x 2-3/4")	1	1
9		1027418-NLA	NLA - Tire, Pneumatic, Grey (9 x 2-3/4")	1	1
9		1028896-NLA	NLA - Tire w/ Foam Fill Insert, Grey (9" x 2-3/4")	1	1
10		1027417	Tube, Pneumatic Inner (9 x 2-3/4")	1	1
11		1024426-NLA	NLA - Nut, Hex (5/16-24)	1	4
12		1024442-NLA	NLA - Washer, Lock (5/16 x 9/16 x 1/16")	1	4
13		1025399-NLA	NLA - Kit, Rim, Outer/Inner Half w/ Hardware (9")	1	1
13		1013460-NLA	NLA - Kit, Rim, Outer/Inner Half w/ Hardware (10")	1	1
14		1013466	Hub with Bearings Assembly	1	1
16		1015610	Bearing with Grip Ring (5/8")	1	2
17	C	1039757	Hardware, 9" Caster, Standard Fork (Per Side)	1	1
NOTE: A - Specify frame color when ordering. See page titled, "Frame Finishes, Touch-Up Paints and Upholstery Options." B - Tire comes with insert molded inside. C - Includes Items 5, 6, 7 and 15. Hardware sold complete. D - 250 lb. standard. E - 400 lb. heavy duty.					