
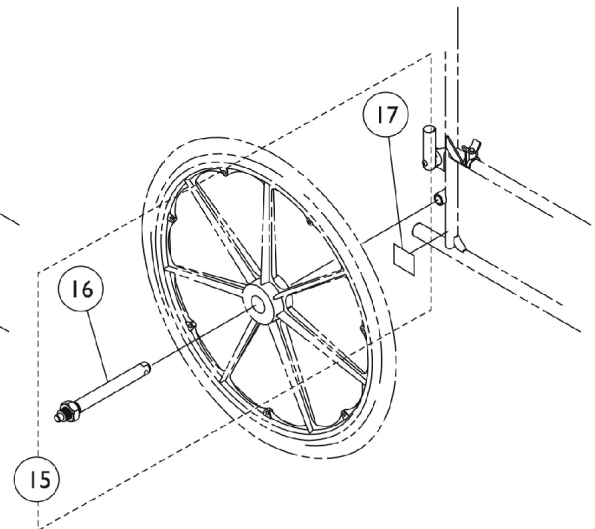
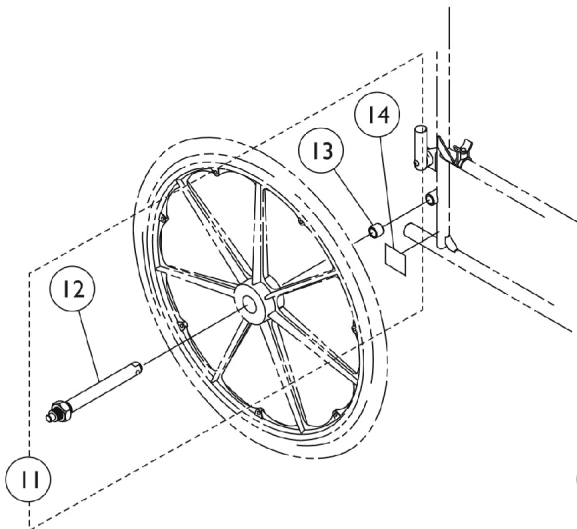
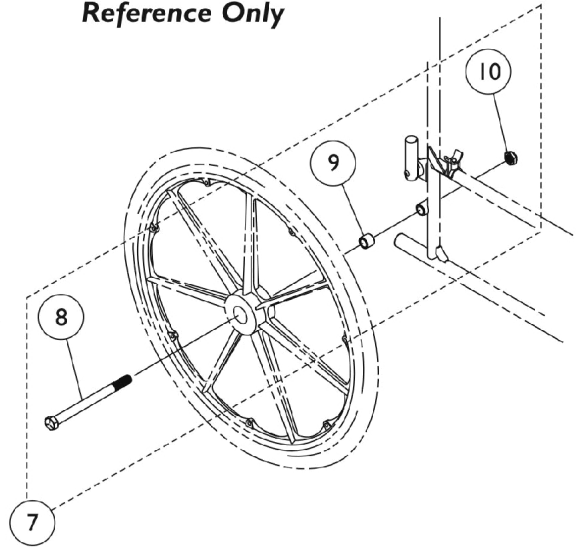


Rear Wheel Hardware

Frame and Wheels for Reference Only

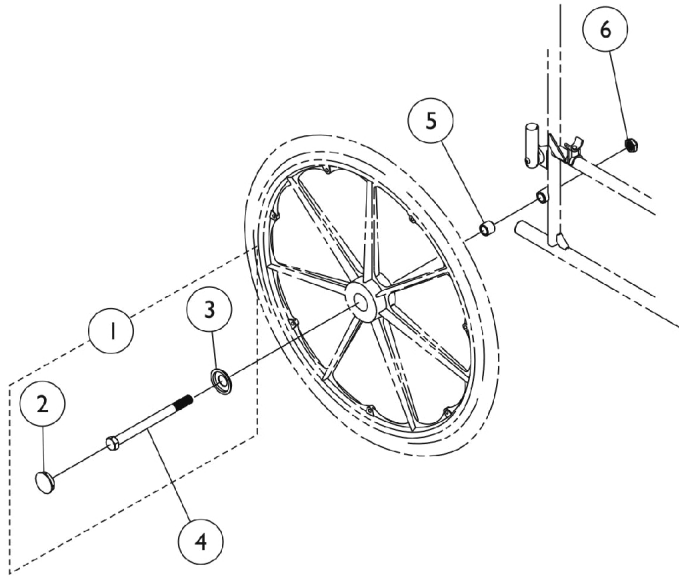

Rear Wheel Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	E	1047943	Kit, Rear Wheel Hub Hardware	1	1
2			Cap, Hub, Black	1	1
3			Washer, Caplugs (15/32" I.D. x 1-3/32" O.D.)	1	1
4	A	47203X025	Bolt, Hex Head Cap (7/16-20 x 4-3/4")	1	1
4	B	1085454	Bolt, Hex Head Cap (7/16-20 x 5-1/8")	1	1
5	A	1009857-NLA	NLA - Spacer (7/16 X 5/8 X 1/8")	1	1
5	B	1022671	Spacer (7/16 x 5/8 x 1/2")	1	1
6		1045608	Locknut (7/16-20)	1	1
7	C	1034109-NLA	NLA - Kit, Rear Wheel Hardware, Conventional	1	1
8			Bolt, Hex Head Cap (7/16-20 x 5-1/8")	1	1
9			Spacer (7/16 x 5/8 x 1/2")	1	1
10			Locknut (7/16-20)	1	1
8		1085454	Bolt, Hex Head Cap (7/16-20 x 5-1/8")	1	1
9		1022671	Spacer (7/16 x 5/8 x 1/2")	1	1
10		1045608	Locknut (7/16-20)	1	1
7	C	1110319	Kit, Rear Wheel Hardware, Space Saver	1	1
8			Bolt, Hex Head Cap (7/16-20 x 4-3/4")	1	1
9			Spacer (7/16 x 5/8 x 3/16")	1	1
10			Locknut (7/16-20)	1	1
11	B,D	1034113	Kit, Quick Release Rear Wheel Hardware, Conventional	1	1
12			Axle, Quick Release (7/16" x 5-7/16")	1	1
13			Spacer (7/16 x 5/8 x 1/2")	1	1
14			NLA - Label, Warning	1	1
15	A,F	1034114	Kit, Quick Release Rear Wheel Hardware, Space Saver	1	1
16			Axle, Quick Release (7/16" x 4-1/2")	1	1
17			NLA - Label, Warning	1	1

NOTE: A - For Space Saver Arms Chairs.
 B - For Conventional Arm Chairs.
 C - Kit includes items 8, 9, and 10. KIT is No Longer Available. Must order the components separately along with 1047943.
 D - Kit includes items 12, 13, and 14.
 E - Kit includes items 2 and 3.
 F - Kit includes items 16 and 17.