


Frame Hardware - Recliner Back

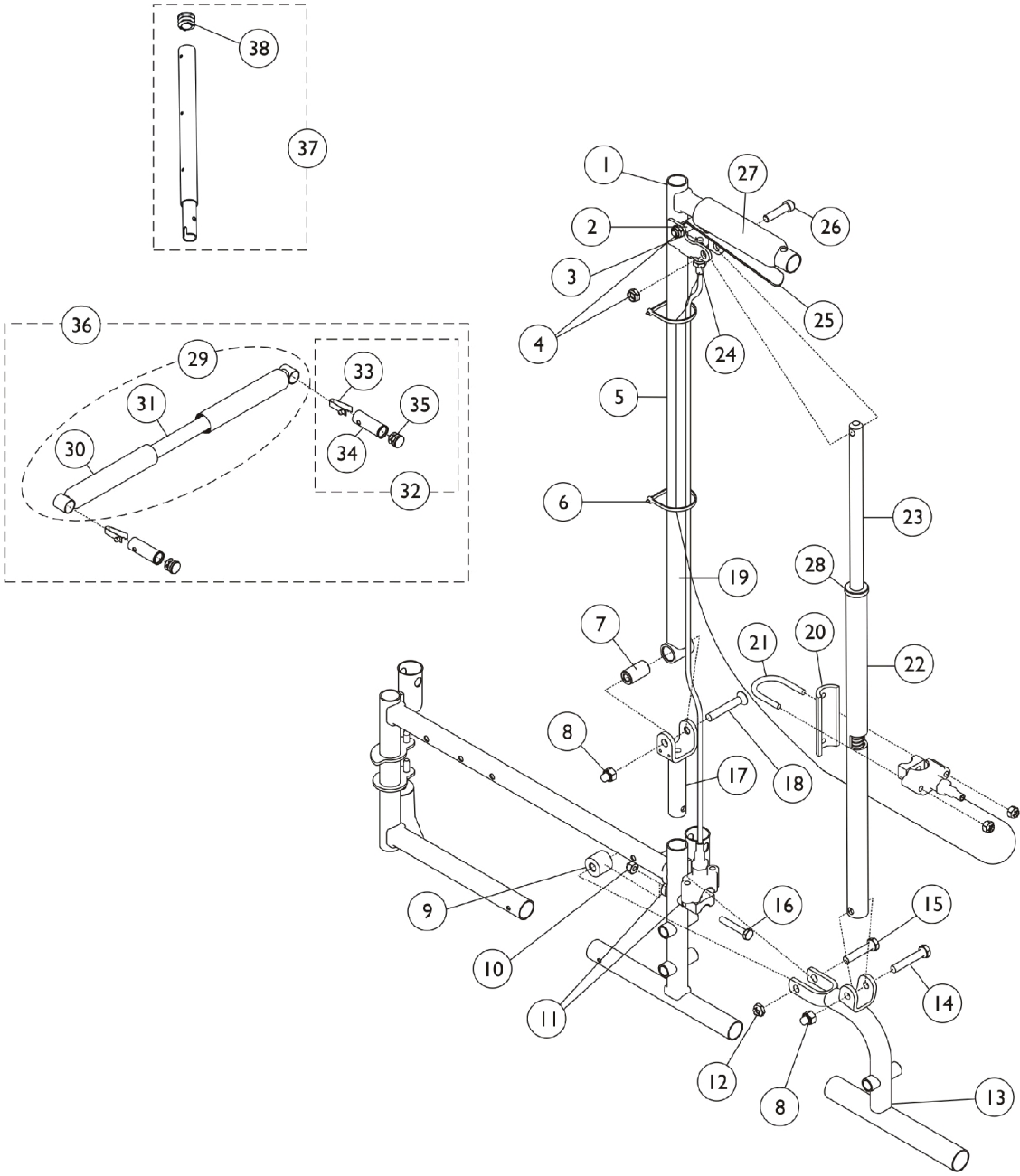

Frame Hardware - Recliner Back

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1074497	Cane Tube Assembly, Rear, Recliner, Black - Right	1	1
1	A	1074498	Cane Tube Assembly, Rear, Recliner, Black - Left	1	1
2		1071676	Fastener, Spring Grip	1	1
3		1021344	Screw, Socket Head (5/16-18 x 1-1/2")	1	1
4		60290X002	Locknut (5/16-18)	1	2
5		1073495	Tube, Back, Black	1	1
6		1021260	Tie Wrap (5-5/8" L)	1	2
7		1058938	Bushing, Pivot (5/16 x 5/8 x 1-1/8")	1	1
8		1060057	Nut, Cap (5/16-24)	1	2
9		47324X011	Bushing, Pivot (5/16 x 7/8 x 3/4")	1	1
10		1008882	Locknut (1/4-20)	1	3
11		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	1
12		1107321	Nut, Low Cap (5/16"-24)	1	1
13		1081095-NLA	NLA - Bracket, Recliner Adapter, Chrome - Right	1	1
13		1081096-NLA	NLA - Bracket, Recliner Adapter, Chrome - Left	1	1
14		11239X000-NLA	NLA - Screw, Hex Head (5/16-24 x 1-3/4")	1	1
15		11239X001	Screw, Hex Head (5/16-24 x 1-1/2")	1	1
16		16720X004	Screw, Hex Head (1/4-20 x 1-1/2")	1	1
17		1073494-NLA	NLA - Bracket, U, Chrome	1	1
18		1058920	Screw, Socket Flat Head (5/16-24 x 2")	1	1
19		1074471	Cable and Housing Assembly	1	1
			Cable Assembly	1	1
			Nut, Hex (5/16"-18)	1	2
			Housing, Support	1	1
			NLA - Spring, Compression (27/64 x 1 x 3/64")	1	1
			NLA - Sector, Threaded	1	1
			Tubing, Heatshrink Adhesive (1-1/2")	1	1
			NLA - Nut, Swivel (5/16-18 x 21/32")	1	1
20		1023329-NLA	NLA - Plate Support	1	1
21		1023328	U-Bolt (1/4-20 x 1-13/16")	1	1
22	J	1086366-NLA	NLA - Tube, Locking, Chrome	1	1
22		1142777	Tube, Locking, Black	1	1
23		1042736	Rod, Locking	1	1
24		1070840-NLA	NLA - Nut, Swivel (5/16-18 x 21/32")	1	1
25		1021085	Lever, Brake Release, Black	1	1
26		1021343	Screw, Socket Head (5/16-18 x 1-1/4")	1	1
27		1061484	Grip, Handle, Foam, Black (5")	1	1
28		1024221	Anti-Rattle, Black	1	1
30	B	1061484	Grip, Handle, Foam, Black (5")	1	2
30	C	1032245	Grip, Foam Hand (6-7/8" L)	1	2
29	D	1088377	Bar Spreader with Grips, Chrome (14" W)	1	1
29	D	1062638-NLA	NLA - Bar, Spreader with Grips, Chrome (16" W)	1	1
29	D,K	1062639-NLA	NLA - Bar, Spreader with Grips, Chrome (18" W)	1	1
29	D,K	1088382-NLA	NLA - Bar Spreader with Grips, Chrome (20" W)	1	1
29	D	1088377	Bar Spreader with Grips, Chrome (14" W)	1	1
32	E	1071120	Lock Handle	1	1
33			Button, Single Snap	1	1

Frame Hardware - Recliner Back

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
34			Insert, Seat Rail	1	1
35			Plug Button, Black (3/4")	1	1
36	F,K	1088378-NLA	NLA - Bar Spreader with Locking Pins, Chrome (14" W)	1	1
36	F,K	1071121-NLA	NLA - Bar, Spreader with Locking Pins, Chrome (16" W)	1	1
36	F,K	1071122-NLA	NLA - Bar, Spreader with Locking Pins, Chrome (18" W)	1	1
36	F,K	1088383-NLA	NLA - Bar, Spreader with Locking Pins, Chrome (20" W)	1	1
36	F,K	1088388-NLA	NLA - Bar, Spreader with Locking Pins, Chrome (22" W)	1	1
36		1182040	Bar, Spreader with Locking Pins, Black (14" W)	1	1
36		1182041	Bar, Spreader with Locking Pins, Black (16" W)	1	1
36		1182042	Bar, Spreader with Locking Pins, Black (18" W)	1	1
36		1182043	Bar, Spreader with Locking Pins, Black (20" W)	1	1
36		1182044	Bar, Spreader with Locking Pins, Black (22" W)	1	1
37	I	1054595-NLA	NLA - Headrest Extension Tube Assembly, Black	1	2
38		1027095	Plug Button, Black (7/8")	1	1
	G,H	1086368-NLA	NLA - Tube Assembly, Locking, Chrome - Right	1	1
	G,H	1086369-NLA	NLA - Tube Assembly, Locking, Chrome - Left	1	1

NOTE: A - Includes Items 2, 5-6, 19, 24, and 27.
 B - Used on 14" Wide Chairs.
 C - Used on 16 - 22" Wide Chairs.
 D - Includes items 30 and 31.
 E - Includes items 33-35.
 F - Includes items 30-35.
 G - Not Shown.
 H - Includes Items 1, 4, 7-18, 20-23, 26, and 28.
 I - Includes item 38
 J - Use 1142777 as a replacement for chrome that is no longer available..
 K - Chrome is No Longer Available. Select Item #36 Spreader Bar in Black as a replacement.