


IH3602GXL, IH3802G, IH3752G - Door Handle and Safety Latch Components

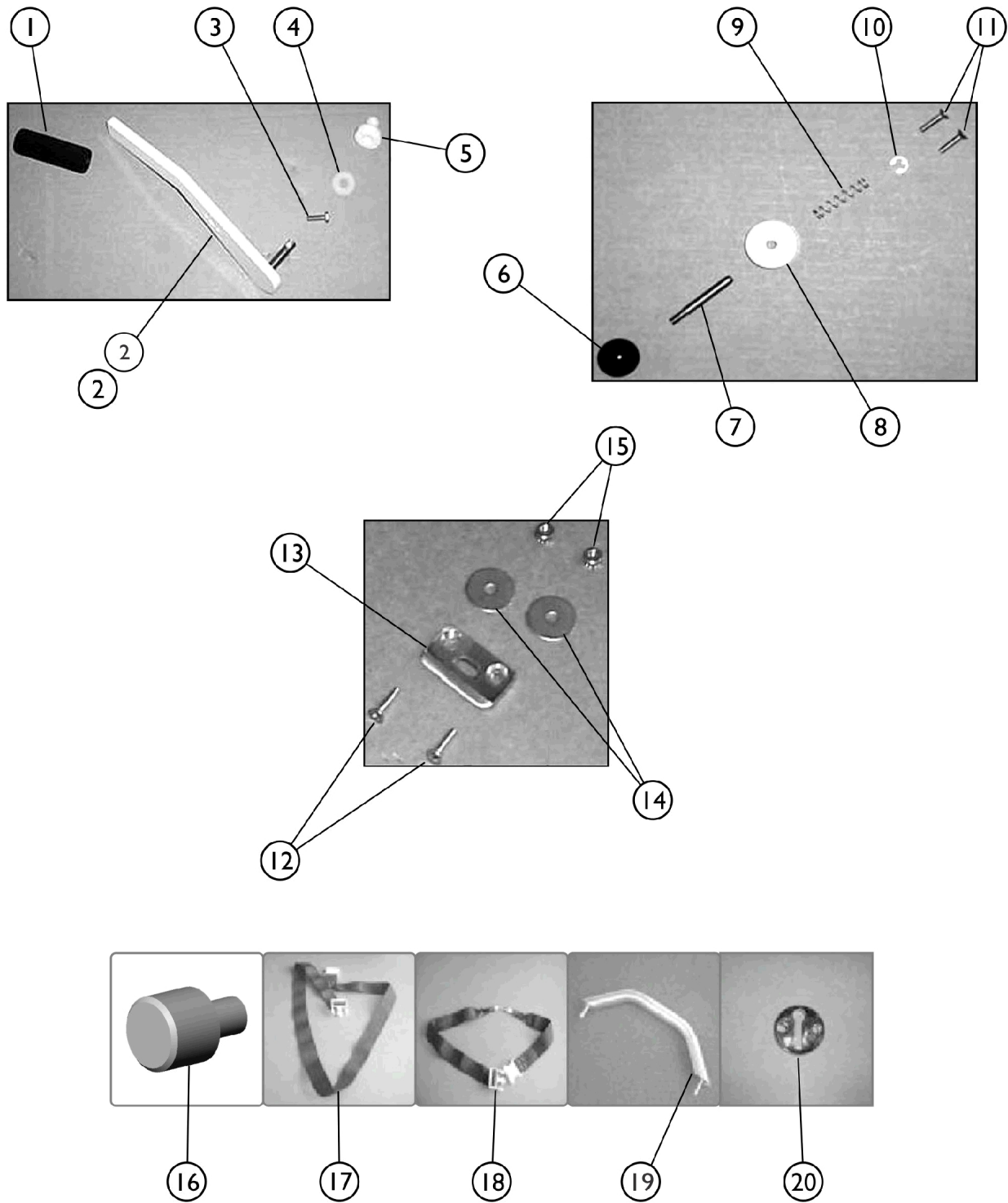

IH3602GXL, IH3802G, IH3752G - Door Handle and Safety Latch Components

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		TRP1036	Door Handle Grip	1	1
2		PAR3620B	Handle, White Door	1	1
3		PAR9901	Bolt (5/16-18 x 1")	1	1
4		PAR9240A	Washer, Nylon	1	1
5		IHPAR3621	Bushing, Door Handle	1	1
6		PAR3606	Knob, Safety Latch	1	1
7		PAR3601	Pin, Safety Latch	1	1
8		PAR3603B	Collar, Safety Latch	1	1
9		PAR3604	Spring, Safety Latch	1	1
10		PAR3605	Cir-Clip	1	1
11		IHPAR9911	Screw, Phillips, Flat Head (#10-24 x 3/4")	2	2
12		IHPAR9912	Screw, Phillips, Flat Head (#10-24 x 1")	2	2
13		PAR3607	Striker Plate, Safety Latch	1	1
14		IHPAR9913	Washer, Fender	2	2
15		IHPAR9910	Nut (#10-24)	2	2
16		PAR3619	Stop, Door	1	1
17		PAR9268ABC	Chest Safety Harness	1	1
18		PAR9260	Seat Safety Harness	1	1
19	A	IHPAR9261	Grab Bar (10")	1	1
20		PAR9269B	Tub Catch, Chrome	1	1

NOTE: A - All Models except for 3602GXL.