


IH3602GXL, IH3752G, IH3802G, IH6302GADH, IH6302G

Access Doors and Panels

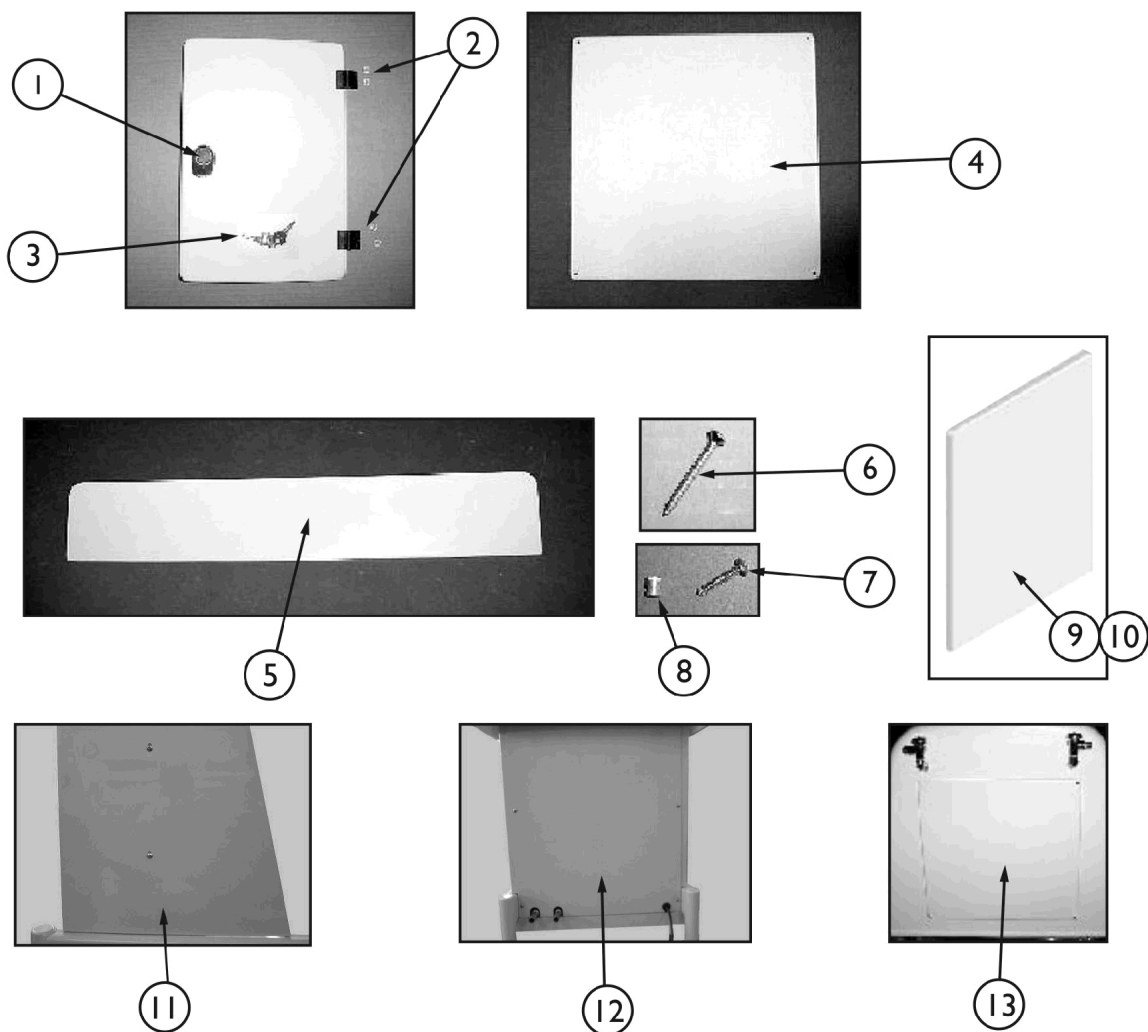

IH3602GXL, IH3752G, IH3802G, IH6302GADH, IH6302G

Access Doors and Panels

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	IHPAR7901	Door Latch Assembly (Includes Keys)	1	1
2	A	PAR5410	Hinge, Access Door	1	1
3	A	IHPAR7903-NLA	NLA - Key Set	1	1
4	B	PARP33W	Panel, White Back Access	1	1
5	C	PARP32W	Panel, White Bottom Front Access	1	1
6	A	IHPAR9909	NLA - Screw, Phillips, Flat Head, Oval (6-32 x 1-1/4")	1	10
7	A	PAR9938-NLA	NLA - Screw, Phillips, Pan Head (10-24 x 1")	1	4
8	A	1097286	Threaded Insert (#10-24)	1	4
9	C	IHPARSPLW	Panel, Access - Left	1	1
10	C	IHPARSPRW	Panel, Access - Right	1	1
11	D	1165912	Cover, Side - Right	1	1
11	D	1165913	Cover, Side - Left	1	1
12	D	1165914	Cover, Rear	1	1
13	D	1096986-NLA	NLA - Panel, Back Access	1	1
NOTE: A - Models IH3602GXL, IH3752G, and IH3802G. B - Models IH3602GXL C - Models IH3752G and IH3802G. D - Models IH6302G and IH6302GADH.					