


Aquatec 900 Toilet Seat Raiser

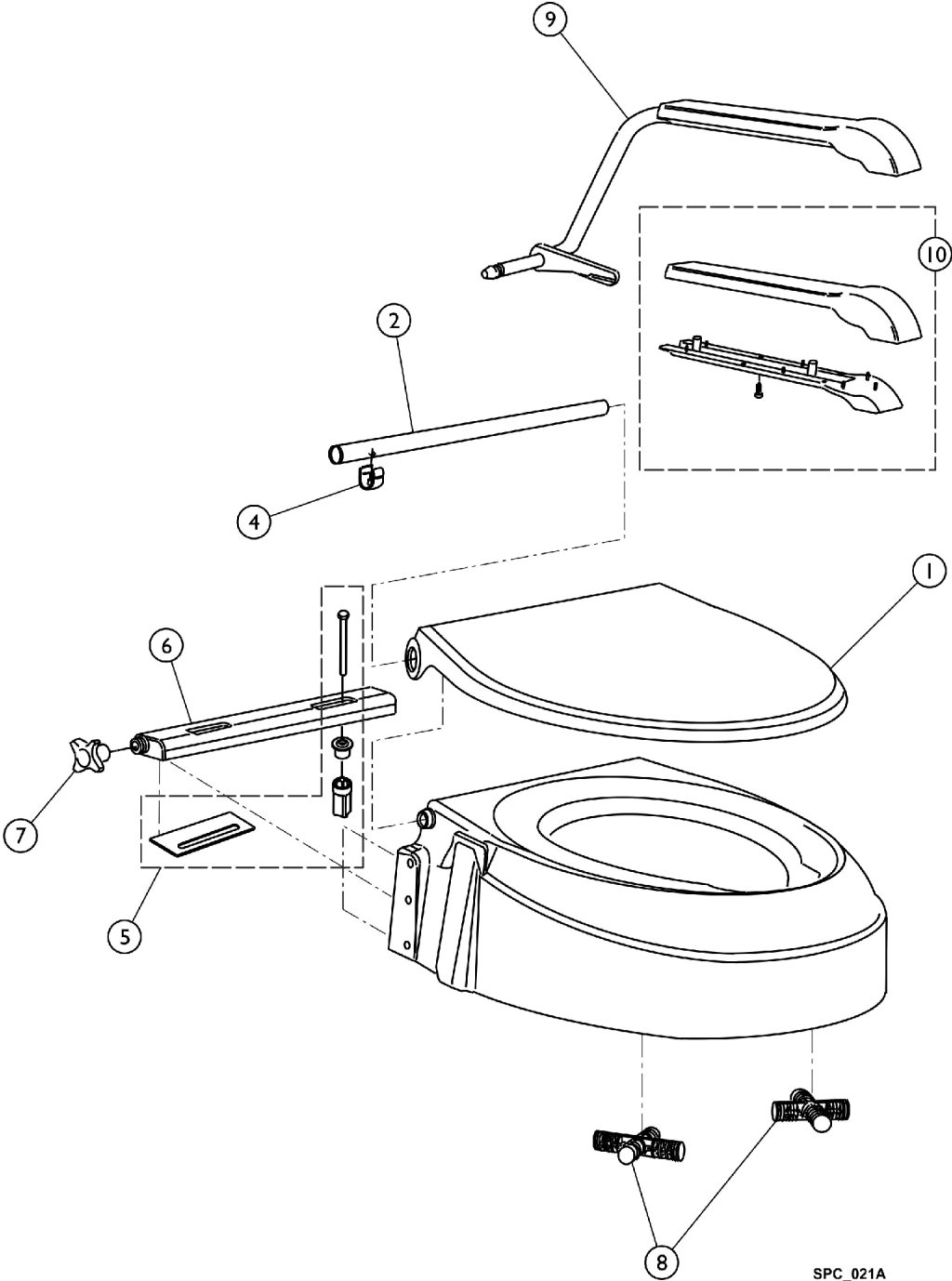

SPC_021A

Aquatec 900 Toilet Seat Raiser

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		A70170	Lid, Seat	1	1
2		72055	Tube, Rear, Lid	1	1
3		70048	Plug, Tube, Lid	1	2
4		70162	Clip, Tube, Lid	1	2
5		16565	Kit, Crossbar Installation, Lid	2	1
5		16582	Screw, Quick Tighten	1	2
5		1470909	Tape, Adhesive	1	2
6		70107	Crossbar, Seat	1	1
7		1470275	Knob, Four Point, Seat	1	2
8		70132	Cross, Attachment (Pair)	2	1
9		10105-10-2	Armrest (Pair)	2	1
10		A10158	Kit, Armrest Pad Upper/Lower with Screws	1	2