
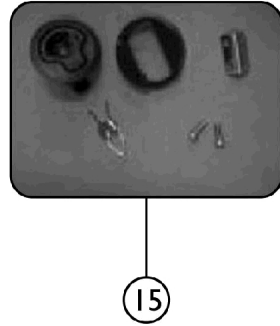
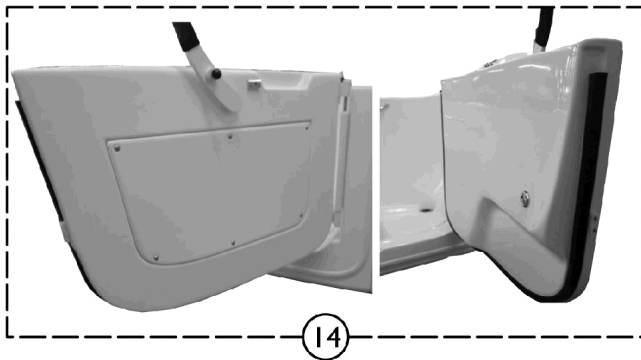
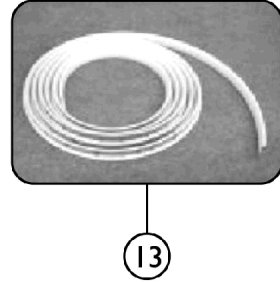
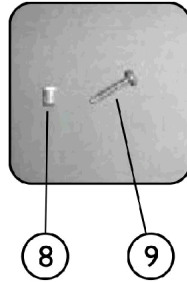
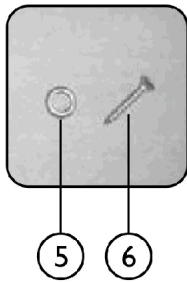
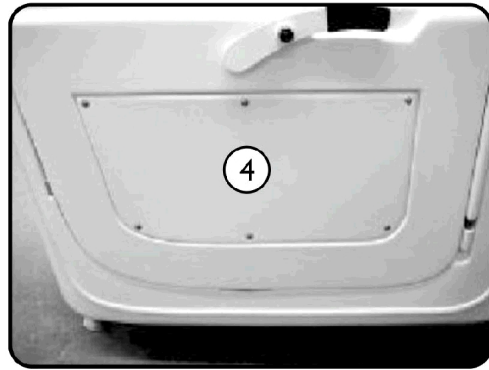
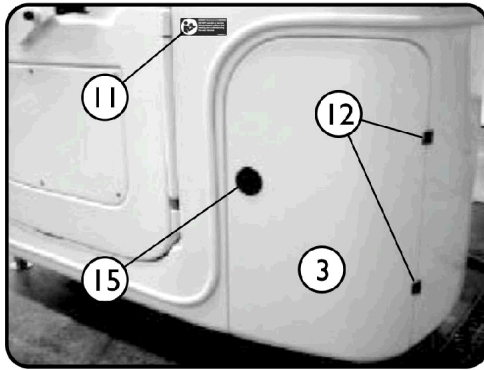
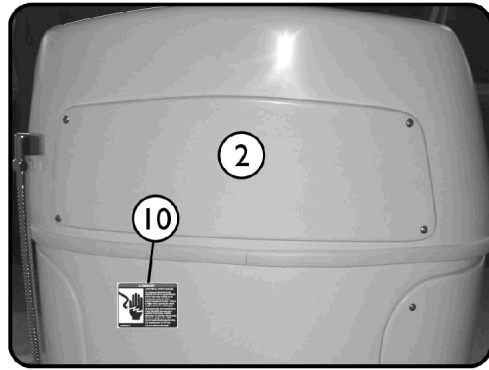


IH3652G - Access Doors and Panels

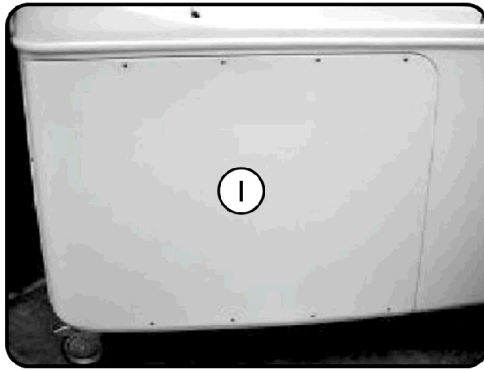

SICC_061A

IH3652G - Access Doors and Panels

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1136825-NLA	NLA - Back Access Panel	1	1
2		1136826-NLA	NLA - Top End Panel	1	1
3		1136827	Disinfectant Door	1	1
4		1136824	Door Panel	1	1
5		IHPAR9907	Washer, #8 Countersunk Finishing	1	1
6		IHPAR9908	Screw	1	1
7		1161794	Door Key	1	1
8		1097286	Threaded Insert (#10-24)	1	1
9		PAR9938-NLA	NLA - Screw, Phillips, Pan Head (10-24 x 1")	1	1
10		1167464	Decal, Warning, Electrical Shock	1	1
11		1167463	Decal, Warning, Read Manual	1	1
12		PAR5410	Hinge, Access Door	1	2
13		1143642	Trim Kit	1	1
14		1161237-NLA	NLA - Bath Door Assembly	1	1
15		1136830	Latch Assembly, Disinfectant Door	1	1
NOTE: A - 18 Feet required.					