


IH3652G - Door Handle and Safety Latch Components

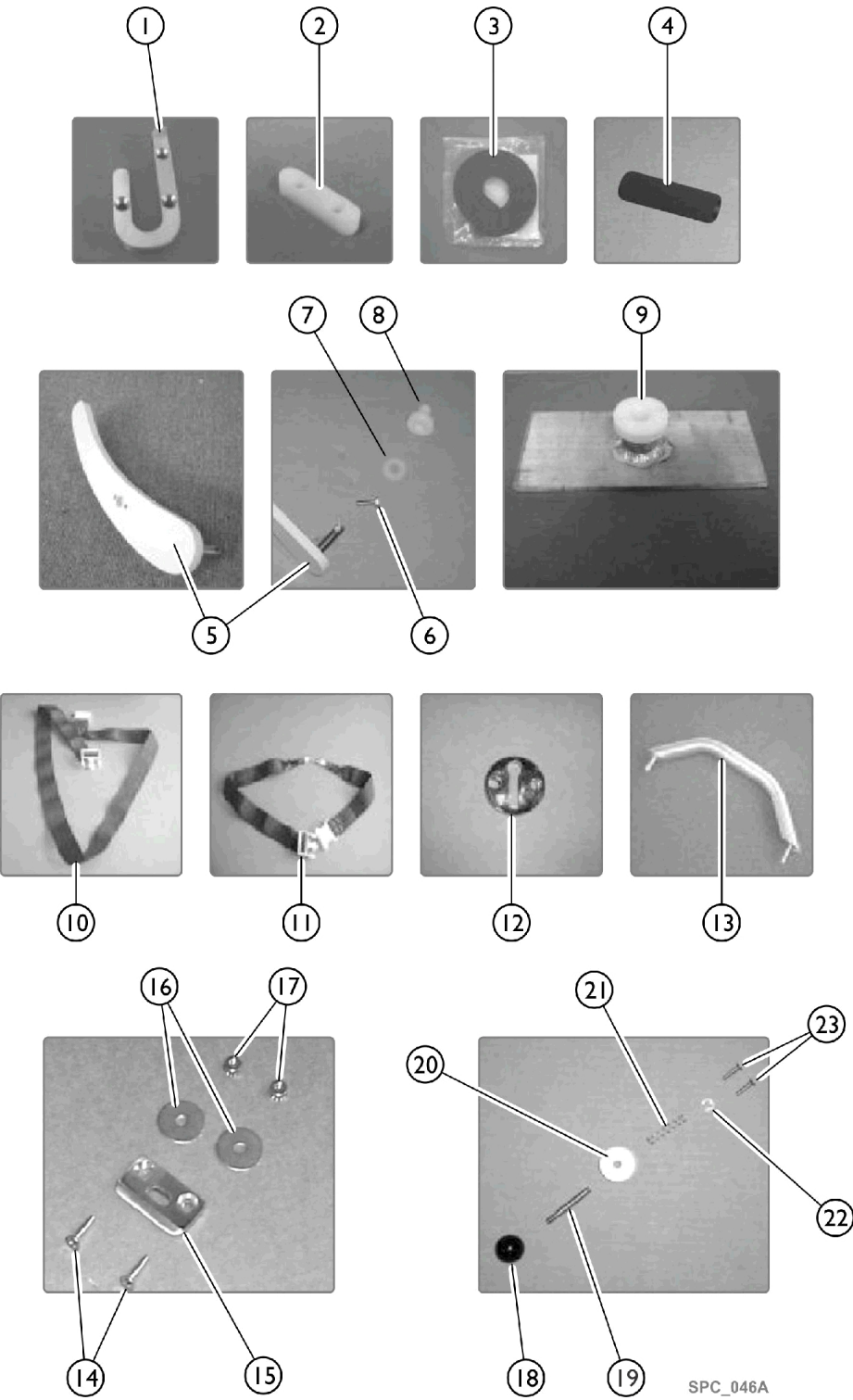

IH3652G - Door Handle and Safety Latch Components

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1127623	Door Receiver	1	1
2		1127624-NLA	Door Stop	1	1
3		1143641	Door Seal Kit	1	1
4		TRP1036	Door Handle Grip	1	1
5		1149638	Door Handle	1	1
6		PAR9901	Bolt (5/16-18 x 1")	1	1
7		PAR9240A	Washer, Nylon	1	1
8		IHPAR3621	Bushing, Door Handle	1	1
9		1127608	Backer, Handle w/ Threaded Bushing	1	1
10		PAR9268ABC	Chest Safety Harness	1	1
11		PAR9260	Seat Safety Harness	1	1
12		PAR9269B	Tub Catch, Chrome	1	1
13		IHPAR9261	Grab Bar (10")	1	1
14		IHPAR9912	Screw, Phillips, Flat Head (#10-24 x 1")	1	2
15		PAR3607	Striker Plate, Safety Latch	1	1
16		IHPAR9913	Washer, Fender	1	2
17		IHPAR9910	Nut (#10-24)	1	2
18		PAR3606	Knob, Safety Latch	1	1
19		PAR3601	Pin, Safety Latch	1	1
20		PAR3603B	Collar, Safety Latch	1	1
21		PAR3604	Spring, Safety Latch	1	1
22		PAR3605	Cir-Clip	1	1
23		IHPAR9911	Screw, Phillips, Flat Head (#10-24 x 3/4")	1	2