


IH3652G - Door Probe Locking Assembly

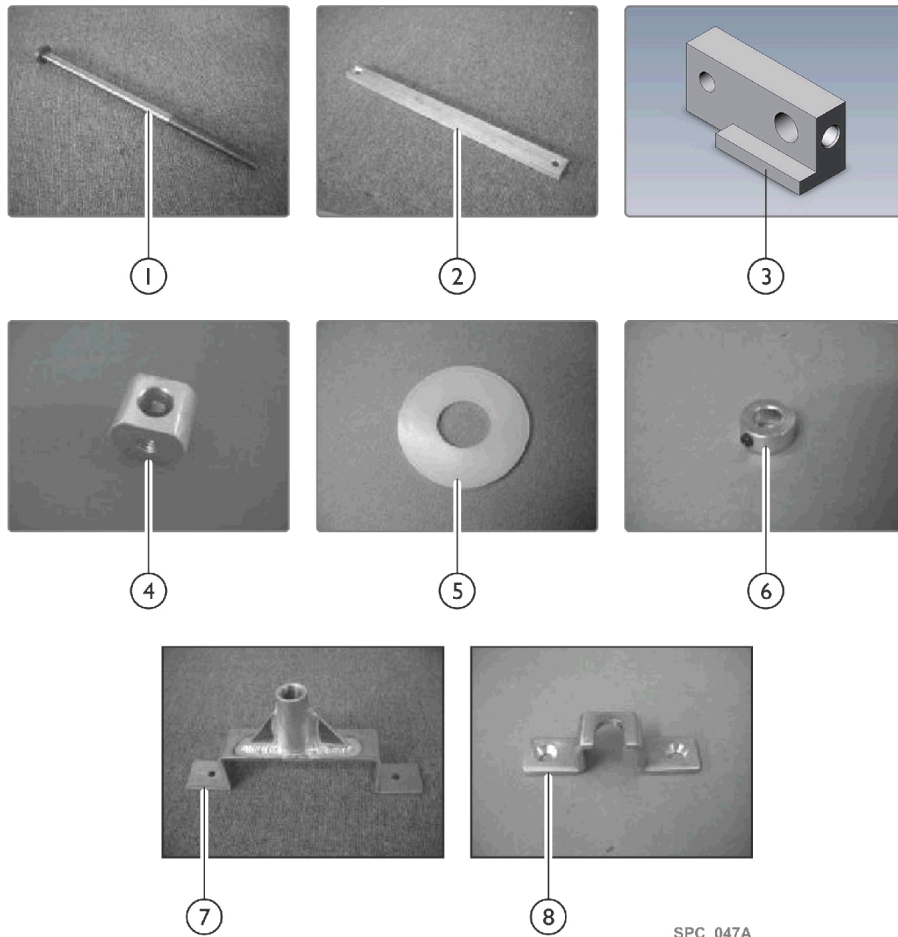

SPC_047A

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1127604	NLA - Door Probe	1	1
2		1127602	Probe Stem	1	1
3		1127601	Probe Cam Assembly	1	1
4		1127603	Probe Linkage Assembly	1	1
5		PAR9240A	Washer, Nylon	1	1
6		1127617	Probe Lock Collar	1	1
7		1127607	NLA - Probe Guide	1	1
8		1127605	Probe Striker Plate	1	1