


Center Mount Rigging w/ Adductors Mounting Hardware (CTMT-F/CTMT-FJ) (12" - 22" Wide) Chairs with ADJASBA Seats

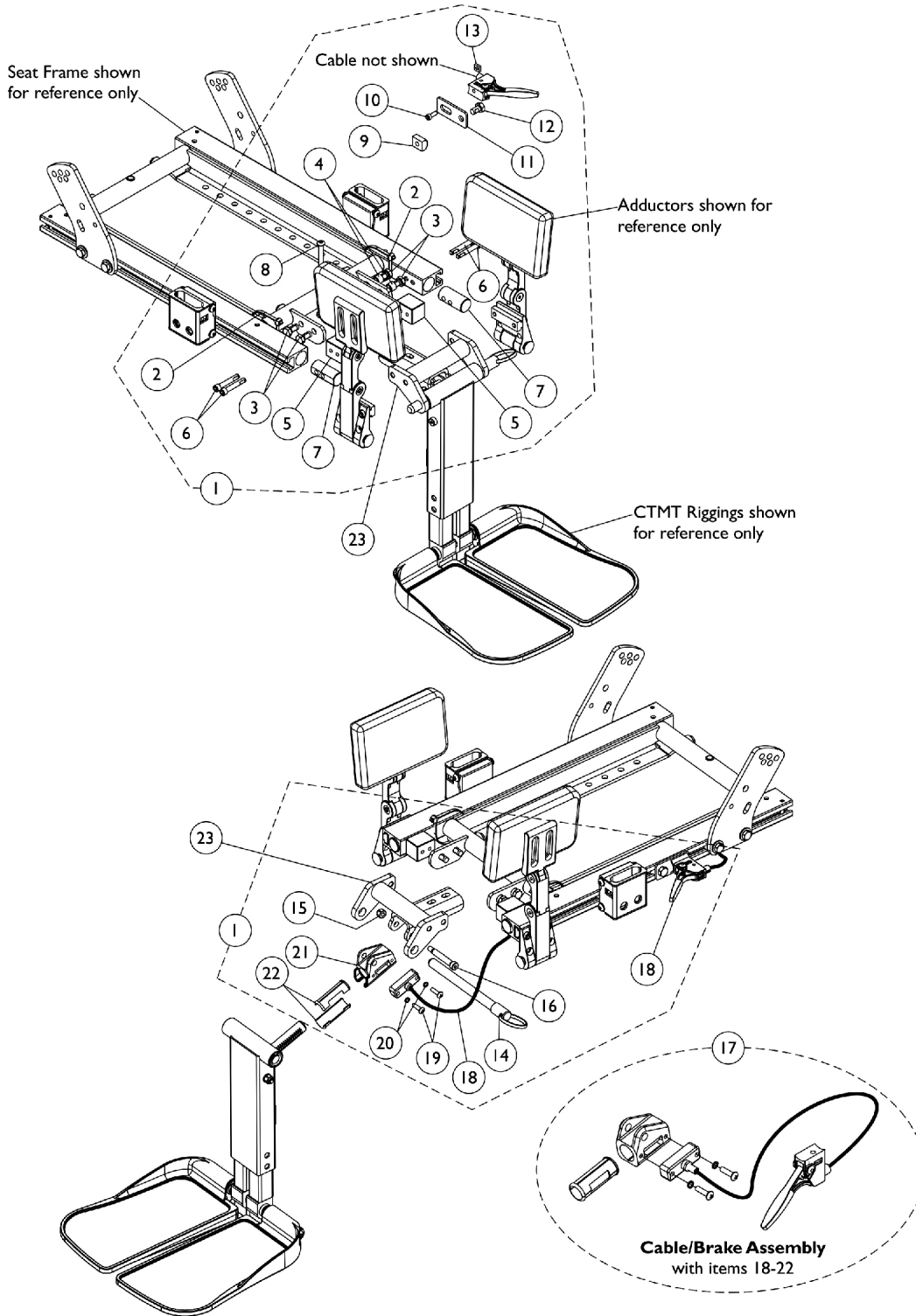

Center Mount Rigging w/ Adductors Mounting Hardware (CTMT-F/CTMT-FJ) (12" - 22" Wide) Chairs with ADJASBA Seats

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1169214-NLA	NLA - Kit, CTMT-F/CTMT-FL/CTMT-FJ Center Mount Riggings w/ Adductors Mounting Hardware	1	1
2			Tie Wrap (11-1/2" L)	1	2
3			Screw, Hex Head w/ Patch (5/16-18 x 5/8")	1	4
4			Spacer (5/16 x 7/16 x 3/8")	1	2
5			Block, Threaded	1	2
6			Screw, Socket Head Cap w/ Patch (1/4-20 x 1-1/2")	1	4
7			Plug, Support	1	2
8			Screw, Socket Button Head (1/4-20 x 1-1/2")	1	1
9			T-Nut (5/16-18)	1	1
10			NLA - Screw, Socket Head (#10-32 x 1/2")	1	1
11			Bracket, Release Handle Mounting, Black	1	1
12			NLA - Screw, Hex Head w/ Patch (5/16-18 x 1/2")	1	1
13			Nut, Square (#10-32)	1	1
14			NLA - Pin, Quick Release (5" Long)	1	1
15			Locknut (1/4-20)	1	1
16			Screw, Shoulder Hex Socket Head (5/16" x 1-5/8", 1/4-20)	1	1
18			Cable/Brake Assembly	1	1
19			Screw, Phillips Pan Head (#10-32 x 11/16")	1	2
20			NLA - Washer, Lock (3/16 x 5/16 x 1/32")	1	2
21			Housing, Ratchet	1	1
22			Bearing, Glide	1	2
17	B	1121079	Cable/Brake Assembly w/ Ratchet Housing and Glide Bearing	1	1
18		1113622	Cable/Brake Assembly	1	1
19		40163X008	Screw, Phillips Pan Head (#10-32 x 11/16")	1	2
20		1028527-NLA	NLA - Washer, Lock (3/16 x 5/16 x 1/32")	1	2
21		1113389	Housing, Ratchet	1	1
22		1113383	Bearing, Glide	1	2
23		1163027	Support, Center Mount Rigging, Black (12" - 19" D)	1	1
23		1163029	Support, Center Mount Rigging, Long, Black (19" - 22" D)	1	1

NOTE: A - Includes items 2-16 and 18-22.
B - Includes items 18-22.