



Miscellaneous Accessories


NOTE: Chest Positioning Straps share the same hardware that attaches a Back Upholstery

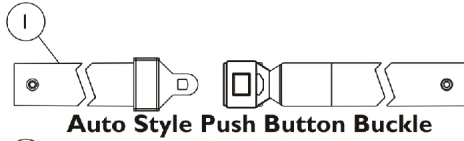

Auto Style Push Button Buckle

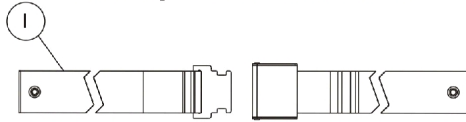


Airline Style Buckle Positioning Strap

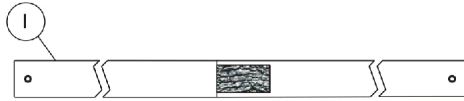


Hook and Loop Style Buckle Positioning Strap


**Positioning Straps
(For ADJASBA Seating
without TRRO Option)**

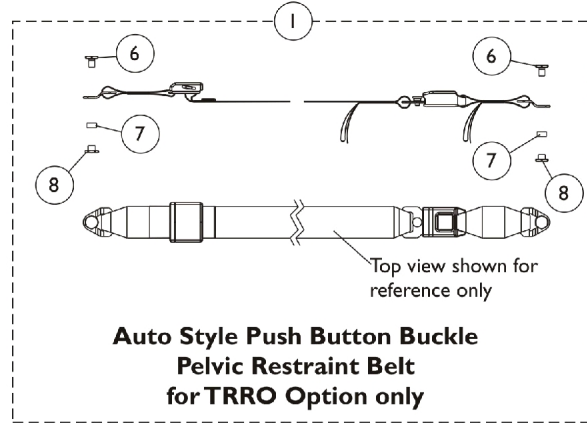

**Auto Style Push Button Buckle
Pelvic Restraint Belt
for TRRO Option only**

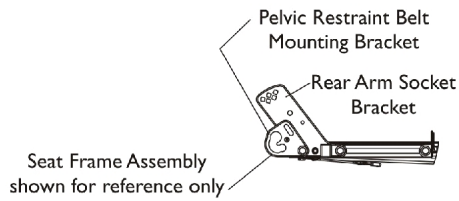

**Pelvic Restraint Belt Mounting Location
(For ADJASBA Seating w/ TRRO Option)**

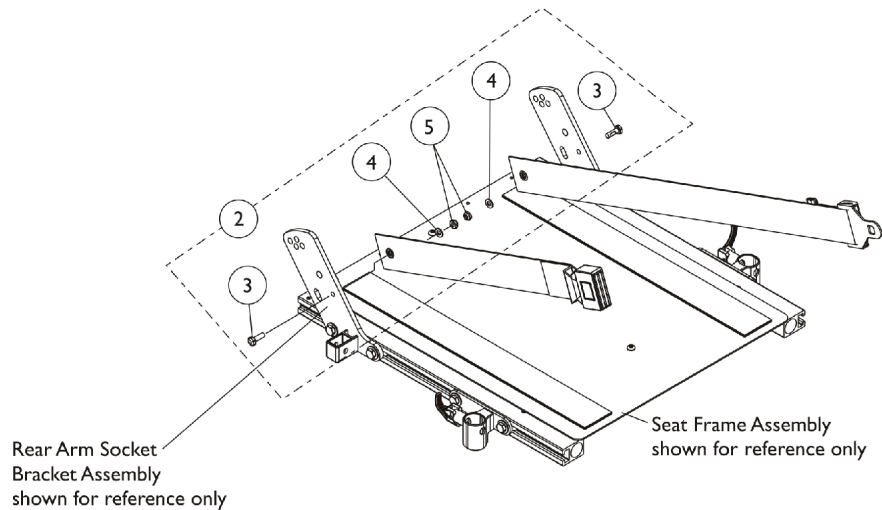

**Seat Positioning Strap
Mounting Hardware
(For ADJASBA Seating)**

Miscellaneous Accessories

NOTE: Chest Positioning Straps share the same hardware that attaches a Back Upholstery

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		See Chart	#1 Auto Style Push Button Positioning Strap	1	1
1		See Chart	#1 Airline Style Positioning Strap	1	1
1		See Chart	#1 Hook and Loop Style Positioning Strap	1	1
1	A	See Chart	#1 Pelvic Restraint Belt w/ Auto Style Buckle (TRRO Option Only)	1	1
2	B	1124385	Kit, Seat Positioning Strap Mounting Hardware (both LT & RT sides)	1	1
3			Screw, Hex Head w/ Patch (1/4-20 x 3/4")	1	2
4			Washer (1/4 x 1/2 x 1/16")	1	2
5			Locknut (1/4-20)	1	2
6		1134055-NLA	NLA - Screw, Hex Head (3/8-24 x 5/8")	1	2
7		1134054-NLA	NLA - Bushing, Nylon (1/2 x 9/16 x 9/32")	1	2
8		1134056-NLA	NLA - Nut, Barrel (3/8-24)	1	2
	C	40176X010-NLA	NLA - Pump, Tire	1	1

NOTE: A - Before ordering a replacement Pelvic Restraint Belt for TRRO Option, Please read the information provided on page titled "TRRO and TRBKTS Option Clarification Reference" page
 B - Includes items 3-5
 C - Not Shown

#1 Auto Style Push Button Positioning Strap

Chair Width	Part Number	Seat/Chest Positioning Strap w/ Auto Style Buckle Length (Eyelet to Eyelet)
12" - 13" Wide	1036527	33" L
14" - 19" Wide	1004310	48" L
20" - 22" Wide	1034625	61" L

#1 Airline Style Positioning Strap

Chair Width	Part Number	Seat/Chest Positioning Strap w/ Airline Style Buckle Length (Eyelet to Eyelet)
16" - 19" Wide	TAGTS1034751-NLA	46" L
20" - 22" Wide	1034752-NLA See Note(s) A	61" L

NOTE: A - 1034752 Airline Buckle Style is No Longer Available. Use Auto Style part number 1034625 as a replacement.

Miscellaneous Accessories

NOTE: Chest Positioning Straps share the same hardware that attaches a Back Upholstery

#1 Hook and Loop Style Positioning Strap

Chair Width	Part Number	Seat/Chest Positioning Strap w/ Hook and Loop Style Buckle Length (Eyelet to Eyelet)
12" - 15" Wide	1034744 See Note(s) A	45" L
16" - 22" Wide	1034745 See Note(s) A	61" L
NOTE: A - Hook and Loop Style Positioning Straps for power chairs is used for chest positioning only.		

#1 Pelvic Restraint Belt w/ Auto Style Buckle (TRRO Option Only)

Chair Width	Part Number	Pelvic Restraint Belt w/ Auto Style Buckle Length For TRRO Option Only (Bushing Hole to Bushing Hole Length)
12" - 16" Wide	1168513	37" L
16" - 22" Wide	1168514 See Note(s) A	55" L
NOTE: A - Includes the Pelvic Restraint Belt and items 8-10		