


Proportional Attendant Control Joystick (PACM6)

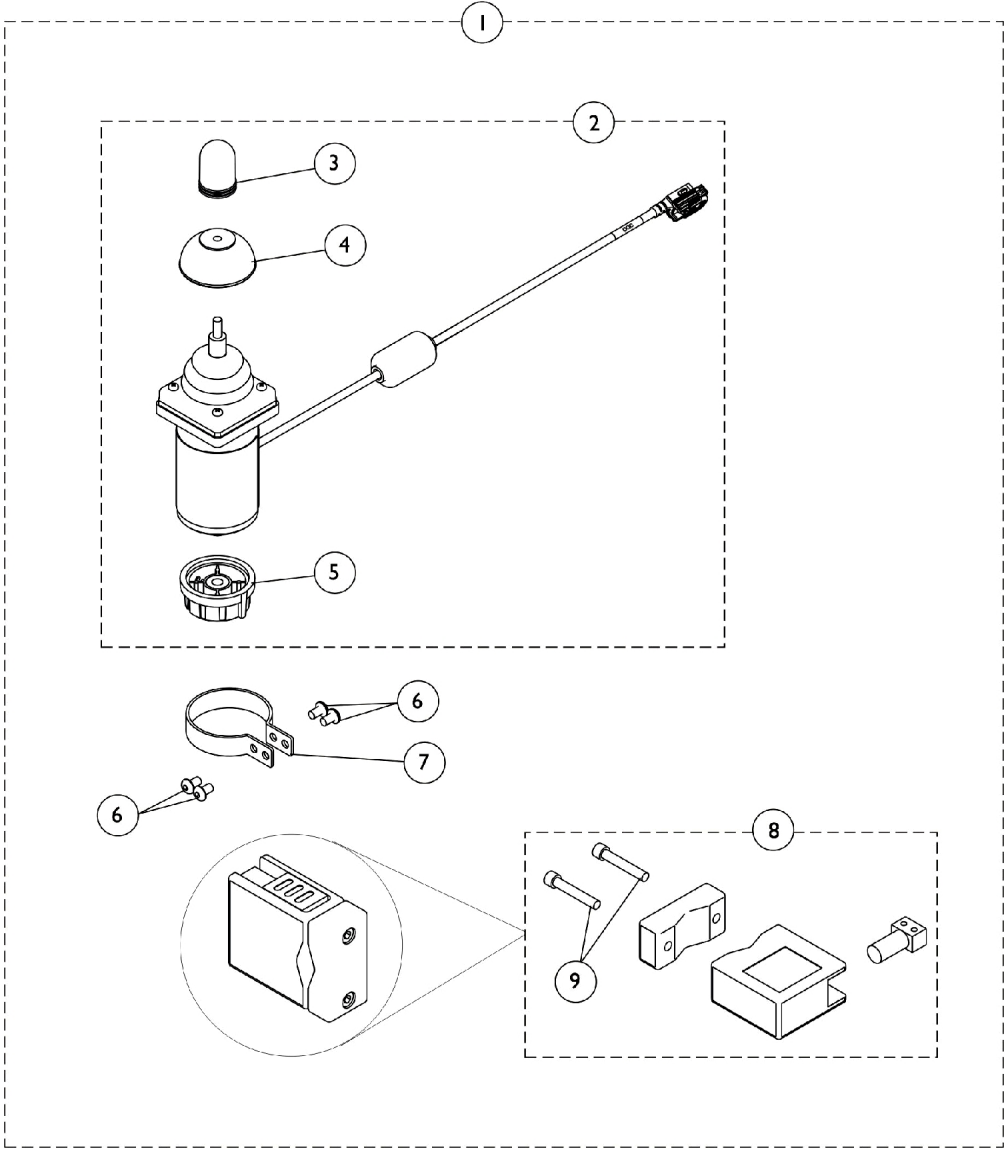

Proportional Attendant Control Joystick (PACM6)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1147097	Kit, Proportional Attendant Control Joystick w/ Bracket & Clamp - MK6i (PACM6) (72" L)	1	1
2	B	1136888	Joystick, Proportional Attendant Control (PACM6) (72" L)	1	1
2	C,B	1167927	Joystick, Proportional Attendant Control (PACM6) (38" L)	1	1
3		1040217	Knob, Joystick Rubber, Black	1	1
4		1040218	Skirt, Joystick Rubber, Black	1	1
5		1114232	Knob, Control	1	1
6		1021460	Screw, Socket Button Head (#10-32 x 5/16")	1	4
7		1025048	Clamp	1	1
8		1123244	Bracket Assembly	1	1
9		0140406-NLA	NLA - Screw, Socket Head, Cap (5/16-18 x 1-1/2")	1	2
NOTE: A - Includes item 2 part #1136888 and items 6-9. For FDX models, order item 2 part #1167927 and items 6-9 individually. B - Includes Joystick plus items 3-5. C - FDX models use part #1167927 Proportional Attendant Control Joystick that has a 38" long cable length.					