


Arm Mounting Hardware

Conventional and Contoura (CGR) & (CGTR/CGTRE) Seating (16"-24" Wide)

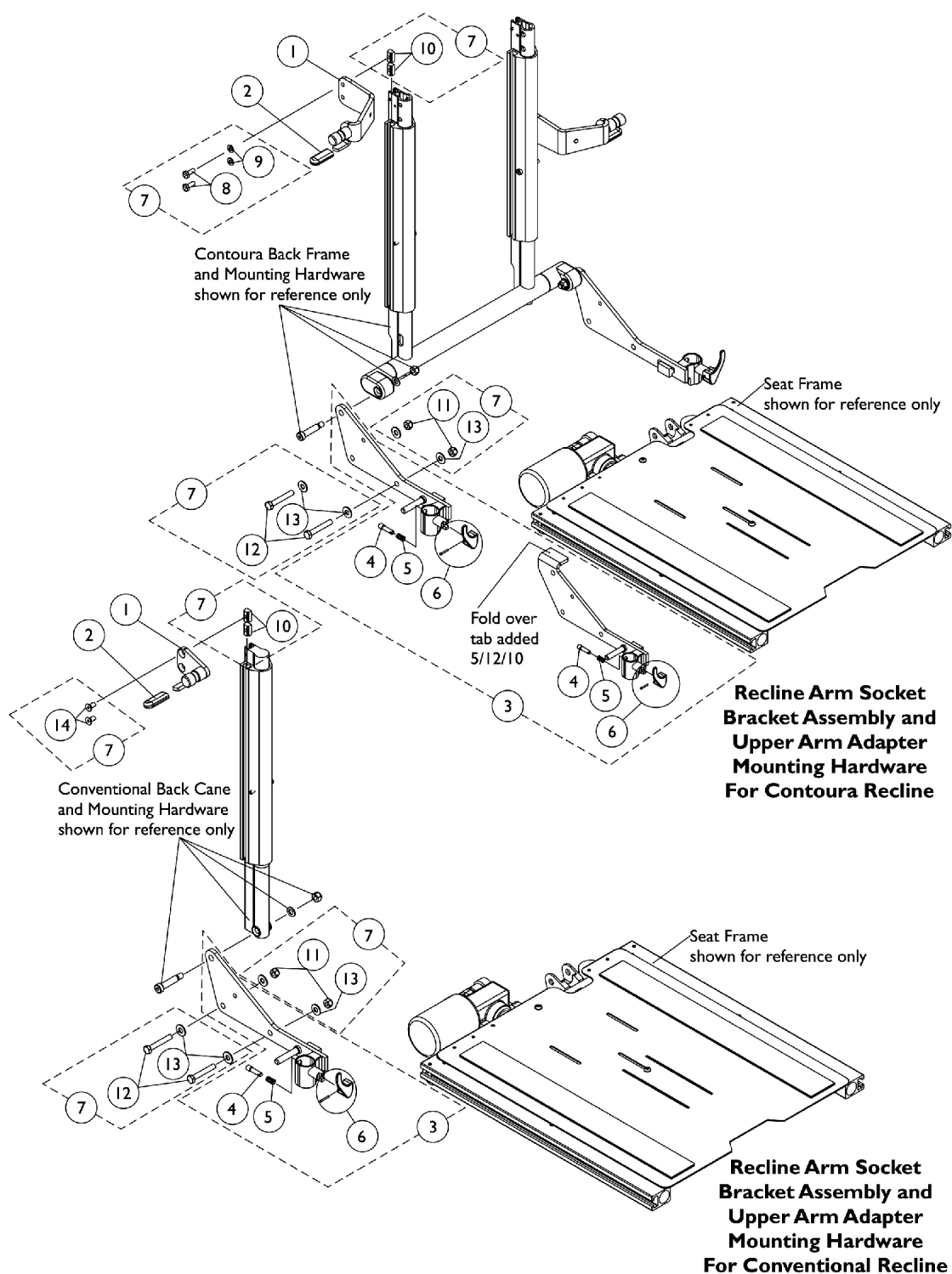

Arm Mounting Hardware Conventional and Contoura (CGR) & (CGTR/CGTRE) Seating (16"-24" Wide)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		See Chart	#1 Adapter, Contoura Recline Upper Arm - Before 5/15/09	1	1
1		See Chart	#1 Adapter, Curve Contoura Recline Upper Arm - After 5/14/09	1	1
1		1122595	Adapter, Conventional Recline Upper Arm - Right	1	1
1		1125809	Adapter, Conventional Recline Upper Arm - Left	1	1
2		1006490	Tip, Plastic, Black	1	1
3	A,C	1124682	Recline Arm Socket Bracket Assembly, Black - Right	1	1
3	A,C	1124683	Recline Arm Socket Bracket Assembly, Black - Left	1	1
3	A,D	1168694	Recline Arm Socket Bracket Assembly, Black - Right	1	1
3	A,D	1168695	Recline Arm Socket Bracket Assembly, Black - Left	1	1
4		1028594	Pin, Arm Lock, Black	1	1
5		47471X074	Spring, Arm Lock	1	1
6		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
7	B	1144582	Kit, Upper Arm Adapter/Recline Arm Socket Bracket Assembly Mounting Hardware	1	1
8			Screw, Hex Head w/ Patch (1/4-20 x 5/8")	1	2
9			Washer (1/4 x 1/2 x 1/16")	1	2
10			T-Nut (1/4-20)	1	2
11			Locknut (5/16-18)	1	2
12			NLA - Screw, Hex Head (5/16-18 x 2-1/4")	1	2
13			Washer, Steel (5/16 x 3/4 x 1/16")	1	4
14			NLA - Screw, Socket Flat Head w/ Patch (1/4-20X1/2")	1	2
NOTE: A - Includes Recline Arm Socket Bracket and items 4 thru 6. B - Includes items 8 thru 14. Hardware accomodates the mounting of item 1 and item 3. C - Used on Conventional, Contoura and (Curve Contoura Backs Before 5/12/10) D - The Recline Arm Socket Assembly with a fold over tab was introduced for all Curve Contoura Backs 5/12/10					

#1 Adapter, Contoura Recline Upper Arm - Before 5/15/09

Adapter Side	16" Wide	18" Wide	20" Wide	22" Wide	24" Wide
Contoura Right	1131416-NLA	1122582-NLA	1122625-NLA	1131418-NLA	1131420-NLA
Contoura Left	1131417-NLA	1125807-NLA	1125808-NLA	1131419-NLA	1131421-NLA

Arm Mounting Hardware
Conventional and Contoura
(CGR) & (CGTR/CGTRE) Seating (16"-24" Wide)

#1 Adapter, Curve Contoura Recline Upper Arm - After 5/14/09

Adapter Side	16" Wide	18" Wide	20" Wide	22" Wide	24" Wide
Curve Contoura - Right	1161802	1161655	1161660	1161796	1161820
Curve Contoura - Left	1161803	1161656	1161661	1161797	1161821