


Carriage Assembly Maxi-Lock

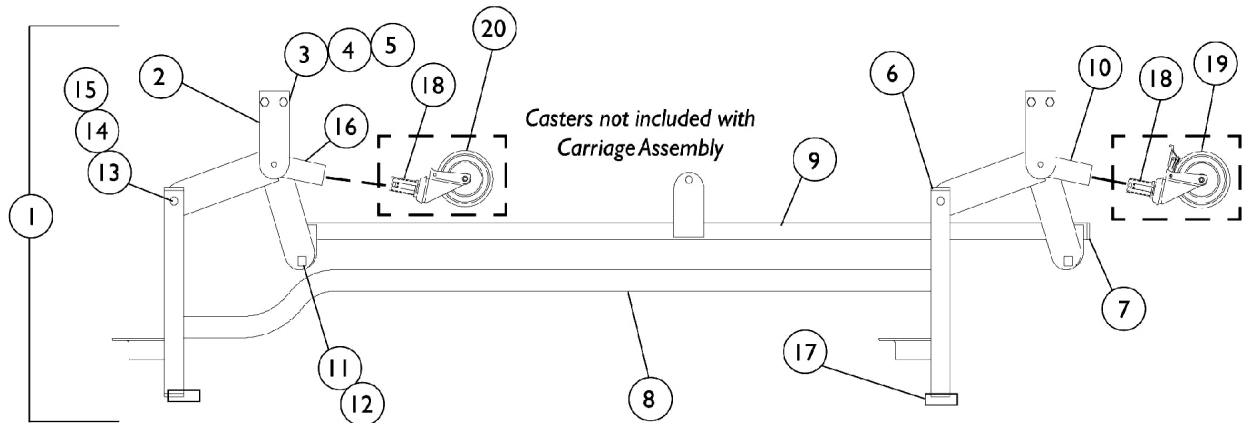

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1114317-NLA	NLA - Carriage Maxi-Lock Assembly	1	1
2		025187	Support, Lift Crank	1	4
3		000037	NLA - Nut, Keps (5/16-18)	1	4
4		000037	NLA - Nut, Keps (5/16-18)	1	4
5		001758	Screw, Hex Head (5/16-18 x 1")	1	8
6		1012756	Plug Button, Black (1-1/4")	1	2
7		020717	Glide, Cap	1	2
8		026586-NLA	NLA - Frame, Carriage Maxi-Lock	1	1
9		025184	Lift Bar	1	1
10		026588-NLA	NLA - Crank, Lift - Foot End	1	1
11		001004	Bolt, Square Head Shoulder (5/16-18 x 7/8")	1	2
12		53000M434	Locknut (5/16-18)	1	2
13		026725	Washer, Flat (3/8")	1	4
14		51000M743	Pin, Hitch (3/8 x 1-3/16")	1	4
15		026976	Pin, Clevis (3/8 x 1-3/4")	1	4
16		026589-NLA	NLA - Crank, Lift - Head End	1	1
17		1131077	Foot, Rubber	1	1
18		51000M083	Socket, Caster	1	4
19		1123551	Caster Assembly, Swivel w/ Brake (3")	1	2
20		1123549	Caster Assembly, Swivel w/o Brake (3")	1	2

NOTE: A - Includes: Items 2-17.