


# Carriage Assembly SC900

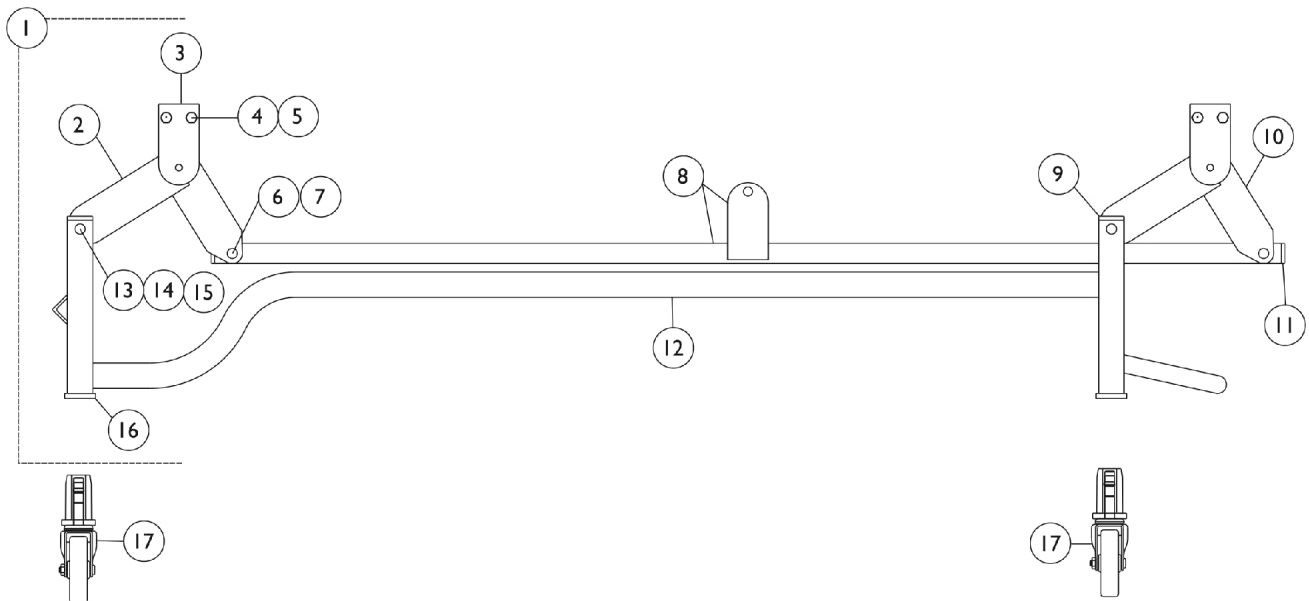

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,C	1113725-NLA	NLA - Carriage Assembly - SC900	1	1
1	A,C	1133049-NLA	NLA - Carriage Assembly, Taupe	1	1
2		027016	Crank, Lift - Head End	1	1
3	C	027010-NLA	NLA - Crank Assembly, Lift Support, Brown	1	4
3	C	1133050	Support, Lift Crank	1	4
4		000037	NLA - Nut, Keps (5/16-18)	1	8
5		001758	Screw, Hex Head (5/16-18 x 1")	1	8
6		027018	Pin, Clevis (3/8 x 2-3/4")	1	2
7		027024	Pushnut, Fastener	1	2
8		027001	Bar, Lift	1	1
9		027047	Plug Button with Hole (1-1/4" Square)	1	4
10		026999	Crank, Lift - Foot End	1	1
11		020717	Glide, Cap	1	2
12		027008-NLA	NLA - Frame, Carriage	1	1
13		026976	Pin, Clevis (3/8 x 1-3/4")	1	4
14		026725	Washer, Flat (3/8")	1	4
15		51000M743	Pin, Hitch (3/8 x 1-3/16")	1	4
16		51000M083	Socket, Caster	1	4
17		027027	Kit, Caster 3" Swivel w/ Bracket	1	2
17	B	1102920	Caster, Swivel w/o Brake (4" Black)	1	2
17	B	1102918	Caster, Swivel w/ Bracket (4" Non-Marking)	1	2

NOTE: A - Includes: Items 2-16.  
 B - These items are optional.  
 C - These items are painted. Please make sure you select the correct color option.