


Motors & Components

Full & Semi Electric

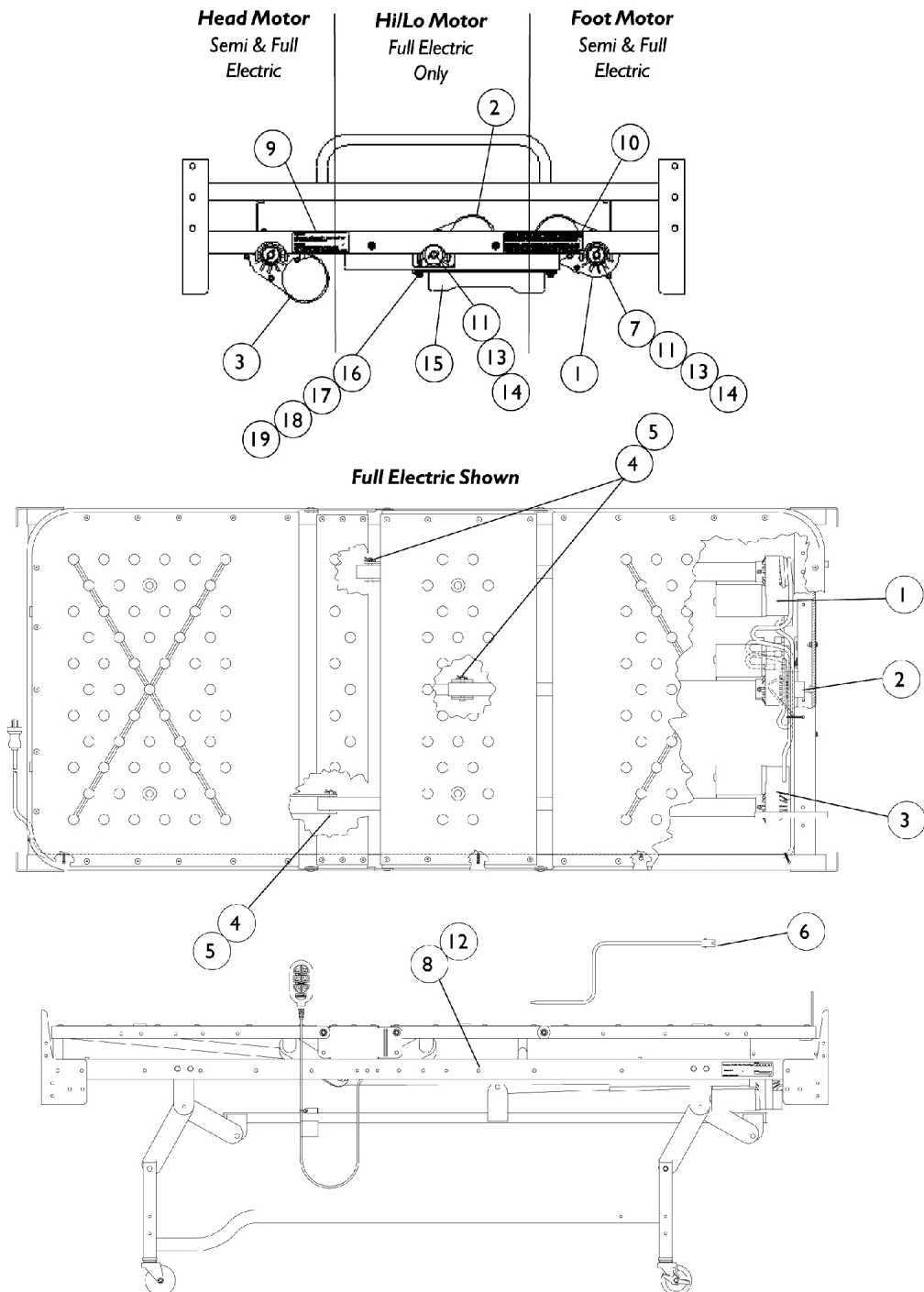

Motors & Components

Full & Semi Electric

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	D	026754-NLA	NLA - Motor, Foot, Dark Brown - IH720/SC900	1	1
1	D	1133053-NLA	NLA - Motor, Foot, White	1	1
2		026755-NLA	NLA - Motor, Hi/Lo	1	1
2		027038-NLA	NLA - Motor, Hi/Lo - IH720	1	1
2		027019-NLA	NLA - Motor, Hi/Lo, Dark Brown - SC900	1	1
3	D	026756-NLA	NLA - Motor, Head, Dark Brown - IH720/SC900	1	1
3	D	1133055-NLA	NLA - Motor, Head, White	1	1
4		005952	Pin, Clevis (5/16 x 2-1/8")	1	3
5		51000M743	Pin, Hitch (3/8 x 1-3/16")	1	3
6	D	5880M004	Crank, Emergency, Dark Brown	1	2
6	D	1133062	Crank, Emergency	1	2
7		001741	Screw, Hex Head (5/16-18 x 3/4")	1	6
8		000438	Tie Wrap (15-1/2" L)	1	8
9		034058	Decal, Warning	1	1
10		031763	Label, Caution	1	1
11		001008	Washer, Split Lock (5/16")	1	6
12		026874	Cord, Power	1	1
13		001405	Bushing, Flanged, Nylon	1	2
14		001443	Bushing, Shoulder - Pivot	1	2
15	A,D	026757-NLA	NLA - Junction Box w/ Bracket Assembly (Dark Brown)	1	1
15	A,D	1133058-NLA	NLA - Junction Box with Bracket Assembly, Taupe	1	1
16		1102550-NLA	NLA - Junction Box	1	1
17		026727	Bracket, Mounting, Brown	1	1
17	D	027014-NLA	NLA - Bracket, Mounting, Dark Brown	1	1
17	D	1133059-NLA	NLA - Bracket, Mounting, Taupe	1	1
18		54000M515	Screw, Slotted Hex Head Tap (1/4-20 x 3/4")	1	2
19		54000M516-NLA	NLA - Spacer, Round (1/4 x 1/2 x 1/4")	1	2
	B,C	020342	Kit, Hardware	1	1

NOTE: A - Includes: Items 16-19.
 B - Not Shown.
 C - Includes: Items 7, 11, & 13-14.
 D - These items are painted. Please make sure you select the correct color option.