
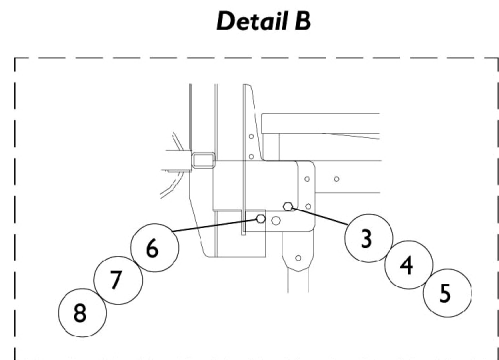
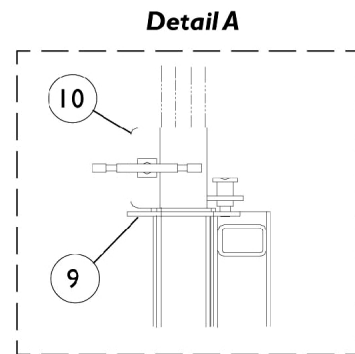


Trapeze

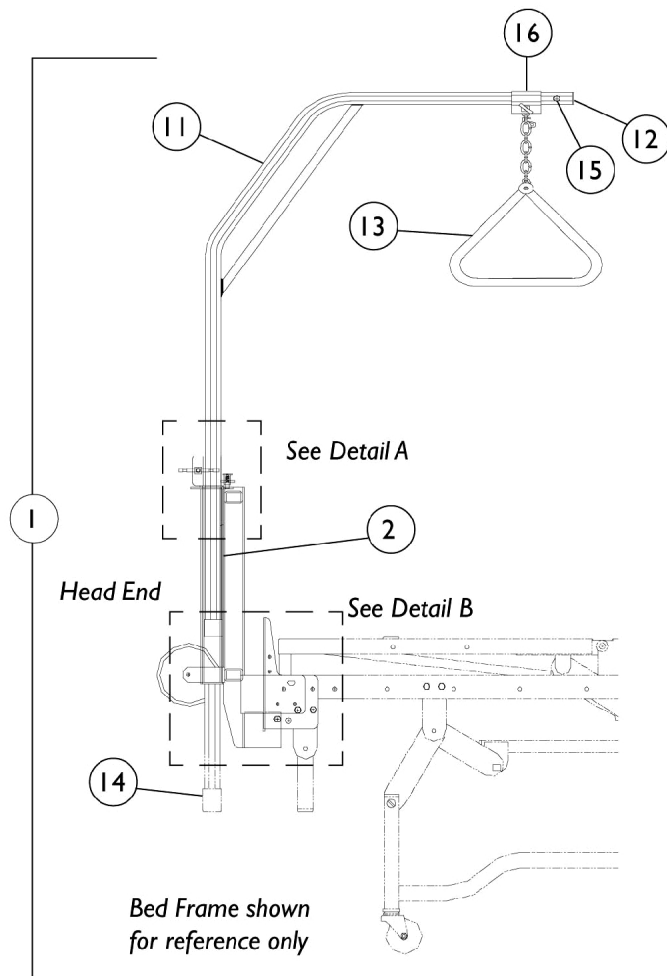

Trapeze

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	026330-NLA	NLA - Trapeze Mounting Frame Assembly	1	1
2		026331	Frame, Trapeze Mounting	1	1
3		004544-NLA	NLA - Screw, Hex Head (5/16-18 x 1-1/2")	1	2
4		031195-NLA	NLA - Spacer (5/16 x 3/4 x 5/16")	1	4
5		000037	NLA - Nut, Keps (5/16-18)	1	2
6		013208-NLA	NLA - Screw, Hex Head (1/4-20 x 1")	1	2
7		007478	Washer (13/32 x 13/16 x 1/16)	1	2
8		000050	Nut, Keps (1/4-20)	1	2
9		026435-NLA	NLA - Adapter, Octagon Trapeze	1	1
10		1009042-NLA	NLA - Clamp, Swivel w/ Plunger and Wing Knob, Chrome	1	1
11		17171X000-NLA	NLA - Bar, "L", Chrome	1	1
12		17290X000-NLA	NLA - Plug Button, Metal (1-1/4")	1	1
13		7941-NLA	NLA - Grab Bar with Chain and Clamp, Chrome	1	1
14		17167X000-NLA	NLA - Bumper, Grey	1	1
15		17125X000-NLA	NLA - Screw, Hex Head (1/4-14 x 5/8")	1	1
16		1011223-NLA	NLA - Clamp Only	1	1
	B,C	026335-NLA	NLA - Trapeze Octagonal Assembly	1	1
	C	1070792	Instruction Sheet	1	1
	C	010196	Clamp, Square	1	1

NOTE: A - Includes: Items 2-15.
B - Includes: Items 11-15.
C - Not Shown.