



Knee Ratchet

Knee Ratchet

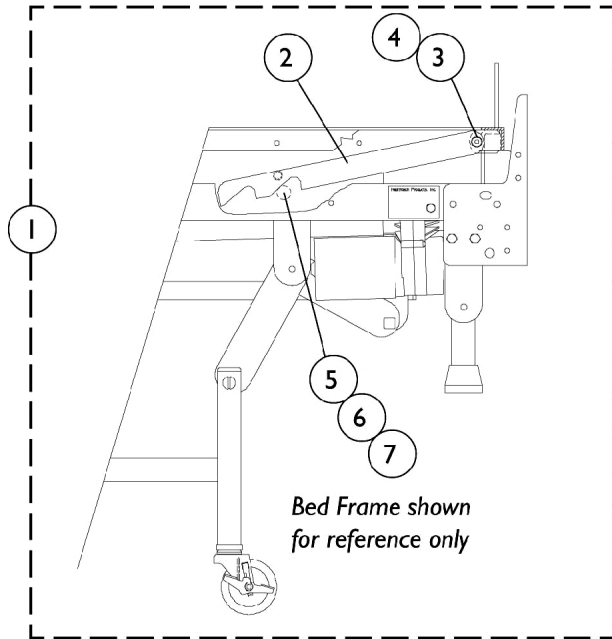

Fixed Link

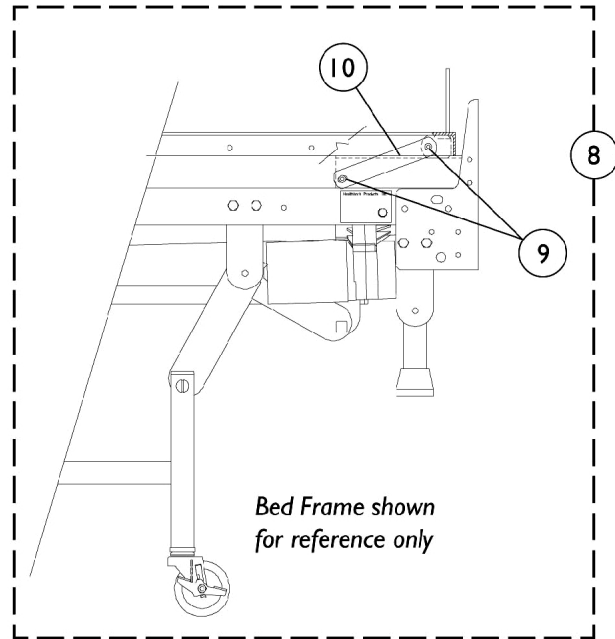

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	026788-NLA	NLA - Knee Ratchet Assembly	1	1
2		025053-NLA	NLA - Ratchet, Leg	1	1
3		025071	Screw, Shoulder Hex Socket Head (3/8 x 1/4, 1/4-20)	1	2
4		000050	Nut, Keps (1/4-20)	1	2
5		027049	Support, Frame Cross	1	1
6		000037	NLA - Nut, Keps (5/16-18)	1	2
7		001741	Screw, Hex Head (5/16-18 x 3/4")	1	2
8	B	026111	Fixed Link Assembly, Knee	1	1
9		000790	Rivet (1/4 x 9/16")	1	4
10		026118	Link, Foot Bow	1	2

NOTE: A - Includes: Items 2-7.
B - Includes: Items 9-10.