


Bed Rails

Model IH5749 - 8 Bar Split Nylasar

Discontinued 6/05

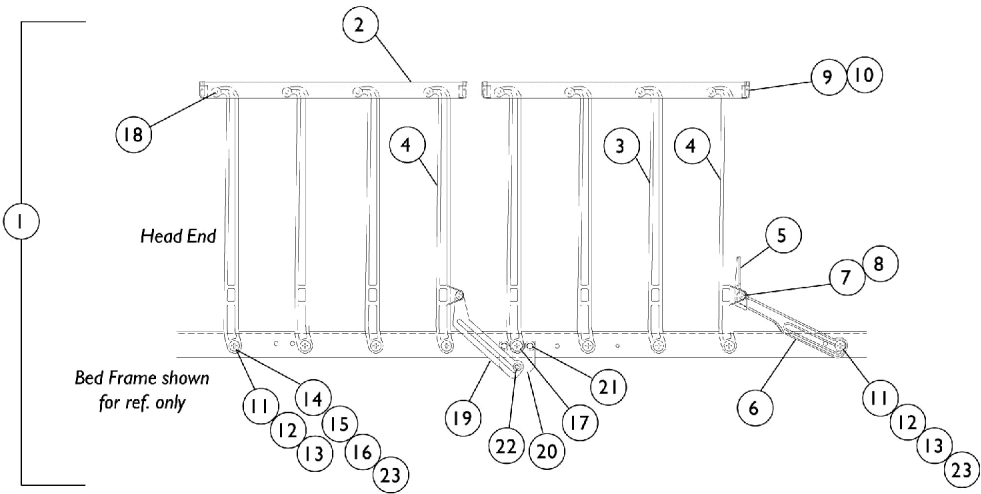

Bed Rails

Model IH5749 - 8 Bar Split Nylasar

Discontinued 6/05

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	025333-NLA	NLA - Safety Side Rail Assembly, 8 Bar, Split - Right	1	1
1	A	025334-NLA	NLA - Safety Side Rail Assembly, 8 Bar, Split - Left	1	1
2		025279-NLA	NLA - Rail, Cover and Guard - 4 Bar	1	2
3		000390-NLA	NLA - Arm, Pivot	1	6
4		000389-NLA	NLA - Arm, Latch	1	2
5		000388-NLA	NLA - Latch, Restraining Side Actuator	1	1
6		000494-NLA	NLA - Arm, Restraining Side Latch	1	1
7		000391-NLA	NLA - Pin, Latch	1	2
8		000173-NLA	NLA - Ring, Retaining (1/4")	1	2
9		000386-NLA	NLA - Cap, Guard Rail	1	4
10	C	009289-NLA	NLA - Adhesive	1	1
11		000397-NLA	NLA - Screw, Shoulder Hex Socket Head (3/8 x 1-1/2", 5/16-18)	1	9
12		53000M434	Locknut (5/16-18)	1	9
13		000387-NLA	NLA - Spacer (3/8 x 3/4 x 1/64")	1	9
14		000394-NLA	NLA - Bearing, Nylon (3/8")	1	16
15		000395	Washer, Steel (7/16 x 13/16 x 1/16")	1	8
16		000393-NLA	NLA - Washer, Rubber (7/16 x 27/32 x 1/8")	1	8
17		000050	Nut, Keps (1/4-20)	1	2
18		033466	Pin, Spring (1/4 x 13/16")	1	8
19		025285-NLA	NLA - Latch, Side Safety	1	1
20		025284-NLA	NLA - Plate, Pivot	1	1
21		007991	Screw, Hex Head (1/4-20 x 5/8")	1	2
22		000587-NLA	NLA - Nut, Push (1/4")	1	1
23		031195-NLA	NLA - Spacer (5/16 x 3/4 x 5/16")	1	10
	B	026715-NLA	NLA - Instruction Sheet, Bed Rail	1	1
	B	1130183	Label, Entrapment Warning	1	1

NOTE: A - Includes: Items 2-23.
B - Not Shown.
C - The Adhesive is No Longer Available as a service part. Please use your own adhesive when installing the Guard Rail Cap.