


Drive Wheels for Non-Free Wheel Hubs (4 Pole Motors Before 8/17/10 US and Canada) & GB Non-Heavy Duty Motors

NOTE: New drive wheel mounting hardware is required when converting from (14" x 3") to (14" x 4") drive wheels.

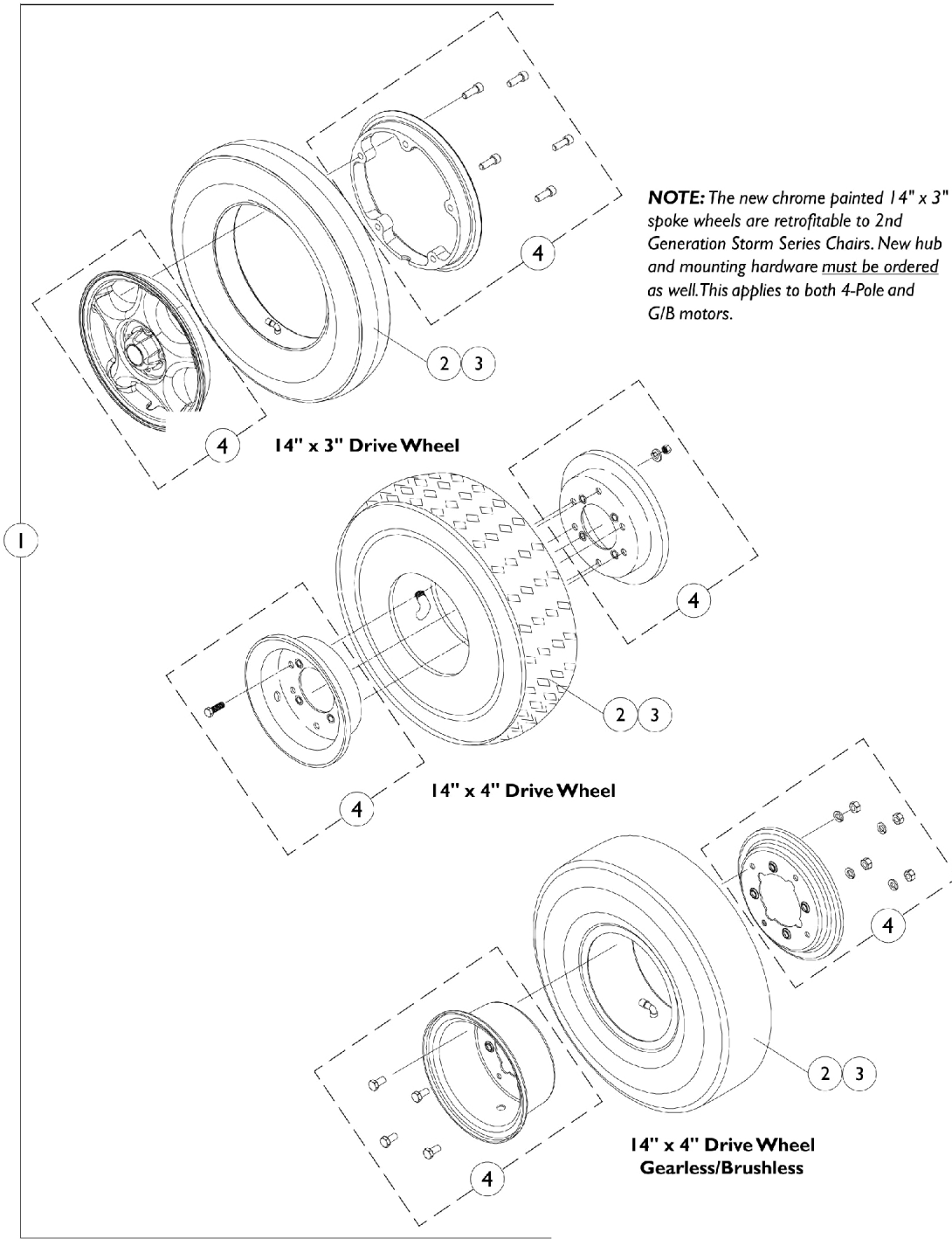

Drive Wheels for Non-Free Wheel Hubs (4 Pole Motors Before 8/17/10 US and Canada) & GB Non-Heavy Duty Motors

NOTE: New drive wheel mounting hardware is required when converting from (14" x 3") to (14" x 4") drive wheels.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1-3		See Chart	#'s 1, 2 and 3 Drive Wheels for Non-Free Wheel Hubs	1	1
4	F	1104882-NLA	NLA - Kit, 8" Rim w/ Hardware (14 x 3" Wheel)	1	1
4	E	1042453-NLA	NLA - Kit, 6" Rim w/ Hardware (14" x 4" Wheel)	1	1
4	B	1078246-NLA	NLA - Kit, 6" Rim w/ Hardware (14" x 4" Wheel)	1	1
	C,D	1101094	Screw, Socket Head w/ Patch (5/16-18 x 7/8")	1	5
	A,C	1024443-NLA	NLA - Screw, Hex Head (5/16-24 x 5/8")	1	6
	A,C	1024442-NLA	NLA - Washer, Lock (5/16 x 9/16 x 1/16")	1	6
	A,C	1024426-NLA	NLA - Nut, Hex (5/16-24)	1	6
<p>NOTE: A - included in rim hardware part number 1042453 and (rim hardware part number 1078246 quantity 4 each) B - For GB motors only C - Not Shown D - included in rim hardware part number 1104882 E - For 4-Pole Motors F - For 4-Pole and GB motors</p>					

Item #'s 1, 2 and 3 Wheel Components

Wheel Size	Item #1 Wheel Complete Pneumatic	Item #1 Wheel Complete w/ Flat Free	Item #2 Tire	Item #2 Tire w/ Flat Free	Item #3 Tube
14" x 3" For 4-Pole & GB Motors	1104438	1104437	1039652	1101754	1039651
14" x 4" For 4-Pole Motors	1042476	1042477	1042474	1042478	1042475
14" x 4" For GB Motors See Note(s) A1, A	1079305-NLA	1079304	1042474	1042478	1042475
<p>NOTE: A - The GB 14" x 4" wheel is used on Non-Heavy Duty GB motors only. The GB 14" x 4" wheel is no longer offered on-chair after 1/25/04 (3/14/04 for VA Government orders) and the GB Heavy Duty motors became the only GB motor choice - A1 - ON-Chair after 1/25/04 (3/14/04 for VA Government orders). See page titled "Drive Wheels For TrueTrack GB Heavy Duty TTHD Motors" for chairs manufactured after 1/25/04 (3/14/04 for VA Government orders).</p>					