



# Axles, Axle Bracket Hardware, and Anti-Tips Chairs without Hublocks

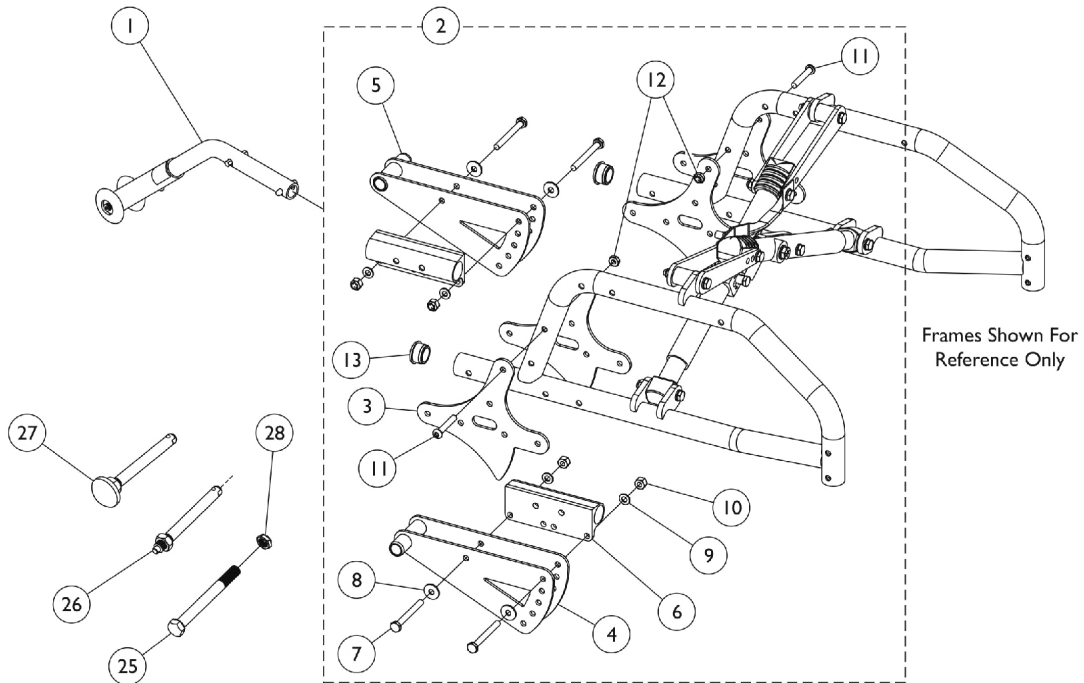

12" and 20" Rear Wheels

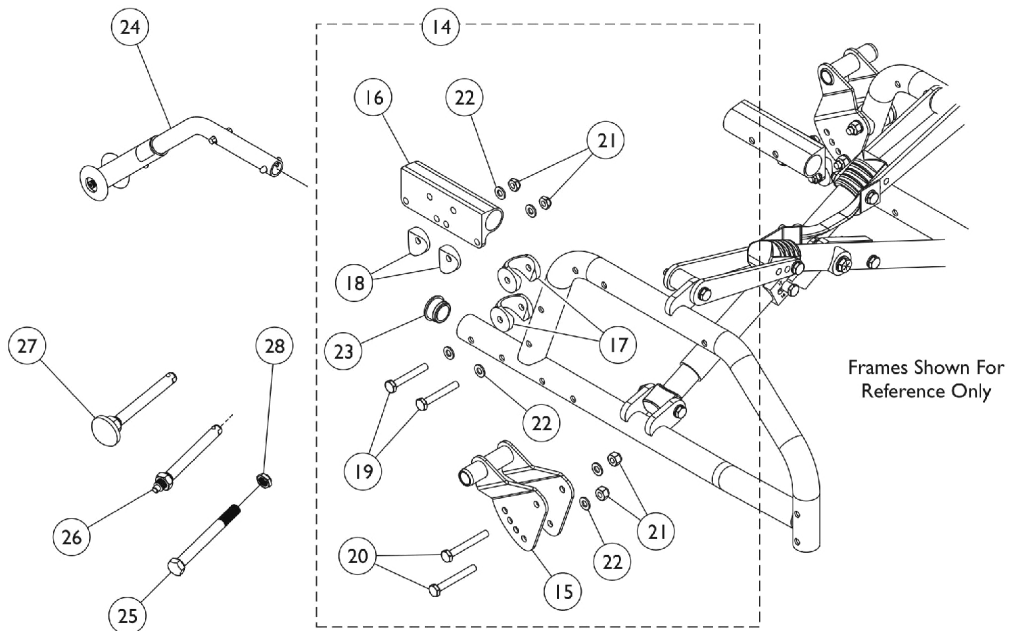

22" and 24" Rear Wheels

## Axles, Axle Bracket Hardware, and Anti-Tips Chairs without Hublocks

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	C	1118549	Anti-Tipper	1	2
2	A	1118553	Hardware, Axle Bracket, Both Sides (12" & 20" Rear Wheels)	1	1
3		1170354	Plate, Axle	1	4
4		1118487	Plate, Axle Tilt -Right	1	1
5		1118488	Plate, Axle Tilt -Left	1	1
6		1118500-NLA	NLA - Bracket, Anti-Tipper, Black	1	2
7		1055350	Screw, Hex Head (1/4-28 x 2-1/4")	1	4
8		1026900	Washer (1/4 x 23/32 x 1/16")	1	4
9		1026907	Washer (1/4 x 1/2 x 1/16")	1	4
10		1065362	Locknut (1/4-28)	1	4
11		1163325	Screw, Socket Button Head (1/4-28 x 1-1/2")	1	2
12		1158026	Locknut (1/4-28)	1	2
13		1056328	Plug Button, Black (1")	1	2
14	B	1118554	Hardware, Axle Bracket, Both Sides (22" & 24" Rear Wheels)	1	1
15	H	1118562-NLA	NLA - Plate, Axle Self Propel -Right	1	1
16		1118500-NLA	NLA - Bracket, Anti-Tipper, Black	1	2
17		1118469	Uni-Washer, Coved	1	4
18		40157X020	Spacer, Coved (1/4 x 1 x 9/16")	1	4
19		1055348	Screw, Hex Head (1/4-28 x 1-3/4")	1	4
20		1055349	Screw, Hex Head (1/4-28 x 2")	1	4
21		1065362	Locknut (1/4-28)	1	8
22		1026907	Washer (1/4 x 1/2 x 1/16")	1	12
23		1056328	Plug Button, Black (1")	1	2
24	D	1118560-NLA	NLA - Anti-Tipper	1	2
25	G	1049227	Screw, Hex Head (1/2-20 x 5-1/4")	1	2
26	F	1040361	Axle, Quick Release w/ Locknut (1/2" x 5")	1	2
27	E	1113049	Axle, Quick Release w/ Dome Button, Black	1	2
28		1025737	Locknut (1/2-20)	1	2

NOTE: A - Includes items 3 - 13.  
 B - Includes items 15 - 22.  
 C - Chairs with 12" and 20" Wheels.  
 D - Chairs with 22" and 24" Wheels.  
 E - Use with Spoke Rear Wheels.  
 F - Use with Composite Rear Wheels.  
 G - Permanent Axle. Use with Spoke/Composite Rear Wheels.  
 H - No longer Available. See item 14 as a replacement.