


Axles, Axle Bracket Hardware, and Anti-Tips Chairs with Hublocks

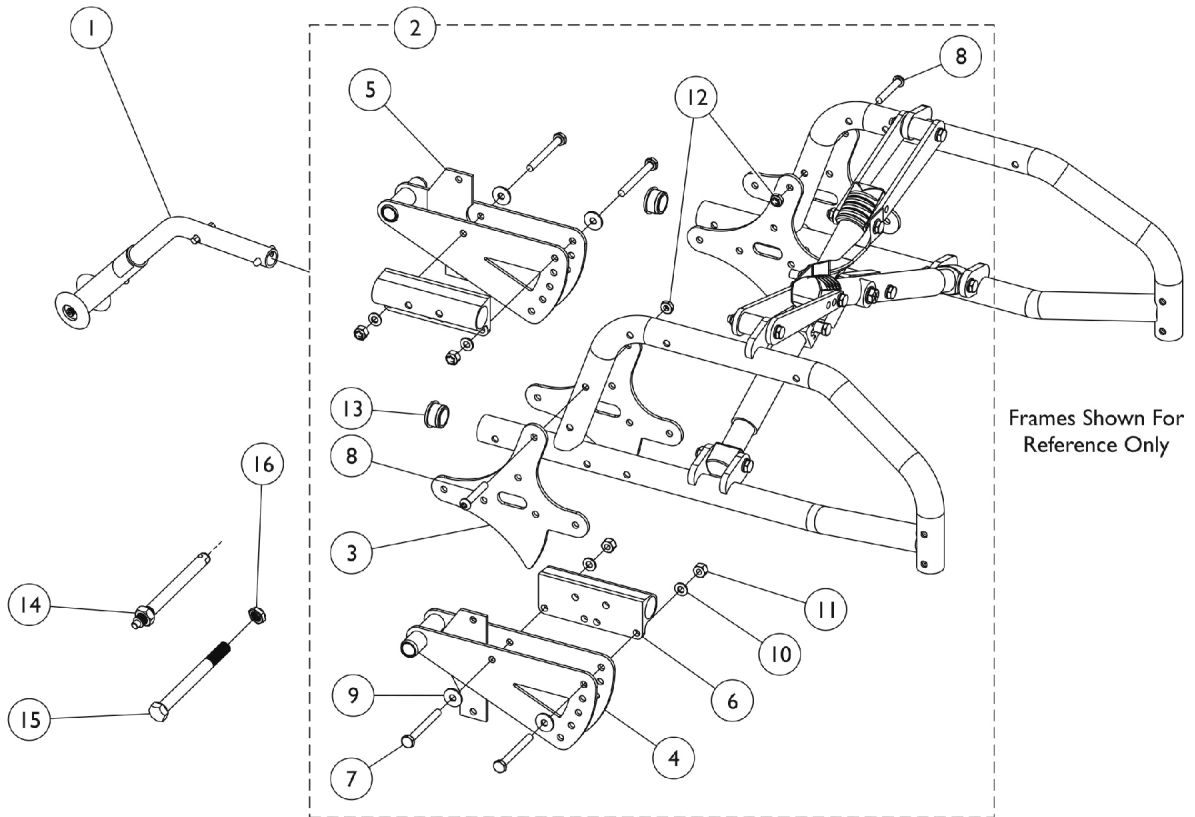

12" and 20" Composite Rear Wheels Only

Axles, Axle Bracket Hardware, and Anti-Tips Chairs with Hublocks

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1118549	Anti-Tipper	1	2
2	A,B	1132603	Kit, Axle Bracket Hublock Hardware	2	1
3		1170354	Plate, Axle	1	4
4		1132563-NLA	NLA - Plate, Axle Tilt Hublock - Right	1	1
5		1132564	Plate, Axle Tilt Hublock - Left	1	1
6		1118500-NLA	NLA - Bracket, Anti-Tipper, Black	1	2
7		1055350	Screw, Hex Head (1/4-28 x 2-1/4")	1	4
8		1163325	Screw, Socket Button Head (1/4-28 x 1-1/2")	1	2
9		1026900	Washer (1/4 x 23/32 x 1/16")	1	4
10		1026907	Washer (1/4 x 1/2 x 1/16")	1	4
11		1065362	Locknut (1/4-28)	1	4
12		1158026	Locknut (1/4-28)	1	2
13		1056328	Plug Button, Black (1")	1	2
14		1040361	Axle, Quick Release w/ Locknut (1/2" x 5")	1	2
15		1049227	Screw, Hex Head (1/2-20 x 5-1/4")	1	2
16		1025737	Locknut (1/2-20)	1	2

NOTE: A - Chairs with 12" and 20" Rear Wheels.
B - Includes items 2 - 13.