
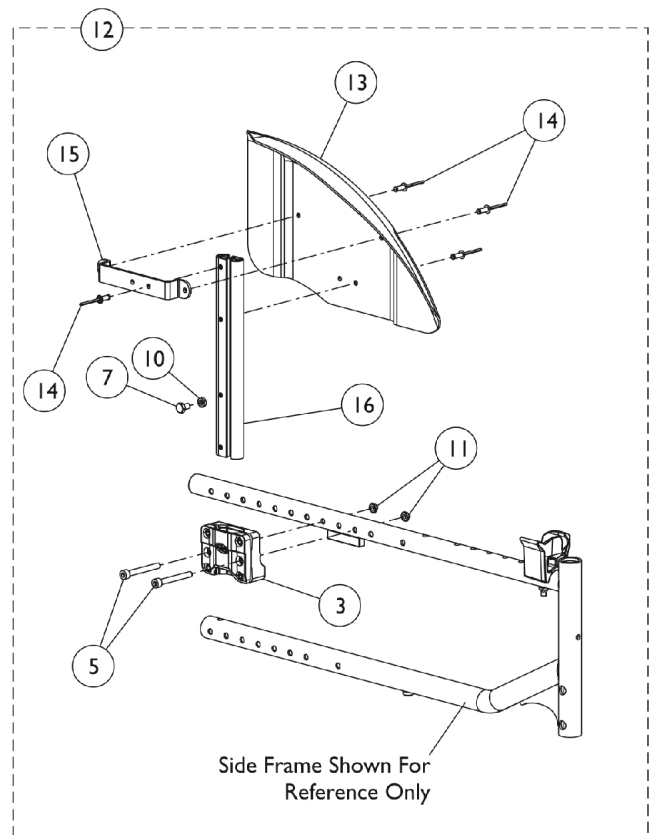


Arms - Clothing and Mudguards - Myon Jr.

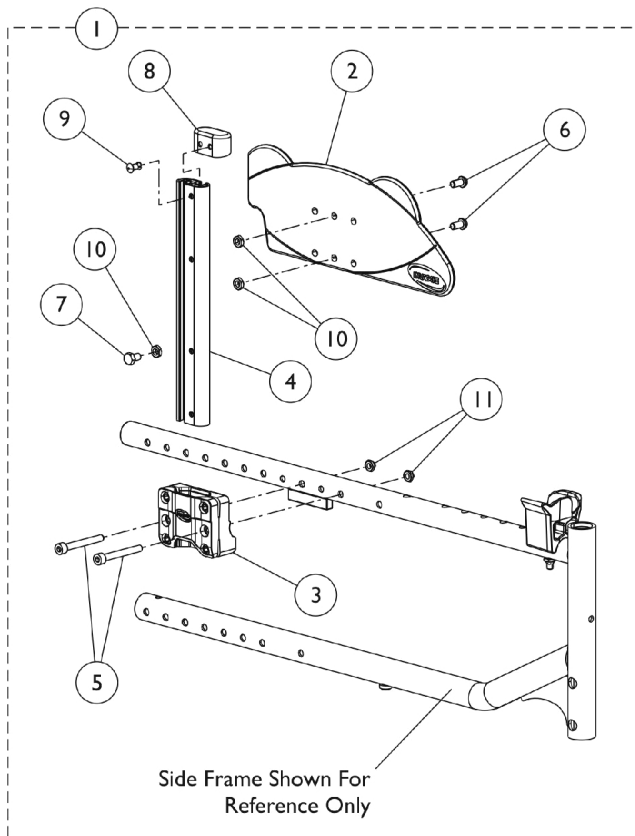

Arms - Clothing and Mudguards - Myon Jr.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1512647	Clothing Guard Assembly, Junior - Right	1	1
2			Guard, Clothing, Junior	1	1
3			NLA - Support, Armrest	1	1
4			Support, Side Guard	1	1
5			Screw, Socket Head (M6 x 50)	1	2
6			Screw, Socket Button Head (M6 x 1 x 12mm)	1	2
7			Screw (M6 x 1 x 10mm)	1	1
8			Cap, Guard Support	1	1
9			Rivet	1	1
10			Nut (M6)	1	3
11			NLA - Locknut (M6 x 1)	1	2
1	A	1513214	Clothing Guard Assembly, Junior - Left	1	1
2			Guard, Clothing, Junior	1	1
3			NLA - Support, Armrest	1	1
4			Support, Side Guard	1	1
5			Screw, Socket Head (M6 x 50)	1	2
6			Screw, Socket Button Head (M6 x 1 x 12mm)	1	2
7			Screw (M6 x 1 x 10mm)	1	1
8			Cap, Guard Support	1	1
9			Rivet	1	1
10			Nut (M6)	1	3
11			NLA - Locknut (M6 x 1)	1	2
12	C	1492738	Side Guard Assembly w/ Hardware, Black - Right	1	1
3			NLA - Support, Armrest	1	1
5			Screw, Socket Head (M6 x 50)	1	2
7			Screw (M6 x 1 x 10mm)	1	1
10			Nut (M6)	1	1
11			NLA - Locknut (M6 x 1)	1	2
13			Guard, Side - Right	1	1
14			Rivet (4.8mm x 12mm)	1	4
15			Plate, Reinforcing	1	1
16			Support, Side Guard	1	1
12	C	1492739	Side Guard Assembly w/ Hardware, Black - Left	1	1
3			NLA - Support, Armrest	1	1
5			Screw, Socket Head (M6 x 50)	1	2
7			Screw (M6 x 1 x 10mm)	1	1
10			Nut (M6)	1	1
11			NLA - Locknut (M6 x 1)	1	2
13			Guard, Side - Left	1	1
14			Rivet (4.8mm x 12mm)	1	4
15			Plate, Reinforcing	1	1
16			Support, Side Guard	1	1
19	C	1486041	Side Guard Assembly w/ Hardware, Forward - Right	1	1
3			NLA - Support, Armrest	1	1
5			Screw, Socket Head (M6 x 50)	1	2
7			Screw (M6 x 1 x 10mm)	1	1
10			Nut (M6)	1	1

Arms - Clothing and Mudguards - Myon Jr.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
11	C	1486042	NLA - Locknut (M6 x 1)	1	2
20			Guard, Side - Right	1	1
21			Rivet (4.8mm x 12mm)	1	4
22			Plate, Reinforcing	1	1
23			Support, Side Guard	1	1
19			Side Guard Assembly w/ Hardware, Forward - Left	1	1
3			NLA - Support, Armrest	1	1
5			Screw, Socket Head (M6 x 50)	1	2
7			Screw (M6 x 1 x 10mm)	1	1
10			Nut (M6)	1	1
11			NLA - Locknut (M6 x 1)	1	2
20			Guard, Side - Left	1	1
21			Rivet (4.8mm x 12mm)	1	4
22			Plate, Reinforcing	1	1
23			Support, Side Guard	1	1
NOTE: A - Includes items 2 - 11. C - Includes items 3, 5, 7, 10, 11, and 20 - 23.					