


Arm/Horn Assembly (After 7/1/10)

Model RPS350-1

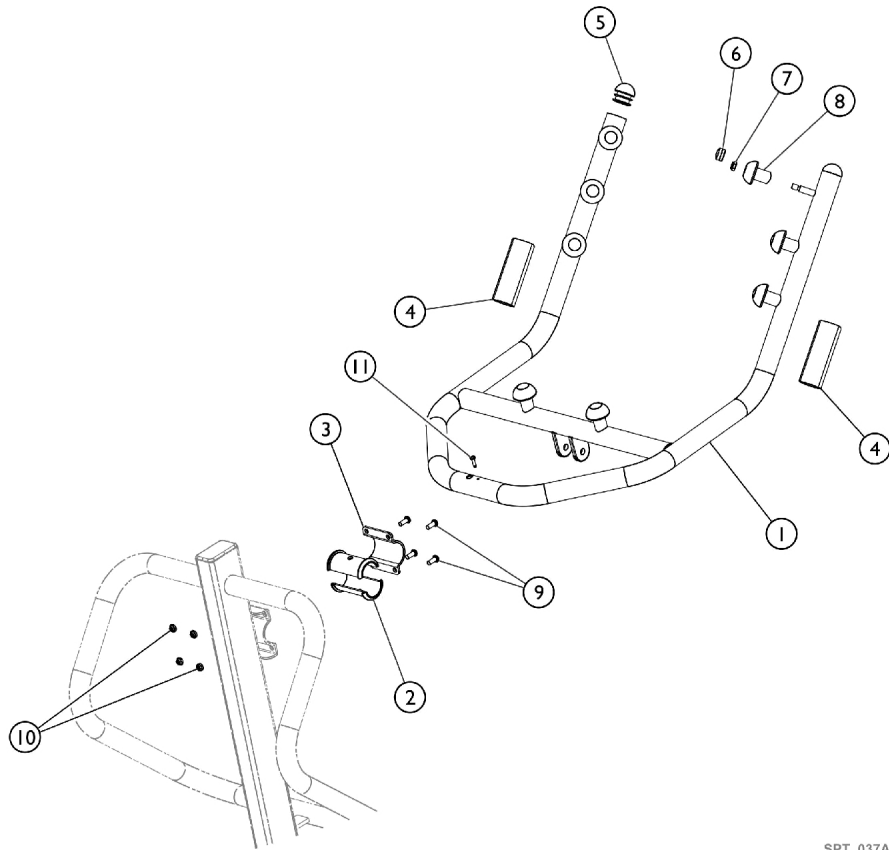

SPT_037A

Arm/Horn Assembly (After 7/1/10)

Model RPS350-1

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1156174	Arm/Horn Tube Assembly	1	1
2		1156173	Bushing, Horn	1	2
3		1071834	Bracket, Horn, Grey	1	1
4		1076388	Pad, Grip	1	2
5		1071249	Plug Button, Dome, Black (1-1/4")	1	2
6		1070902	Cap, Hook	1	8
7		1028769	Locknut (5/16-18)	1	8
8		1070177	Knob, Hook, Black	1	8
9		1007269	Screw, Socket Button Head (1/4-20 x 5/8")	1	4
10		1025195	Locknut (1/4-20)	1	4
11		12642X000	Screw (#8-18 x 5/8")	1	1