


Jasmine Full Body Lift - Mast/Boom Assembly

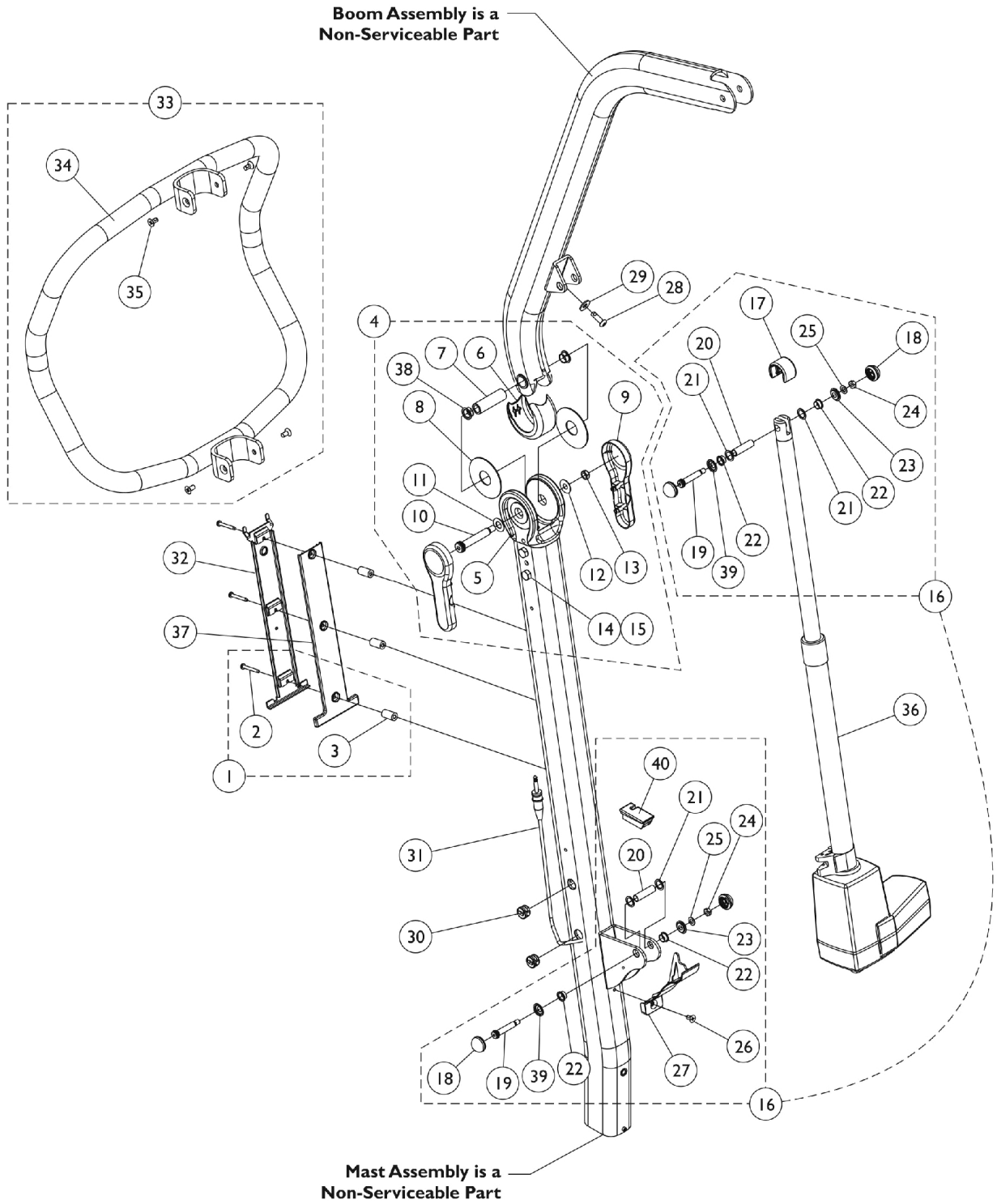

Jasmine Full Body Lift - Mast/Boom Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1161590	Kit, Battery Bracket Hardware	1	1
2			Screw, Phillips, Pan Head (10-32 x 1 1/2")	1	3
3			Spacer, Battery	1	3
4	B	1161579	Kit, Mast Connector	1	1
5			Mast Connector	1	1
6			Cover, Boom	1	1
7			Bushing (33/64 x 25/32 x 2-17/32")	1	1
8			Washer, Nylon, Boom and Mast	1	2
9			Cover, Mast	1	2
10			Screw, Hex Head (7/16-20 x 2 7/8")	1	2
11			Washer, Plain (1/2 x 1 x 1/32")	1	1
12			Washer, Plain (13/32 x 1 x 1/32")	1	1
13			Nut, Lock, Hex, Thin (3/8-16)	1	1
14			Locknut (7/16-20)	1	2
15			Screw, Shoulder Hex Socket Head (1/2 x 2-1/2", 3/8-16)	1	1
38			Bearing, Teflon	1	2
			Instruction Sheet, Jasmine Full Body Lift Mast	1	1
16	C	1169843-NLA	NLA - Kit, Mast/Boom Actuator Hardware	1	1
17			Cap, Actuator	1	1
18			Cap, Nut, Actuator, Rubber	1	4
19			Screw, Shoulder Hex Socket Head (3/8 x 1-11/16", 5/16-18)	1	2
20			Bushing (3/8 x 1/2 x 1-23/32")	1	2
21			Washer, Nylon (9/16 x 25/32 x 1/32")	1	4
22			Bearing, Flanged (1/2 x 9/16 x 9/32")	1	4
23			Washer, Screw, Actuator	1	2
24			Nut, Lock, Hex (5/16-18)	1	2
25			Washer, Flat (21/64 x 37/64 x 1/2")	1	2
26			Screw, Phillips, Flat Head (1/4-20 X 1/2")	1	1
27			Cover, Bottom, Actuator Plate	1	1
39			Washer, Nut, Actuator	1	2
40			Cover, Pinch Prevent	1	1
			Instruction Sheet, Jasmine Full Body Lift Mast	1	1
28		1155292	Screw, Socket, Button Head w/ Patch (M8 x 25mm)	1	1
29		1152387	Washer (11/32 x 7/8 x 5/64)	1	1
30		1161587-NLA	NLA - Package, Wire Lock	10	1
31	D	1161583	Kit, Controller Wire	1	1
			Cable, Base to Controller	1	1
			Wire, Brown, Motor Actuator	1	1
			Wire, Blue, Motor Actuator	1	1
32		1078284	Bracket, Battery - Electric Only	1	1
33	E	1161580	Kit, Push Handle, Blue	1	1
34			Push Handle Assembly, Blue	1	1
35			Screw, Phillips, Flat Head (1/4-20 X 1/2")	1	4
36		1148483	Motor, Actuator	1	1
37		1155348	Cover, Battery, Bracket	1	1

Jasmine Full Body Lift - Mast/Boom Assembly

NOTE: A - Includes Items 2 and 3.
B - Includes Items 5 thru 15 and 38. Instruction Sheet Not Shown on Illustration.
C - Includes Items 17 thru 27, 39, and 40.
D - Motor Actuator Wires not shown here. See Page titled "Jasmine Full Body Lift Base Assembly".
E - Includes Items 34 and 35.