




Caster Wheels and Mounting Hardware

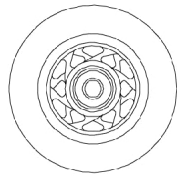

3" Urethane

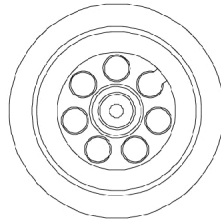

5" Aluminum Urethane

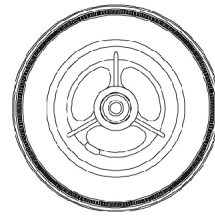



5" / 6" Composite Urethane

Caster Wheels and Mounting Hardware

Caster Wheels

Wheel Style	Wheel Style	Tire Type	Caster w/ Bearings	Bearing	Beaing Spacer	Tire	Tube
AT900	3"	Urethane	1032190-NLA	40117X026	1026172	N/A	N/A
5ALUM	5"	Urethane	1026952	40117X026	1026172	N/A	N/A
AT901	5"	Urethane	1029496	40117X026	1026172	N/A	N/A
AT902	6"	Urethane	1034352 See Note(s) A	N/A	N/A	N/A	N/A
6PNEU	6"	Pneumatic	1044040	40117X026	1026172	1026010-NLA	1026011-NLA

NOTE: A - Consists of (2) Casters.

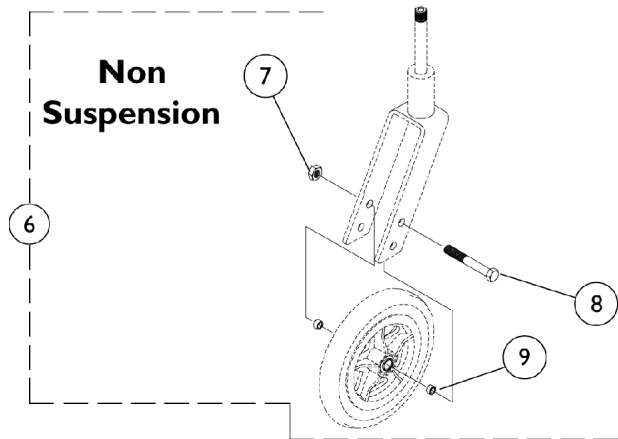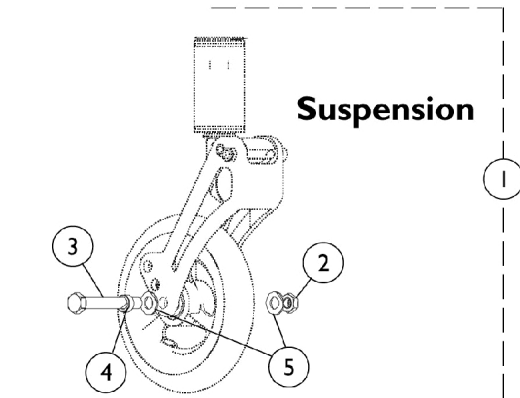

Caster Wheels and Mounting Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1098114-NLA	NLA - Hardware, Suspension Fork	1	1
2		1025730	Locknut (5/16-24)	1	1
3		1025764	Screw, Hex Head (5/16-24 x 2-1/4")	1	1
4		1028539	Spacer (15/32 x 5/16 x 3/16)	1	2
5		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	2
6	B	1026926-NLA	NLA - Hardware, Front Wheel	2	1
7		1025730	Locknut (5/16-24)	1	1
8		1026908	Screw, Hex Head (5/16-24 x 2-1/2")	2	1
9		1026160	Spacer (5/16 x 7/16 x 3/8")	4	1
	C	See Chart	Caster Wheels	1	1

NOTE: A - Includes items 2-5
 B - Includes items 7-9
 C - Not Shown