



Miscellaneous Accessories

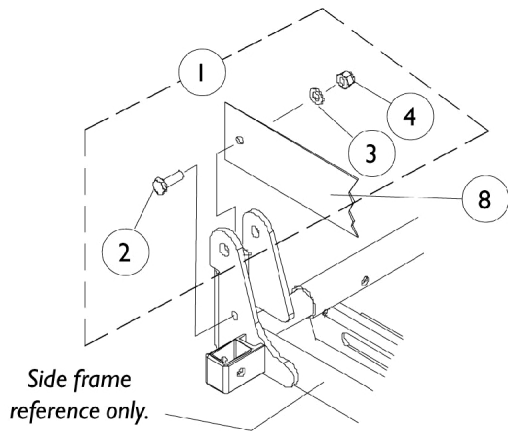

**Seat Positioning Strap
Attaching Hardware**

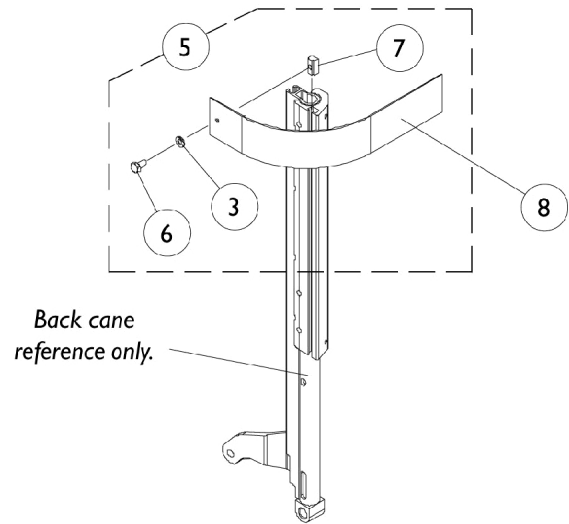

**Chest Positioning Strap
Attaching Hardware**

Miscellaneous Accessories

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	B	1091505-NLA	NLA - Hardware, Seat Positioning Strap Attaching (Pair)	2	1
2		1091870	Screw, Socket Button Head w/ Patch (1/4-20 x 3/4")	1	2
3		1026900	Washer (1/4 x 23/32 x 1/16")	1	2
4		7930333	Locknut (1/4-20)	25	1
5	C	1090212	Kit, Chest Positioning Strap Mounting Hardware For Conventional Seating	2	1
3			Washer (1/4 x 23/32 x 1/16")	1	2
6			Screw, Hex Head w/ Patch (1/4-20 x 1/2")	1	2
7			T-Nut (1/4-20)	1	2
8	E,F	1004310	Strap, Positioning, Seat/Chest w/ Auto Style Push Button Buckle (Eyelet to Eyelet Length 48")	1	1
8	E,G	1034625	Strap, Positioning, Seat/Chest w/ Auto Style Push Button Buckle (Eyelet to Eyelet Length 61")	1	1
8	E,F	TAGTS1034751-NLA	NLA - Strap, Positioning, Seat/Chest w/ Airline Style Buckle (Eyelet to Eyelet Length 46")	1	1
8	E,G,K	1034752-NLA	NLA - Strap, Positioning, Seat/Chest w/ Airline Style Buckle (Eyelet to Eyelet Length 61")	1	1
8	D,E	1034745	Strap, Positioning, Seat/Chest w/ Hook and Loop Style Buckle (Eyelet to Eyelet Length 61")	1	1
	A	0020200-NLA	NLA - Kit, Legrest with Screws, Otto Bock (6" W x 9" H)	1	1
	A	0020300-NLA	NLA - Adapter Plate, Otto Bock Legrest Pad	1	1
	A,I	0143900-NLA	NLA - Flap, Back Pan - Tarsys	1	1
	A,J	0144000-NLA	NLA - Flap, Back Pan - Tarsys	1	1
	A,G	0144100-NLA	NLA - Flap, Back Pan - Tarsys	1	1

NOTE: A - Not Shown
 B - Includes Items 2-4
 C - Includes Items 3, 6, and 7
 D - Chest Only
 E - Positioning strap attaching hardware not included.
 F - 16-19" Wide
 G - 20-22" Wide
 I - 16-17" Wide
 J - 18-19" Wide
 K - 1034752 Airline Buckle Style is No Longer Available. Use Auto Style part number 1034625 as a replacement.