


Caster Wheels & Mounting Hardware (Before 7/1/11)

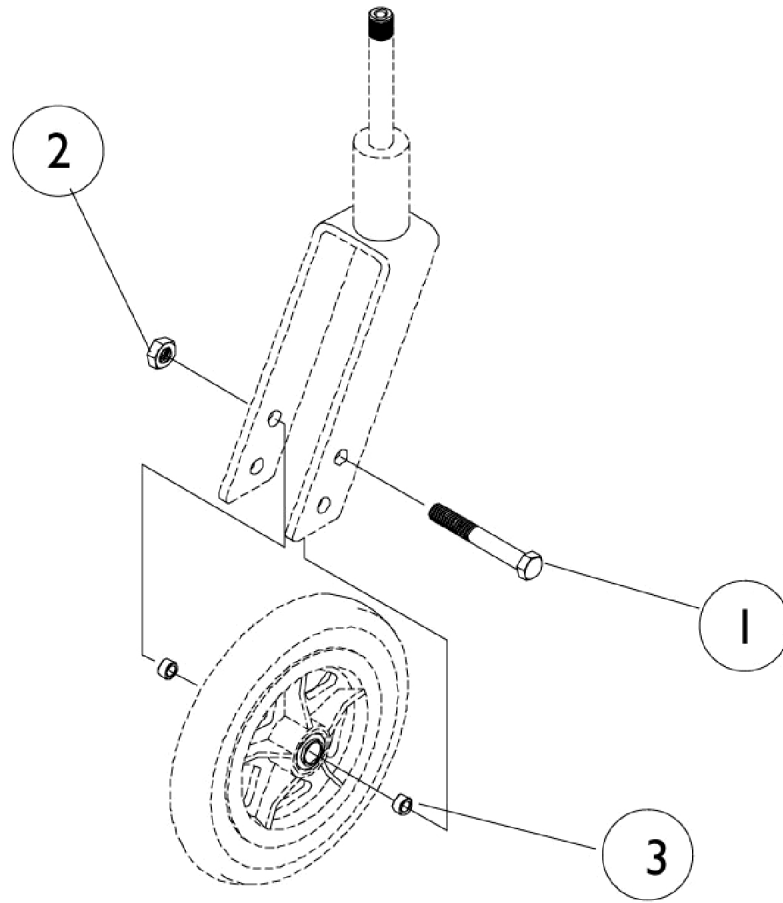

Caster Hardware

Wheel Style	Wheel Size	Tire Type	#1 Axle	#2 Axle Nut	#3 Axle Spacer
Composite	3"/5"	Urethane	1026908	7930330-NLA	1026102
Composite	6"	Urethane	1025764	7930330-NLA	1026102
Composite	6"	Semi-Pneumatic	1025764	7930330-NLA	1026102

Caster Wheels & Mounting Hardware (Before 7/1/11)

Caster Wheels

Wheel Style	Wheel Size	Tire Type	Bearing	Bearing Spacer	Caster Single #	Tire	Tube/Insert
					See Note(s) A		
AT900	3"	Urethane	40117X026	1026172	1032190-NLA	N/A	N/A
5ALUM	5"	Urethane	40117X026	1026172	1026952	N/A	N/A
AT901	5"	Urethane	40117X026	1026172	1029496	N/A	N/A
AT902	6"	Urethane	N/A	N/A	1034352 See Note(s) B	N/A	N/A
6SKYWAY	6"	Urethane	40117X026	1026172	1026956	N/A	N/A
6PNEU	6"	Pneumatic	40117X026	1026172	1044040	1026010-NLA	1026011-NLA

NOTE: A - Casters include Tire, Tube (if applicable), and Bearings.

B - Consists of (2) Casters.

6" Pneumatic Wheels with Flat-Free Inserts are No Longer Available (NLA). 6" Pneumatic Wheels are suggested as a replacement. It is recommended that they are ordered in pairs to match.