
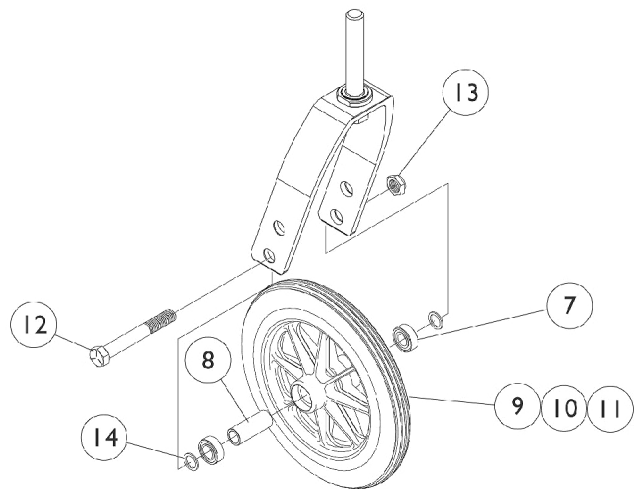


Casters - Front , Forks and Mounting Hardware

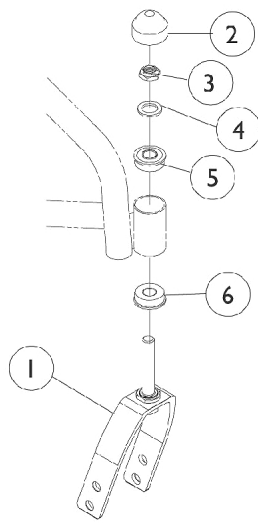

Casters - Front , Forks and Mounting Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1037420	Fork and Stem Assembly, Black (6 x 1" Caster)	1	1
1	B	1050138	Fork and Stem Assembly	1	1
1	C	1065589	Fork and Stem Assembly, Adult/Hemi	1	1
2		1001843	Cap, Head Tube, Black	1	1
2		1131189	Package, Head Tube Cap	10	1
3		1001979	Locknut (1/2-20)	1	1
4		1021508	Washer, Nylon (5/8 x 1 x 3/32")	1	1
5	D	1001801	Bearing, Flange (1/2")	1	2
5	E	1002626-NLA	NLA - Bearing	1	2
6	D	1085298	Bearing (1-1/4" O.D. x 1/2" I.D.)	1	2
6	E	1074930	Sleeve w/ Bearing, Black, Adult/Hemi	1	2
7-11		See Chart	#7-11 Caster Wheels	1	1
12-14		See Chart	#12-14 Caster Hardware	1	1
NOTE: A - 6 x 1" Caster B - 6 x 2" and All 8" Casters with Standard Duty Chairs. C - 6 x 2" and 8 x 1-3/4" Casters with Heavy Duty Chairs. D - Standard Duty Chairs E - Heavy Duty Chairs (250 - 350 Lbs.)					

Casters - Front , Forks and Mounting Hardware

#7-11 Caster Wheels

Wheel Style	Wheel Size	Tire Type	#7 Bearing	#8 Bearing Spacer	#9 Wheel w/ Bearings	#10 Tire	#11 Tube/Insert
AT902	6 x 1"	Urethane	N/A	N/A	1034352 See Note(s) C	N/A	N/A
1259 (Before 8/15/03) See Note(s) B, A	6 x 2" Deep Profile	Semi Pneumatic Urethane	1006049	1020592	1122066 See Note(s) C, B	N/A	N/A
1259 (After 8/14/03) See Note(s) B, A	6 x 2" Deep Profile	Semi Pneumatic Urethane	1006049	1020592	1115179	N/A	N/A
1299	8 x 1"	Solid Rubber	1006049	1018982	1058137	1040144	N/A
1228	8 x 1-3/4"	Semi-Pneu.	1006049	1020592	1070811	N/A	N/A
1292	8 x 1-1/4"	Pneumatic	1006049	1020592	1009481	40176X040	40176X041
1492	8 x 1-1/4"	Pneu. w/ Flat Free	1006049	1020592	1030773	40176X040	1010904-NLA
NOTE: A - Caster Style 1259, 6 x 2" caster uses an 8" fork. B - Due to product improvement, previous caster is NLA. Order (pair) part number 1122066 so casters will match. The deep profile caster has 2 tread grooves w/ round corners instead of 4 tread grooves w/ square corners. C - Consists of (2) Casters.							

#12-14 Caster Hardware

Wheel Style	Wheel Size	Tire Type	#12 Axle	#13 Locknut	#14 Axle Spacer
AT902	6 x 1"	Urethane	1025764	1025730	1026102
1259	6 x 2"	Semi-Pneumatic Urethane	1025726	1025734	1025724
1299	8 x 1"	Solid Rubber	1025726	1025734	1026101
1228	8 x 1-3/4"	Semi-Pneumatic	1025726	1025734	1025724
1292	8 x 1-1/4"	Pneumatic	1025726	1025734	1025724
1492	8 x 1-1/4"	Pneu. w/ Flat Free	1025726	1025734	1025724