


Casters and Mounting Hardware

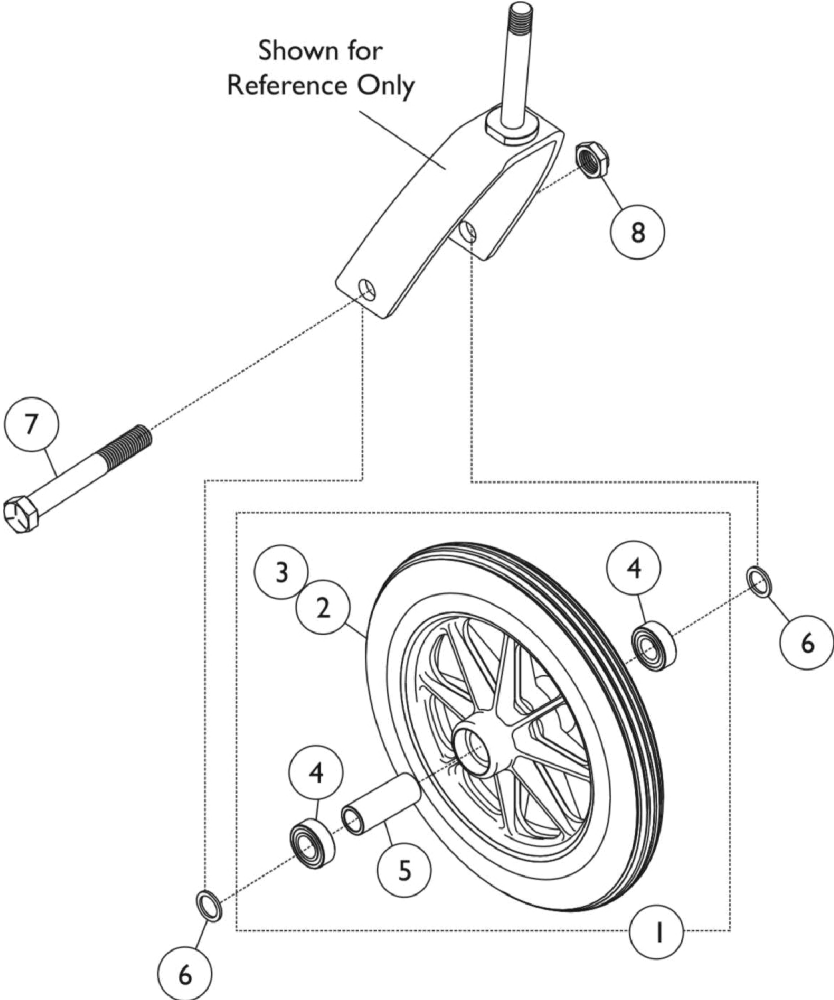

Casters and Mounting Hardware

Part Description	AT902 Composite Urethane Dark Grey (6 x 1")	AT903 Composite Urethane (8 x 1-1/4")	1228 Composite Semi-Pneumatic (8 x 1-3/4")	1252 Composite Pneumatic (8 x 1-1/4")
#1 Wheel Assembly	1034352 See Note(s) B, A	1058649 See Note(s) A	1070811 See Note(s) A	1041692 See Note(s) A
#2 Tire	N/A	1038157	N/A	40176X040
#3 Tube or Insert	N/A	N/A	N/A	40176X041
#4 Bearing	N/A	1010810	1006049	1010810
#5 Bearing Spacer	N/A	1020592	1020592	1020592
#6 Washer	1053202	N/A	1016032	1016032
#7 Axle	40317X002	57667P007	57667P007	57667P007
#8 Locknut	1107321	20225X000	20225X000	20225X000
NOTE: A - Includes items 2-5. B - Consists of (2) Casters.				

Casters and Mounting Hardware

Part Description	4252 Composite Pneumatic W/Flat Free (8 x 1-1/4")	1255 Composite Solid Rubber Dark Grey (8 x 1")	1228BK Composite Semi-Pneumatic Dark Grey (8 x 1-3/4")
#1 Wheel Assembly	1041688 See Note(s) A	1144243 See Note(s) A	1119532 See Note(s) A
#2 Tire	40176X040	Available thru Kit Only	N/A
#3 Tube or Insert	1010904-NLA	N/A	N/A
#4 Bearing	1010810	Available thru Kit Only	1006049
#5 Bearing Spacer	1020592	Available thru Kit Only	1020592
#6 Washer	1016032	N/A	1016032
#7 Axle	57667P007	57667P007	57667P007
#8 Locknut	20225X000	20225X000	20225X000
NOTE: A - Includes items 2-5.			