



Front Casters and Mounting Hardware

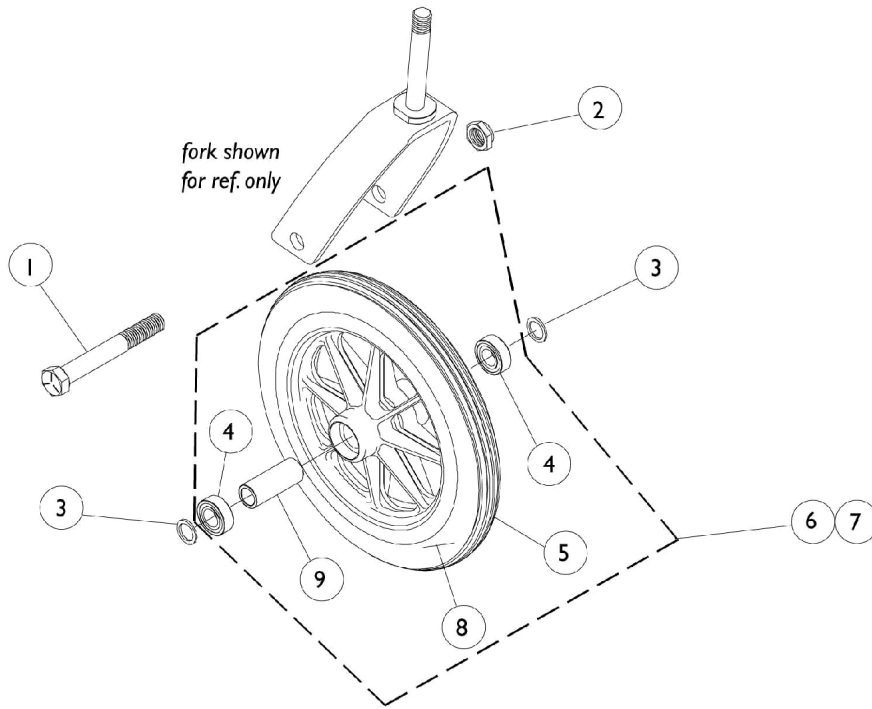

All 8" Casters

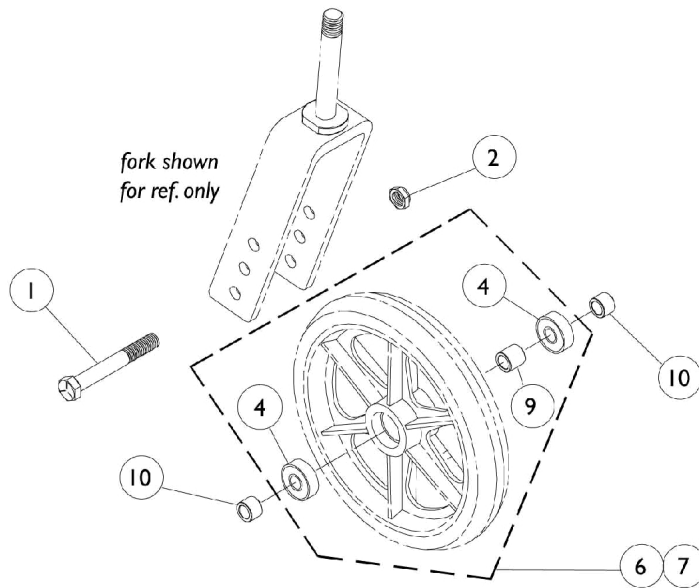

6" Caster

Front Casters and Mounting Hardware

Item Number	Description	1686 Composite Light Grey Urethane 6 X 1"	AT902 Composite Dark Grey Urethane 6 X 1"	1290 Composite Cast-Charcoal 8 X 1"	AT903 Composite Urethane 8 X 1-1/4"
1	Front Axle	40317X002	40317X002	57667P007	57667P007
2	Axle Locknut	60264X001	60264X001	20225X000	20225X000
3	Axle Washer	N/A	N/A	1026101	7930312
4	Bearing	40117X026	N/A	1006049	1040102
5	Tire	N/A	N/A	1075458 See Note(s) B	1038157
6	Wheel Assembly - 1pc	1026956 See Note(s) A	1029500	1086841	1058649 See Note(s) A
7	Wheel Assembly - 1pr	N/A	1034352	1086842	N/A
8	Tube/Insert	N/A	N/A	N/A	N/A
9	Hub Spacer	1026172	N/A	1018982	1037911
10	Axle Spacer	1053202	1053202	N/A	N/A
NOTE: A - Includes tire, tube/insert, bearings, and spacer.					
NOTE: B - Effective 1/14/05, tire color changed. Due to change, it is recommended tires be ordered in pairs.					
NOTE: #1686 and AT902 are not available for 9000XT Recliners.					

Front Casters and Mounting Hardware

Item Number	Description	1228 Composite Semi-Pneumatic After 6-9-97 8 X 1-3/4" See Note(s) A	1292 Composite Pneumatic 8 X 1-1/4"	1492 Composite Pneu. w/ Flat Free Insert 8 X 1-1/4"	1299 Composite Solid Rubber 8 X 1"
1	Front Axle	57667P007	57667P007	57667P007	57667P007
2	Axle Locknut	20225X000	1001979	1001979	1001979
3	Axle Washer	1016032	1016032	1016032	1026101
4	Bearing	1006049	1006049	1006049	1040102
5	Tire	--	40176X040	40176X040	1040144 See Note(s) C
6	Wheel Asm. - 1pc	1070811 See Note(s) B	1009481 See Note(s) B	1030773 See Note(s) B	1058648 See Note(s) B
7	Wheel Asm. - 1pr	1071666 See Note(s) B	--	--	--
8	Tube/Insert	--	40176X041	1010904-NLA	--
10	Axle Spacer	--	--	--	--

NOTE: A - Order in Pairs (#1071666) for chairs manufactured before 6-9-97.
 B - Includes tire, tube/insert, bearings, and spacer.
 C - Effective 1/14/05, tire color changed from grey to light grey. Due to change, it is recommended tires be ordered in pairs.